






research is the cure

Eurofarm Spa
Zona Industriale Piano Tavola 95032 Belpasso (CT) Italy
eurofarm-spa.com
igor.pinieri@eurofarm-spa.com
marketing@eurofarm-spa.com
phone: +39 095391346
fax: +39 095391507



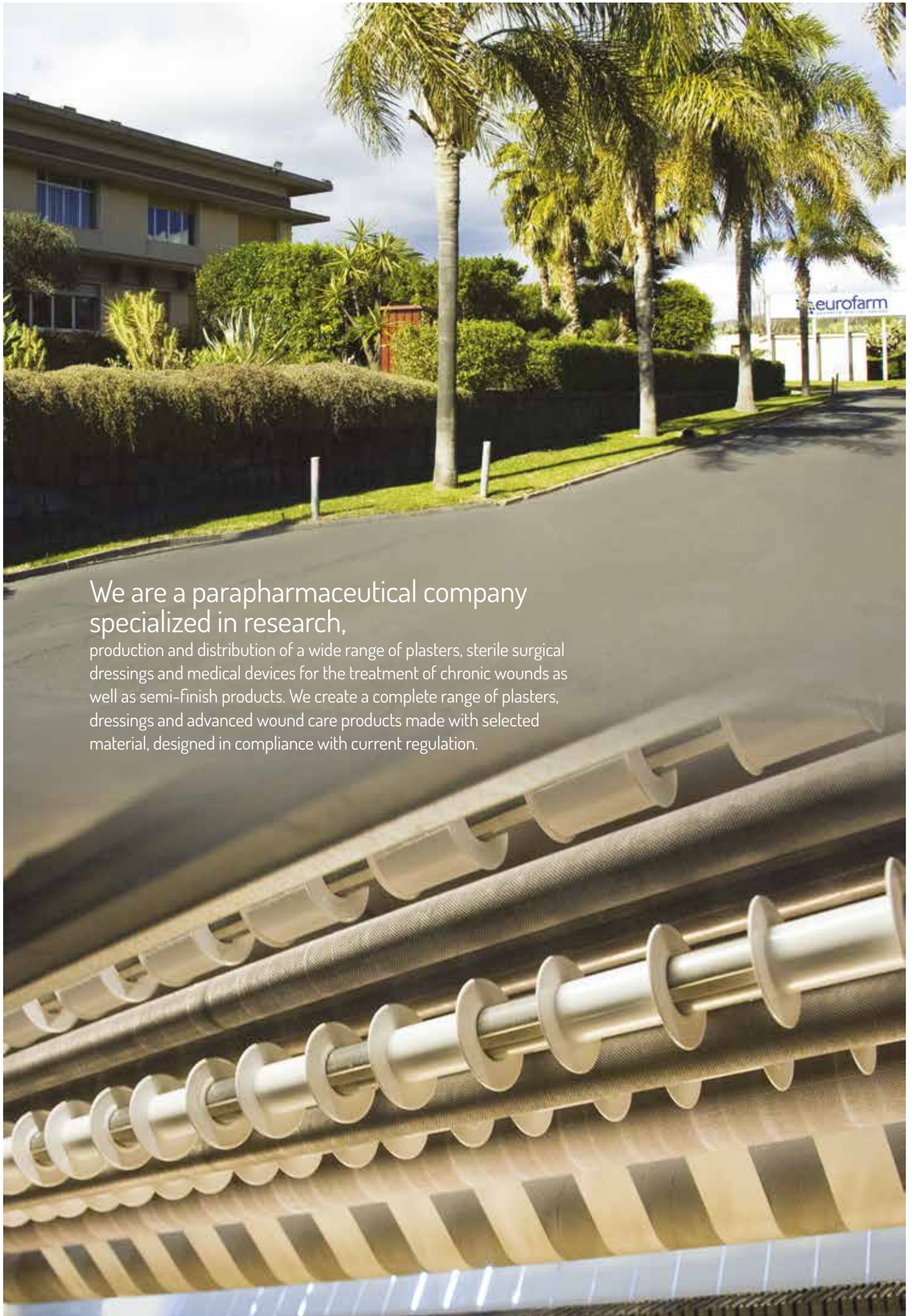

We are a parapharmaceutical company specialized in research, production and distribution of a wide range of plasters, sterile surgical dressings and medical devices for the treatment of chronic wounds as well as semi-finish products. We create a complete range of plasters, dressings and advanced wound care products made with selected material, designed in compliance with current regulation.

For over 40 years we are among the few production companies operating in Italy.

Experience, knowhow combined with a flexible manufacturing system is our value added to be competitive and to give effective-innovative answers for the treatment of skin lesions and chronic wounds to achieve a high level of patient satisfaction.

Our goal is primarily based on the organic growth of our existing brands and encourage the creativity and agility of our employees to expand into new, growth markets, reinforce our presence in mature markets and develop our distribution network and channels.



The company covers a floor area of 32,000 m²,

13,000 of which are covered by offices, storage, and controlled microclimate environments. A modern ISO 8 clean room of nearly 800 m², two coating lines, a quality control lab with advanced technology together with a sophisticated ethylene-oxide sterilisation plant ensure the complete integration of the production cycle, from raw materials to the sterile product.

The company is certified according to UNI EN ISO 9001 - 2015 and UNI EN ISO 13485 - 2016 regulations.



Eurofarm worldwide

- | | | | | |
|----------------------|-----------|-------------|--------------|----------------------|
| Austria | Germany | Lebanon | New Zealand | Slovenia |
| Belgium | Greece | Japan | Pakistan | South Africa |
| Bosnia - Herzegovina | India | Kuwait | Poland | Spain |
| Bulgaria | Indonesia | Madagascar | Portugal | Turkey |
| Czech Republic | Iran | Malta | Romania | Uganda |
| Cyprus | Ireland | Mexico | Saudi Arabia | United Arab Emirates |
| Congo | Kenya | Netherlands | Serbia | USA |
| Egypt | | | | Vietnam |





silver and aloe detergent spray

Vulnomed pag 7



dressings

Euromed pag 9
Euroderm plus pag 10
Eurometal pag 10
Eurodressing metal pag 11
Eurodressing pag 11



continuous length plasters

Long fix aloe pag 13
Long fix pag 13



plasters

Eurostrips aloe pag 15
new Eurostrips emo-stop pag 16
Eurostrips pag 16
Euroreel white pag 17
Euroreel skin pag 17
Europor pag 17
Eurosilk pag 17
Eurotrasp pag 18
new Enteralfix pag 18
Eurofix pag 19
Euroderm roll pag 19



eye plasters

Europtical occlusive pag 21
Europtical non occlusive pag 21



suture strips

Eurosuture pag 23



tracheostomy dressings

Tracheopad pag 25



advanced wound management

new Euronet paraffin pag 27
Hydromed pag 28
Hydromed gel pag 29
new Euroderm pag 30
Silvermed pag 31
Silvermed sb pag 31
Eurosorb pag 32
Eurosorb roll pag 33
Eurocell hydro pag 34
Eurocell hydro roll pag 35
Euroderm foam pag 36
Euroderm foam plus pag 37
Euroderm foam heel pag 38
Euroderm foam tracheo pag 39
Euroderm foam plus concave pag 40
Euroderm foam plus sacral pag 41
new Euroderm foam cavity pag 42
new Euroderm foam silicone pag 43



sports line

new Euroathletic KLS pag 45
new Euroathletic sport tape pag 46
Eurosave pag 47



bandages

Eurelasto L pag 49
Eurelasto pag 50
Eurelasto F pag 50
Eurozinc pag 51
Eurozinc elast pag 51
new Eurozinc alginate pag 52
new Eurozinc bielast pag 52
Flebimed short pag 53
Flebimed media pag 53
Flebimed forte pag 54
Flebimed bielast pag 54
Eurobandage pag 55
Eurocollar pag 55

before any dressing... vulnomed

VULNOMED is a sterile isotonic saline spray solution based on Silver Salt and Aloe Vera for the cleansing, irrigation and hydration of the skin. It is indicated for acute and chronic wounds, dermatological lesions of varying etiology, intact or injured skin surfaces. It allows a complete and thorough cleansing of the damaged skin and the neighboring area. It may be useful in the debridement of the crusty necrotic tissue.



Silver Salt prevents microbial contamination of the wound and keeps the environment free from exogenous bacteria.



Aloe Vera is well known for its healing, restorative and antiseptic properties.



VULNOMED®

Cleansing spray indicated for all wounds, sores, burns and rashes..

VULNOMED is a sterile isotonic saline solution made of silver salt and Aloe Vera indicated to clean, irrigate and hydrate the skin.



item no.	size	cont.	pkg.	code
----------	------	-------	------	------

501815

spray 125 ml

1 pcs.

12



8 057681 905607



dressings

Euromed

Euroderm plus

Eurometal

Eurodressing metal

Eurodressing

dressings

Euromed

Sterile Non-Woven, Post-operative adhesive island dressing with an absorbent central pad that does not stick to the wound thanks to a particular non-adherent layer. Breathable. Non-adherent and absorbent central pad.



 Fast and vertical absorbency

 Breathable

 Conformable

 Latex free

item no.	size	cont.	pkg.	code
891800	cm 5 x 7	5 pcs.	48	8 057681 905546
891801	cm 6 x 9	6 pcs.	48	8 057681 904969
891802	cm 10 x 8	6 pcs.	48	8 057681 904976
891804	cm 15 x 8	6 pcs.	48	8 057681 904990
891805	cm 20 x 10	4 pcs.	48	8 057681 905003
717807	cm 10 x 25	50 pcs.	1	8 059616 066940
717808	cm 10 x 30	50 pcs.	1	8 059616 066957

Features

- Individually packed in a peel away pouch and sterilized by E.O. gas.
- Soft breathable cover.
- Highly absorbent and adaptable pad coated with Hypoallergenic adhesive.

Benefits

- **Highly absorbent and low-adherent pad**
The Polyethylene wound contact layer enables the

passage of blood or exudates into the highly absorbent pad while maintaining a good healing environment and minimal pain upon dressing removal.

• **Safe and Secure Fixation**

The hypoallergenic acrylic adhesive is evenly spread onto the non-woven backing surface providing a safe and secure dressing adhesion over the wound site.

• **Soft and Adaptable**

Ensures patient comfort and adaptability to all

wound sites (even in the more difficult areas).

• **Patient Comfort**

The thick pad minimizes fluid break-through and helps to protect the wound from trauma.

Indications

- **Euromed**, adhesive wound dressing is ideally suited for post-operative use. They are also useful in accident & emergency for cuts, lacerations and sutured wounds.

Euroderm plus

Waterproof, sterile post-operative adhesive island dressing made of a highly breathable, bacteria barrier and transparent polyurethane film with an absorbent and non-adherent central pad.

Description

- Transparent protective wound dressing for light to moderately secreting wounds.
- Made of polyurethane bacteria proof, virus proof and waterproof film coated with hypoallergenic adhesive.
- The polyurethane film backing paper is kiss cut into an outer frame and central window for easy application.
- Ethylene oxide gas sterilized, supplied with a central non-adherent pad.

Benefits

- Maintains the undisturbed state of the wound.
- Very easy to apply.
- Provides moist wound environment to enhance healing and decrease pain.
- Patient can shower without risk.
- PU Frame style allows wounds to breathe and water vapour to escape protecting skin against maceration where dressing can be left in place for 5 days minimizing interference at wound site.

Indication

- Wound dressing for primary and secondary care of light to moderately secreting wounds.
- Fixation of I.V. catheters.
- Antibacterial and waterproof coverage of wounds.
- Post-operative dressings to protect clean closed wounds.



Low exudate



Breathable



Acts as bacteria barrier and Waterproof



Dermatologically tested



Latex free

item no.	size	cont.	pkg.	code
897600	cm 5 x 7	5 pcs.	48	8 057681 908400
897602	cm 6 x 9	5 pcs.	24	8 057681 908424
897603	cm 10 x 10	5 pcs.	24	8 057681 908431
897605	cm 15 x 8	5 pcs.	24	8 057681 908455
897607	cm 10 x 25	3 pcs.	24	8 057681 908479

Eurometal

Sterile post-operative adhesive island dressing with a non-adherent and absorbent aluminised central pad. The aluminised central pad facilitates a fast healing process.



item no.	size	cont.	pkg.	code
891901	cm 12 x 7	6 pcs.	12	8 057681 905034
891902	cm 15 x 8	6 pcs.	12	8 057681 905041



Fast and vertical absorbency



Latex free



Breathable



Promotes healing



Conformable



Dermatologically tested




Eurodressing metal

Sterile, non-adherent and absorbent dressing with aluminium surface that promotes a quicker healing. It is indicated for superficial wounds, in postoperative treatment; first aid (abrasions, cuts); light to moderately exuding ulcerations; minor burns; and under compression.



item no. size cont. pkg. code

892113 cm 10 x 10 6 pcs. 12

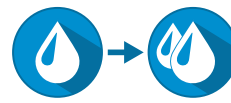

Low exudate



Moderate exudate



Promotes healing



Breathable



Latex free


Eurodressing

Sterile non-adherent absorbent dressing, indicated for superficial wounds, in postoperative treatment; first aid (abrasions, cuts); light to moderately exuding ulcerations; minor burns; and under compression.



item no. size cont. pkg. code

892103 cm 10 x 10 6 pcs. 12

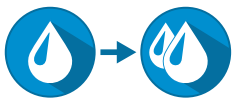

Low exudate



Moderate exudate



Hypo-allergenic



Breathable



Latex free





continuous length plasters

Long fix aloe

Long fix

continuous length plasters

Long fix aloe



Adhesive continuous length plaster with an Aloe Vera absorbent central pad. Suitable for sensitive skins. Any minor wound needs a suitable plaster to help the healing process. Aloe Vera pad technology promote healing and is capable of performing an exceptional epithelizing and antiseptic action.

item no.	size	cont.	pkg.	code
861002	m 0,5 x cm 6	1 pcs.	48	8 057681 909476
861003	m 0,5 x cm 8	1 pcs.	48	8 057681 909483
861005	m 1 x cm 6	1 pcs.	48	8 057681 909506
861006	m 1 x cm 8	1 pcs.	48	8 057681 909513

Long fix



Continuous length adhesive plaster in white non-woven with a central, non-adherent, absorbent wound-pad. Skin friendly and highly breathable. The plaster can be cut in one's favourite size. Suitable for minor wounds and general abrasions.

item no.	size	cont.	pkg.	code
740002	m 0,5 x cm 6	1 pcs.	24	8 057681 01364
740003	m 0,5 x cm 8	1 pcs.	24	8 057681 901371
740005	m 1 x cm 6	1 pcs.	24	8 057681 901395
740006	m 1 x cm 8	1 pcs.	24	8 057681 901401

Latex free



Low exudate



Promotes healing





plasters

Eurostrips aloe

Eurostrips emo-stop

Eurostrips

Euroreel white

Euroreel skin

Europor

Eurosilk

Eurotrasp

Enteralfix

Eurofix

Euroderm roll



plasters

Eurostrips aloe

Polyethylene plaster with an Aloe Vera absorbent central pad. Suitable for sensitive skins. Any minor wound needs a suitable plaster to help the healing process. Aloe Vera pad technology promote healing and is capable of performing an exceptional epithelizing and antiseptic action. It is indicated for for first aid, medication, wellness, for the cure of small wounds and for fingers or other parts of the body.



With Aloe



Antiseptic



Promotes healing

item no.	size	cont.	pkg.	code
826051	medium	20 pcs.	72	8 057681 909407
826056	medium/large	20 pcs.	72	8 057681 909773
826055	assorted	20 pcs.	72	8 057681 909766



new

Eurostrips emo-stop

Hemostatic pressure bandages with a central pad composed by super-absorbent fibers that rapidly increase its volume immediately after blood absorption and should therefore effect a compressive action on the wound with fast stop of the bleeding. Eurostrips emo-stop is flexible, hypoallergenic, elastic and breathable.



item no.	size	cont.	pkg.	code
----------	------	-------	------	------

826283 mm 31x72 12 pcs. 72



Hemostatic



Latex free



Hypo-allergenic



Breathable



Elastic



Eurostrips

Breathable, sterile, water resistant polyethylene plasters. The elastic support provides optimal conformability.

item no.	size	cont.	pkg.	code
----------	------	-------	------	------

826002 medium 24 pcs. 72



826003 medium/large 20 pcs. 72



826021 assorted 20 pcs. 72



Latex free



Breathable



Elastic



Hypo-allergenic



Euroreel white white cotton tape

100% White cotton adhesive surgical tape coated with a zinc-oxide adhesive mass into micrometrical spaced lines. It is air permeable, skin-respectful and highly hypoallergenic. Suitable for the fixation of medical devices



Latex free

item no.	size	cont.	pkg.	code
801011	m 5 x cm 1,25	1 pcs.	36	8 057681 903047
801012	m 5 x cm 2,5	1 pcs.	24	8 057681 903054
801015	m 5 x cm 5	1 pcs.	12	8 057681 903061
801020	m 5 x cm 10	1 pcs.	6	8 057681 903085

Euroreel skin skin cotton tape

100% Skin cotton adhesive surgical tape coated with a zinc-oxide adhesive mass into micrometrical spaced lines. It is air permeable, skin-respectful and highly hypoallergenic. Suitable for the fixation of medical devices.



Latex free

item no.	size	cont.	pkg.	code
802011	m 5 x cm 1,25	1 pcs.	36	8 057681 903122
802012	m 5 x cm 2,5	1 pcs.	24	8 057681 903139
802015	m 5 x cm 5	1 pcs.	12	8 057681 903146

Europor non woven tape

Non-woven porous adhesive plaster, breathable, radio transparent, hypoallergenic. Suitable for the fixation of medical devices.



Latex free



Delicate skin

item no.	size	cont.	pkg.	code
807001	m 5 x cm 1,25	1 pcs.	36	8 057681 903290
807002	m 5 x cm 2,5	1 pcs.	24	8 057681 903306
807005	m 5 x cm 5	1 pcs.	12	8 057681 903313

Eurosilk silk tape

100% Acetate adhesive silky appearance surgical plaster, breathable, strong and non-elastic, hypoallergenic, easy to tear in both directions by hands, particularly suitable on sensitive skins.



Latex free



Delicate skin

item no.	size	cont.	pkg.	code
809001	m 5 x cm 1,25	1 pcs.	36	8 057681 903399
809002	m 5 x cm 2,5	1 pcs.	24	8 057681 903405
809005	m 5 x cm 5	1 pcs.	12	8 057681 903412

Eurotrasp trasparente polyethylene tape

Transparent porous plaster in polyethylene. It is quick and easy to tear (even when using surgical gloves) both in transversal and longitudinal way, as well as in the middle. Permeable to air and to water vapour but impermeable to liquids.

item no.	size	cont.	pkg.	code
812101	m 5 x cm 1,25	1 pcs.	36	8 057681 909186
812102	m 5 x cm 2,5	1 pcs.	24	8 057681 909193
812105	m 5 x cm 5	1 pcs.	12	8 057681 909209



Latex free

new



Enteralfix nasogastric plasters for nose tube fixing

Sterile non woven plaster for fixation of gastric nose tubes for enteral nutrition. The high elasticity allows easy application and high conformability. Enteralfix has excellent adhesion properties resulting in long lasting wear time without causing pain on removal, from even the most delicate skin. No residue is left after removal. Skin friendly. Air and water vapor permeable. Radiotranslucent. Enteralfix is latex free.

item no.	size	cont.	pkg.	code
724551	cm 6,2 x 5	100 pz	2	8 057018 290635



Hypo-allergenic



Latex free



Delicate skin

Eurofix



Elastic polyester net fixation plaster. It is very soft, skin respectful and highly hypoallergenic. Allows a comfortable fit of the plaster to the skin and to the other particularly irregular anatomical parts of the body, ensuring the highest moisture permeability. The adhesive is not weakened by water nor secretions. Precut blue squared printed paper protects the adhesive side. Designed for an easy fixation of primary dressings.

item no.	size	cont.	pkg.	code
891702	m 5 x cm 5	1 pcs.	24	8 057681 904938
891704	m 10 x cm 5	1 pcs.	12	8 057018 291274
891705	m 10 x cm 10	1 pcs.	12	8 057018 291267
891706	m 10 x cm 15	1 pcs.	12	8 057018 291250
715004	m 10 x cm 20	1 pcs.	12	8 059616 065608
715006	m 10 x cm 30	1 pcs.	12	8 059616 065622

Breathable



Conformable



Delicate skin



Latex free



Euroderm roll



Transparent, waterproof and highly breathable adhesive Polyurethane film plaster, flexible, conformable to irregular parts of the body, easy to cut in any required size. Designed for an easy fixation of primary dressings and allows the skin to breath. Waterproof. It offers the patient the opportunity to perform regular and daily hygiene activities. The transparent support allows continuous monitoring of the wound, limiting the need of dressing change.

item no.	size	cont.	pkg.	code
715601	m 10 x cm 5	1 pcs.	12	8 059616 065936
715602	m 10 x cm 10	1 pcs.	12	8 059616 065943
715603	m 10 x cm 15	1 pcs.	12	8 059616 065950

Breathable



Conformable



Latex free





eye plasters

Euoptical occlusive

Euoptical non occlusive

eye plasters

Europtical occlusive

Sterile eye plaster, ready for use, with an occlusive pad, ideal for the correction of strabismus and for the prophylaxis and treatment of amblyopia. The highly porous non-woven backing material preserves the physiological functions of the skin and reduces the risk of maceration. Besides, the high permeability both to air and to water vapour avoids heat congestion in the eye socket. Its anatomical shape enables easy and accurate applications, even when worn beneath glasses.



Latex free



Ready to use



item no.	size	cont.	pkg.	code
892300	cm 5,2 x 6,4	6 pcs.	24	8 057681 908295
892301	cm 5,7 x 8,2	7 pcs.	24	8 057681 905317
717200	cm 5,2 x 6,4	50 pcs.	1	8 059616 066254

Europtical non occlusive

Sterile eye plaster with a white, absorbent, non-occlusive pad. It is ideal for the post-operative protection of the eye. The highly porous non-woven backing material preserves the physiological functions of the skin and reduces the risk of maceration. Besides, the high permeability both to air and to water vapour avoids heat congestion in the eye socket. Its anatomical shape enables easy and accurate applications, even when worn beneath glasses.



item no.	size	cont.	pkg.	code
892351	cm 5,2 x 6,4	6 pcs.	24	8 057681 908301
892352	cm 5,7 x 8,2	6 pcs.	24	8 057681 905577
717210	cm 5,2 x 6,4	50 pcs.	1	8 059616 066285



suture strips

Eurosuture

suture strips

Eurosuture



Sterile suture strips provides maximum dynamic adherence and superior wound security. It is made of macroporous non-woven polyamide with a pressure sensitive adhesive. The proprietary materials allow Eurosuture to have a perfect balance between elasticity and adherence, closely mimicking the elastic modulus of human skin. Eurosuture are easy to apply. Their unique rounded ends resist curl and premature lifting.

item no.	size	cont.	pkg.	code
817410	mm 3 x 75	10 pcs.	24	8 057681 908646
817420	mm 6 x 38	12 pcs.	24	8 057681 908653
817430	mm 6 x 75	6 pcs.	24	8 057681 908660
817450	mm 13 x 100	12 pcs.	24	8 057681 908684

Benefits

- Breathable, porous, elastic backing can be used on contoured body areas or where bending and movement occur.
- Faded colour for a cosmetically pleasing appearance.
- Eliminates puncture marks for better cosmetic result than sutures or staples. Fast and easy to apply to save time.

- Hypoallergenic adhesive.
- Available in a variety of sizes to meet your needs.
- Rounded edges help prevent roll-up and premature lifting.
- Available in sealed pouches E.O sterilized.

User

- For use in highly contoured areas or body areas of musculoskeletal movements like joints.
- Providing wound support following early suture or staple removal.
- Useful for wound support post operatively and following suture removal.

For use in

- Emergency Rooms.
- Operating Rooms.
- Clinics and physician's offices.



Latex free



Hypo-allergenic




Dermatologically tested





tracheostomy dressing

Tracheopad

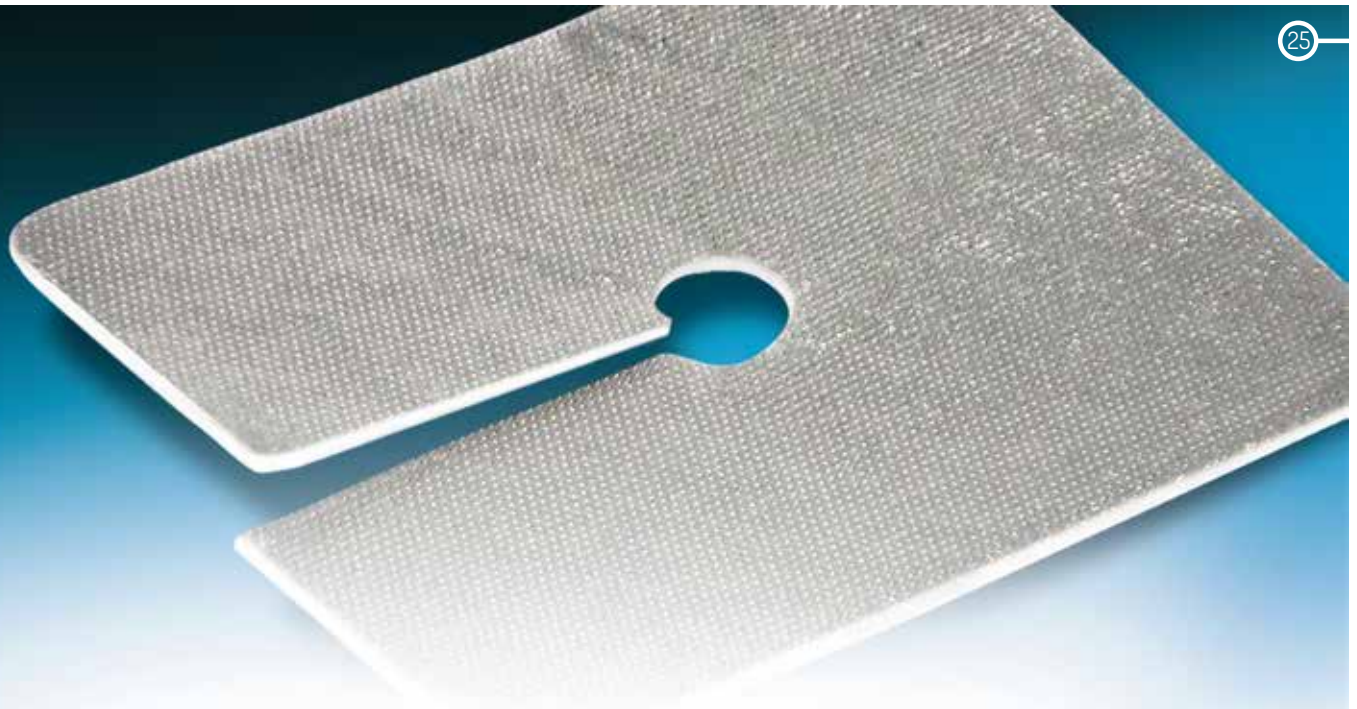

tracheostomy dressing

Tracheopad

Sterile, non-adherent and absorbent Aluminized dressing, especially designed for use around tracheotomies, nephrostomies and large calibre drains. The excellent quality of the aluminized wound contact layer reduces the adherence of the dressing neither to the peristome skin, nor to the tracheotomy tube and minimises the trauma for the patient during the removal.

Furthermore, the high porosity of this layer permits good skin transpiration, permits a rapid absorption of blood and exudates into the dressing and consequently keeps the wound surface dry.

Tracheopad acts as a reliable barrier against bacterial contamination.



Acts as bacteria barrier and Waterproof



Low exudate Moderate exudate

item no. size cont. pkg. code

717320

cm 8 x 9,3

50 pcs.

2



8 057681 908110



advanced wound management

Euronet paraffin

Hydromed

Hydromed gel

Euroderm

Silvermed

Silvermed sb

Eurosorb

Eurosorb roll

Eurocell hydro

Eurocell hydro roll

Euroderm foam

Euroderm foam plus

Euroderm foam heel

Euroderm foam tracheo

Euroderm foam plus concave

Euroderm foam plus sacral

Euroderm foam silicone

Euroderm foam cavity

advanced wound management

Euronet paraffin new

Euronet Paraffin is a 100 % absorbent cotton thread gauze, impregnated with white soft paraffin ointment. The gauze makes it a non-adherent and non-allergenic dressing. Euronet Paraffin is used as a primary wound contact layer in the treatment of burns, ulcers, skin grafts and a variety of traumatic injuries. The material is also used as a transfer medium for skin during grafting. Euronet Paraffin is latex free



Moderate
exudate



Heavy
exudate



Latex free

item no.	size	cont.	pkg.	code
560001	cm 5 x 5	10 pcs.	10	8 057018 290260
560110	cm 10 x 10	10 pcs.	10	8 057018 290338
560020	cm 20 x 7,5	10 pcs.	10	8 057018 290321


Indications

- Euronet Paraffin is used as a primary wound contact layer in the treatment of burns, ulcers, skin grafts and a variety of traumatic injuries. The material is also used as a transfer medium for skin during grafting.

- Under the medical supervision of a healthcare professional, Euronet Paraffin can be used to manage infected wounds or wounds in which there is an increased risk of infection.



Contraindications

- Although there are no contra-indications to the use of Euronet Paraffin, if the dressing is placed upon a heavily exuding wound its semi-occlusive nature may cause tissue maceration by preventing the free movement of exudates away from the surface of the wound.

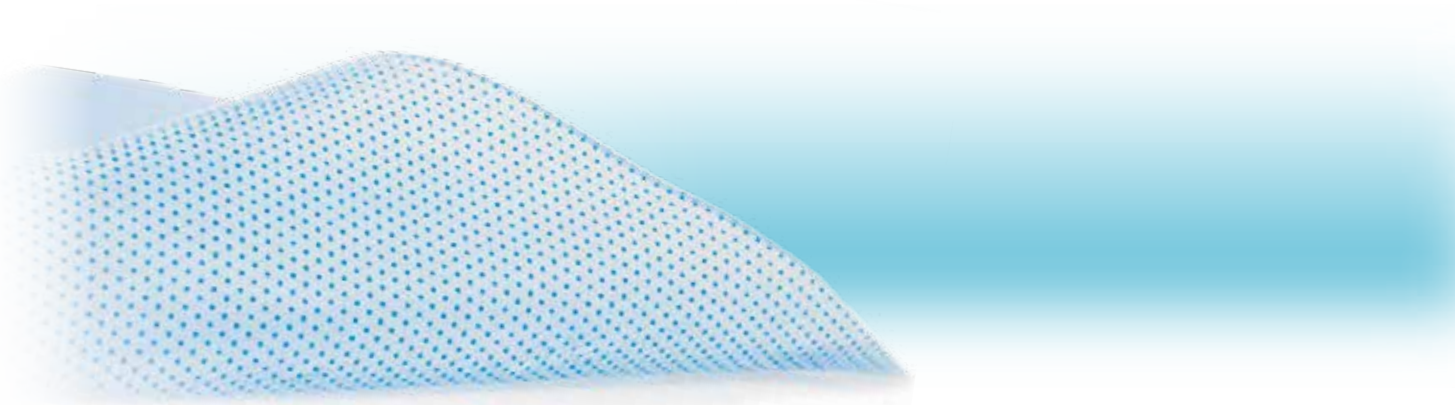

Hydromed

Hydrogel dressing designed to manage pain, moisture, and protease activity in low exudate or sloughy wounds as absorbs quickly and locally. This reduces the risk of periwound maceration. Suitable for: 1st and 2nd degree burns, abrasions, dermabrasions, radiodermatitis, torpid wounds and ulcers with fibrinous or necrotic bases. Refreshing.

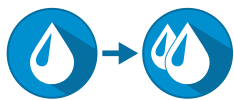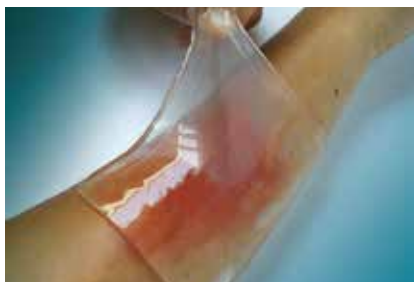

Low exudate Moderate exudate



Refreshing



Designed to remove odours resulting from necrotic tissue



Latex free

item no.	size	cont.	pkg.	code
892011	cm 6,5 x 5	5 pcs.	10	8 057681 905065
892013	cm 10 x 10	5 pcs.	10	8 057681 905072
892015	cm 15 x 15	5 pcs.	10	8 057681 905089
892017	cm 20 x 20	5 pcs.	10	8 057681 905096

Unlike typical Sheet Hydrogels, HYDROMED absorbs quickly and locally. This reduces the risk of periwound maceration.

Indications

- Suitable for treatment of chronic and slow healing wounds in granulation epithelialization phase.
- Due to a three dimensional structure and high water content HYDROMED is suitable for use in debriding and for second-degree burns.

Contraindications

Hydromed should not be used on infected wounds nor on third degree burns. Also they should not be used on highly exuding wounds nor on exposed muscle, bones nor tendons.

Hydromed gel

Sterile, transparent amorphous gel suitable for the removal of necrotic tissue. The gel creates a moist wound environment, preventing eschar formation and encouraging healing.

By keeping fragile granulation tissue moist, the gel helps to create a moist wound healing environment, allowing the migration of epithelializing cells.



Low exudate



Hydrating



Latex free

item no. size cont. pkg. code

501705

Single 15 gr
tube

1 pcs.

10



Features

- **Debriding action:** HYDROMED GEL rehydrates necrotic tissue and aids debridement.
- **Desloughing action:** HYDROMED GEL absorbs slough and exudate without damaging fragile granulation tissue.
- **Creates moist wound healing environment:** HYDROMED GEL helps to create an optimal moist wound management environment, which promotes healing, reduces the likelihood of scarring and helps soothe pain.
- **Stable and counteract leakage and maceration:** The viscous texture of the gel makes it stay in place, and even after absorption of debris and exudate the gel will remain cohesive and counteract leakage and maceration.
- **Easy to remove:** The gel is easy to remove due to high cohesion even after absorption of debris and exudate.

Indications

- HYDROMED GEL is primarily intended for the treatment of necrotic and sloughy wounds, e.g. leg ulcers, pressure ulcers and non-infected diabetic foot ulcers.
- HYDROMED GEL may also be used on 1st and 2nd degree burns.
- The gel may be used throughout the healing process to provide a moist wound healing environment in all types of wounds.

Contraindications

- Full thickness burns.

Precautions and Warnings

- May be used on patients with local and systemic infections under the discretion of a healthcare professional.
- In case of allergic reactions, please discontinue use directly and consult your doctor.

- For external use only.
- Do not reuse the product. Reuse may pose risk of infection.
- If packaging is broken do not use or re-sterilize product.

Feature

Good donation properties.
High viscosity.

Benefit

Aids autolytic debridements facilitating faster healing rates.
Minimises the need for sharp debridement
Easy to apply: remains in-situ.



Euroderm new

Transparent sterile adhesive dressing made of a highly resistant thin polyurethane film. Stretchable, hypoallergenic and radiotranslucent. Its transparency allows continuous monitoring of the wound without having to remove the dressing.

Euroderm's waterproof polyurethane support allows patients the opportunity to do regular shower and bath.

Euroderm can be left in place for periods greater than average dressing.

Euroderm is latex free.



Acts as bacteria barrier and Waterproof



Breathable



Latex free



item no.	size	cont.	pkg.	code
717507	cm 6 x 7	100 pz	2	8 059616 066490
717512	cm 10 x 12	50 pz	2	8 059616 066520

Indications

- For the protection of insertion site for venous and arterial catheter or other percutaneous devices
- I and II degree burns
- To Protect the skin at risk of decubitus or subject to redness sites for prolonged immobilization
- As secondary fixation of other dressings

Silvermed with adhesive borders

Sterile island dressing with a central absorbent and non-adherent pad with silver ions.

Thanks to its metallic silver micronized net, preserves the microenvironment of the wound, stimulates the development of granulation tissue and acts as an antibacterial for wounds, ulcers, pressure sores, burns, minor abrasions and donor sites.



Latex free



Low exudate



Micronized silver



Easy to remove



Hypo-allergenic



item no.	size	cont.	pkg.	code
892063	cm 10 x 10	5 pcs.	12	8 057681 905140
892066	cm 10 x 20	3 pcs.	12	8 057681 905157

Silvermed sb without adhesive borders

Sterile and absorbent non adherent dressing with silver ions, without adhesive border. Thanks to its metallic silver micronized net, preserves the microenvironment of the wound, stimulates the development of granulation tissue and acts as an antibacterial for wounds, ulcers, pressure sores, burns, minor abrasions and donor sites.



Latex free



Low exudate



Micronized silver



Easy to remove



Hypo-allergenic



item no.	size	cont.	pkg.	code
892073	cm 10 x 10	5 pcs.	12	8 057681 905171
892076	cm 10 x 20	3 pcs.	12	8 057681 905188

Description

Absorbent antimicrobial wound pad, made of viscose and PE fibres, laminated with silver colour net. The nonwoven layer contains silver ions.

Silvermed is a primary wound dressing. The version without borders (SILVERMED sb) should be fixed with secondary fixation dressing e.g. Euroderm frame style, Euroderm Roll or Eurofix.

Features

- Antimicrobial barrier to prevent colonization on the dressing.
- Prophylaxis against infection especially for wounds at high risk of infection.
- Biocompatible and complying to ISO 10993.

Indications

- Infected wounds, Dry to minimal drainage wounds, partial thickness pressure ulcers, first and second degree burns.

Contraindications

- There are no absolute contraindications for using silvermed.
- Do not use for individuals with known sensitivity to any of the dressing components.
- Due to presence of antimicrobial silver ions, the experience with neonates / small babies is limited.



Eurosorb Highly Integral Alginate (HI)


Sterile, soft and highly absorbent dressing made of 100% calcium alginate fibers. Eurosorb can absorb exudates up to 15 times its weight and can remain in situ up to 7 days, depending on the patient's condition and the amount of exudates. Eurosorb is an integral alginate absorbing exudates between and within its fibers. The advantage is that it can be removed in one piece and the structure of Eurosorb reduces to the minimum the risk of release of alginate fibers into the wound and does not cause trauma to newly formed tissue, does not adhere to the wound bed and release of exudates preserving skin maceration. Eurosorb is indicated for the treatment of leg ulcers, skin donor site and pressure sores.



Highly integral alginate remains integral when wet



Easy to remove

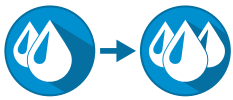

Moderate exudate



Heavy exudate



Latex free

item no.	size	cont.	pkg.	code
501505	cm 5 x 5	10 pcs.	10	8 059616 060191
501510	cm 10 x 10	10 pcs.	10	8 059616 060207
501520	cm 10 x 20	5 pcs.	10	8 059616 060221



Indications:

- Pressure ulcers
- Leg ulcers
- Cavity wounds
- Skin donor sites

Contraindications:

- Surgical implantation
- Third degree burns
- Dry wounds

Features:

- High Absorbency
- High wet strength
- Retains Exudate
- Turns into a gel that will not stick to a wound
- Maintains a 'Moist Wound Healing' environment.
- Donates calcium to the wound.

Benefits:


- Longer wear time between dressings.
- One-piece removal.
- Minimal residues in the wound bed.
- Reduce risk of maceration.
- Reduce trauma during a dressing change.
- Reduce requirement for debridement.
- Optimum wound healing conditions.
- Helps to stop minor bleeding.



Eurosorb roll Highly Integral Alginate (HI)

Sterile, soft high integrity calcium alginate dressing in roll made of 100% calcium alginate fibers. Eurosorb can absorb exudates up to 15 times its weight and can remain in situ up to 7 days, depending on the patient's condition and the amount of exudates. Eurosorb is an integral alginate absorbing exudates between and within its fibers.


The advantage is that it can be removed in one piece and the structure of Eurosorb reduces to the minimum the risk of release of alginate fibers into the wound and does not cause trauma to newly formed tissue, does not adhere to the wound bed and release of exudates preserving skin maceration. Eurosorb Roll is indicated for the treatment of cavity wounds.

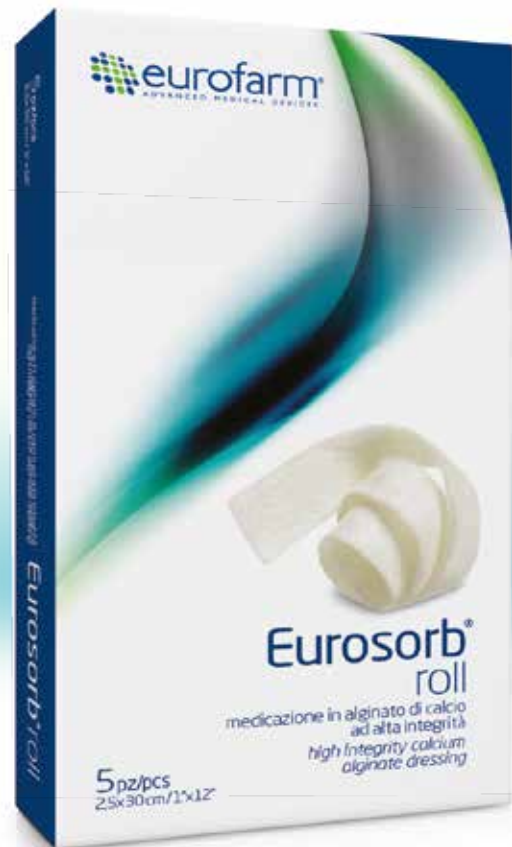

Highly integral alginate remains integral when wet



Easy to remove

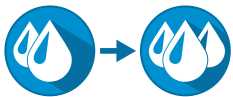

Moderate exudate

Heavy exudate



Latex free

item no. size cont. pkg. code

501630

cm 2,5 x 30

5 pcs.

5



Indications:

- Pressure ulcers
- Leg ulcers
- Cavity wounds
- Skin donor sites

Contraindications:

- Surgical implantation
- Third degree burns
- Dry wounds

Features:

- High Absorbency
- High wet strength
- Retains Exudate
- Turns into a gel that will not stick to a wound
- Maintains a 'Moist Wound Healing' environment.
- Donates calcium to the wound.


Benefits:

- Longer wear time between dressings.
- One-piece removal.
- Minimal residues in the wound bed.
- Reduce risk of maceration.
- Reduce trauma during a dressing change.
- Reduce requirement for debridement.
- Optimum wound healing conditions.
- Helps to stop minor bleeding.



Eurocell hydro Gelling Fiber Dressing

Highly absorbent, conformable and wet integral wound dressing incorporating with the Carboxymethyl Cellulose (CMC) super gelling technology. On contact with the wound exudates, CMC forms a crystal clear gel which helps to keep a moist environment to support the wound healing process and aid in the removal of nonviable tissue from the wound, without damaging newly formed tissue. The dressing is indicated for the management of moderate to heavily exuding partial and full-thickness chronic wounds and it is indicated for acute wounds, it controls minor bleeding in superficial wounds.

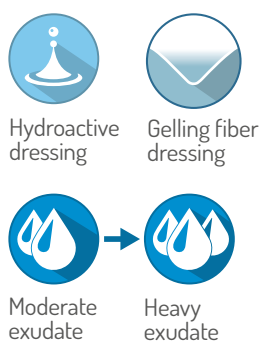

item no.	size	cont.	pkg.	code
895005	cm 5 x 5	10 pcs.	10	8 057681 908929
895010	cm 10 x 10	10 pcs.	10	8 057681 908936
895012	cm 10 x 20	5 pcs.	10	8 057681 908943
895015	cm 15 x 15	5 pcs.	10	8 057681 908950

EUROCEL HYDRO® CMC dressing is designed for the management of moderate to heavily exuding partial and full-thickness chronic wounds and it is indicated for acute wounds, it controls minor bleeding in superficial wounds. **EUROCEL HYDRO® CMC Dressing can be used on the following wounds:**

- Leg ulcers, pressure ulcers (stage II-IV) and diabetic ulcers.
- Cavity wounds.

- Surgical wounds.
 - Superficial and partial thickness burns.
 - Traumatic wounds.
 - Exudates absorption in oncology wounds (eg. Fungating cutaneous tumors, cutaneous metastases and Kaposi's sarcomas).
- EUROCEL HYDRO® CMC Dressing can be left in place up to 7 days where clinically indicated (except for burns where it can be left in place up to 14 days).

Comparison testing has demonstrated that **EUROCEL HYDRO® CMC wound dressing has an excellent performance over the leading gelling dressing in:**

- Absorbency, capacity to absorb high levels of exudates during use.
- Fluid Retention, capacity to retain exudate under compression.
- Wet Strength, to remain intact when being removed from the wound bed.



Eurocell hydro roll Gelling Fiber Dressing

Highly absorbent, conformable and wet integral wound rope incorporating with the Carboxymethyl Cellulose (CMC) super gelling technology.

On contact with the wound exudates, CMC forms a crystal clear gel which helps to keep a moist environment to support the wound healing process and aid in the removal of nonviable tissue from the wound, without damaging newly formed tissue.

The dressing is indicated for the management of moderate to heavily exuding partial and full-thickness chronic wounds and it is indicated for cavity acute wounds.



Hydroactive dressing



Gelling fiber dressing



Moderate exudate



Heavy exudate

item no.	size	cont.	pkg.	code
----------	------	-------	------	------

895630

cm 2,5 x 45

5 pcs.

10



8 057681 908967

EUROCEL HYDRO® CMC dressing is designed for the management of moderate to heavily exuding partial and full-thickness chronic wounds and it is indicated for acute wounds, it controls minor bleeding in superficial wounds.

EUROCEL HYDRO® CMC Dressing can be used on the following wounds:

- Leg ulcers, pressure ulcers (stage II-IV) and diabetic ulcers.
- Cavity wounds.

- Surgical wounds.
 - Superficial and partial thickness burns.
 - Traumatic wounds.
 - Exudates absorption in oncology wounds (eg. Fungating cutaneous tumors, cutaneous metastases and Kaposi's sarcomas).
- EUROCEL HYDRO® CMC Dressing can be left in place up to 7 days where clinically indicated (except for burns where it can be left in place up to 14 days).

Comparison testing has demonstrated that EUROCEL HYDRO® CMC wound dressing has an excellent performance over the leading gelling dressing in:

- Absorbency, capacity to absorb high levels of exudates during use.
- Fluid Retention, capacity to retain exudate under compression.
- Wet Strength, to remain intact when being removed from the wound bed.




Euroderm foam

Euroderm Foam is a multi-layer hydro cellular polyurethane foam with full hydroactive area consisting of a highly absorbent soft polyurethane membrane and of a breathable polyurethane film which is water permeable and bacteria impermeable. This allows the excess of water vapour to evaporate protecting in the meantime the wound area from external contaminations maintaining the optimal moist wound environment for healing process.

Indications: I to IV degree pressure ulcers, post-operative wounds, vascular lesions, 1st and 2nd degree burns, lesions with fragile, delicate periwound skin and skin donor sites.



item no.	size	cont.	pkg.	code
892910	cm 10 x 10	5 pcs.	10	8 057681 905430
893010	cm 10 x 10	10 pcs.	10	8 057681 905447
893015	cm 15 x 15	5 pcs.	10	8 057681 905454
893020	cm 20 x 20	3 pcs.	10	8 057681 905461

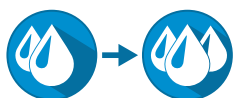

Moderate exudate



Heavy exudate



Hypo-allergenic



Latex free

Benefits

- Rapid absorption of 90%, or more, of wound exudates under compression.
- High water vapor permeability Euroderm Foam 5000gms/sqm/24hrs .
- Waterproof foamed PU film Laminated cover which is permeable to gases and water vapor.
- Euroderm Foam traps cellular debris and proteinaceous materials in its pores of wound contact surface.
- Euroderm Foam maintains moist warm environment at the surface of the wound, which induces formation of granulation tissue and epithelialization. In addition, these conditions favours auto-debridement of the wound.

Indications

- Euroderm Foam is used to dress highly exuding wounds, leg ulcers, pressure ulcers, diabetic ulcers, second degree burns and traumatic wounds.

Contraindications

- Euroderm Foam should not be applied to wounds covered with dry scab or hard black necrotic tissue until this has been removed surgically, also Euroderm Foam should not be used in the treatment of infected wounds nor wounds extending into muscle, bone and tendon.

Performance


- The key performance parameter for foam dressings is the ability to absorb high levels of exudate whilst also maintaining a moist wound environment.

Feature

Good donation properties
High viscosity

Benefit

Aids autolytic debridements facilitating faster healing rates.
Minimises the need for sharp debridement.
Easy to apply: remains in-situ.

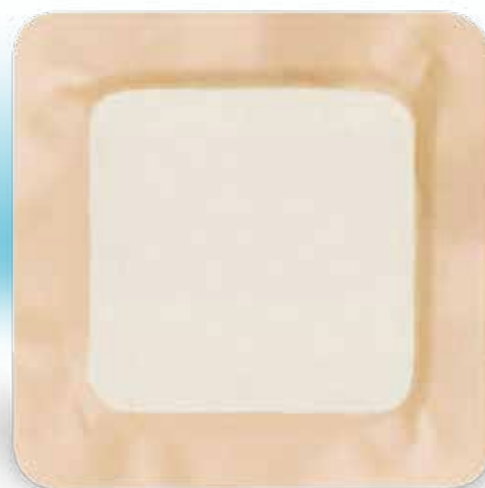

Euroderm foam plus

Multi-layered, hydrocellular polyurethane foam pad combined with an adhesive border, which grants an effective fixation, even on difficult-to-dress areas. It consists of hypoallergenic semi-permeable adhesive polyurethane top layer and a soft highly absorbent foam pad. Fluids are uniformly distributed into the foam pad which, thus, perfectly conforms to the wound-bed maintaining the optimal moist wound environment for healing process.



Moderate
exudate



Heavy
exudate



Hypo-allergenic



Latex free

item no.	size	cont.	pkg.	code
893110	cm 10 x 10	10 pcs.	5	8 057681 905492
893115	cm 15 x 15	5 pcs.	5	8 057681 905508
893120	cm 20 x 20	3 pcs.	5	8 057681 905515




Euroderm foam heel

Multilayered hydrocellular shaped polyurethane foam for the management of wounds on the heel or on any similar anatomical areas.

Euroderm Foam Heel is non-adhesive. This allows an atraumatic and pain free dressing removal causing minimal distress to the patient, even on fragile tissues. The high absorption of the foam dressing provides superior fluid management for a longer period of time.

Euroderm Foam Heel is indicated for diabetic foot ulcers, stage I to IV pressure ulcers, prevention of pressure ulcers on patient with partial ability to move.

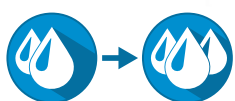

Moderate exudate

Heavy exudate



Hypo-allergenic



Latex free

item no. size cont. pkg. code


893060

cm 12 x 13

3 pcs.


5



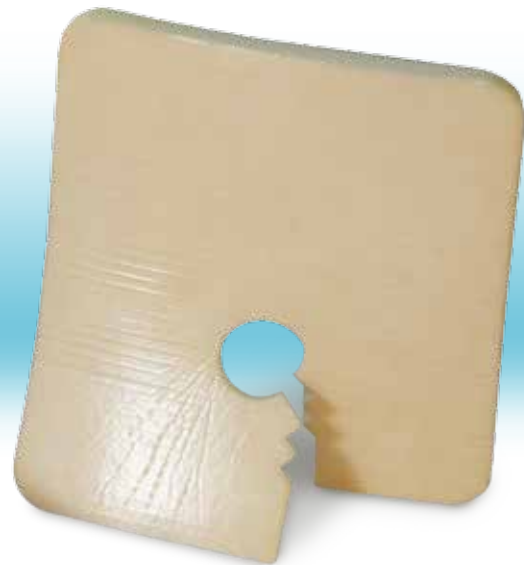

Euroderm foam tracheo

Euroderm Foam Tracheo is a “Key-hole” opened hydrocellular polyurethane foam device. Highly absorbent soft polyurethane foam layer and of a breathable polyurethane film which is water and bacteria impermeable and vapour permeable. Fluids are uniformly distributed into the foam dressing, thus, perfectly conforms to the tracheotomy or drainage tube assuring the maximum of comfort to the patient. The high absorption of the foam dressing provides superior fluid management for a longer period of time. Euroderm Foam Tracheo is indicated for fluid management around tracheotomy and drainage sites.

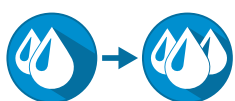

Moderate exudate

Heavy exudate



Hypo-allergenic



Latex free


item no.	size	cont.	pkg.	code
893040	cm 8,5 x 8,5	10 pcs.	5	8 057681 905478
893035	cm 6,5 x 6,5	10 pcs.	5	8 057681 908103



Euroderm foam plus concave

Multi-layered hydrocellular shaped polyurethane foam with adhesive border and the advantage of an anatomical design well suited for the management of wounds on the heel or on any similar anatomical areas. Provides superior fluid management for a longer period of time.

This Concave is indicated for diabetic foot ulcers, stage I-IV pressure ulcers, prevention of pressure ulcers on patients with partial ability to move, surgical wounds, 1st and 2nd degree burns, donor sites.

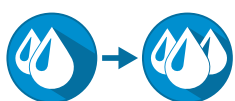

Moderate exudate

Heavy exudate



Hypo-allergenic



Latex free

item no. size cont. pkg. code

893140

cm 18 x 16,5

3 pcs.

5



Euroderm foam plus sacral




Multi-layered hydrocellular shaped polyurethane foam with adhesive border and the advantage of an anatomical design well suited for the sacral area.

It consists of a hypoallergenic semi-permeable adhesive polyurethane top layer and a soft highly absorbent foam pad. Euroderm Foam Plus Sacral well suited for wounds of the sacral area where a waterproof covering is recommended. The thickness of the foam ensures a cushioning effect.

Euroderm Foam Plus Sacral reduces pressure over the wound area, thereby reducing friction and shear stresses and giving excellent patient comfort and protection.

It is indicated for pressure ulcers of the sacral area, stage I-IV pressure ulcers, surgical wounds, 1st and 2nd degree burns, donor sites.

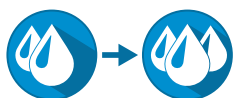

Moderate
exudate

Heavy
exudate



Hypo-allergenic



Latex free

item no. size cont. pkg. code

893150

cm 22 x 22

3 pcs.

5



Euroderm foam cavity new

Sterile, non-adhesive polyurethane foam dressing composed by an hydroactive surface. It is highly absorbent, for medium or high level of exudate. Euroderm cavity foam conforms perfectly to the cavity wound and to the different anatomical part of the body with excellent comfort for the patient, it does not adhere to the wound surface.


The product has a good conformability and softness.

Furthermore, the thickness of the foam creates the bearing effect, which reduces the pressure on the lesion and avoid friction.

Euroderm foam cavity is latex free.

Indicated for:

- Treatment of cavity wounds.
- Skin donor sites.
- Pressure ulcers.
- Wounds after excision of fistulas and abscesses.
- Venous ulcers.

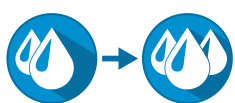

Moderate exudate

Heavy exudate




Hypo-allergenic



Latex free

item no.	size	cont.	pkg.	code
893205	Ø cm 5	10 pcs.	10	8 057681 908998
893207	Ø cm 7,5	3 pcs.	10	8 057018 290222
893210	Ø cm 10	5 pcs.	10	8 057681 909001

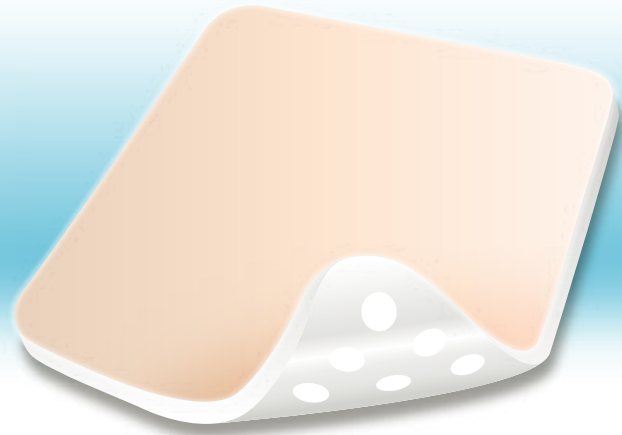

Euroderm foam silicone new

Euroderm foam silicone is a highly absorbent dressing made of polyurethane foam with polyurethane top sheet and perforated soft silicone gel wound contact layer. The dressing does not adhere to wound bed thereby minimizing disruption of healing tissue.

It is indicated for the management of moderate to heavily exuding partial to full thickness wounds including:

- Suitable for use on fragile skin.
- Stage I - IV pressure ulcers, neuropathic (diabetic), Venous and Arterial leg ulcers.
- Diabetic foot ulcers.
- Suitable for use with compression.
- First/Second degree burns.
- Skin tears.
- Surgical wounds.
- Donor sites.



Moderate
exudate



Heavy
exudate



Hypo-allergenic



Latex free

item no.	size	cont.	pkg.	code
893310	cm 10 x 10	10 pcs.	10	8 057018 291205
893315	cm 15 x 15	5 pcs.	10	8 057018 291212
893320	cm 20 x 20	3 pcs.	10	8 057018 291229



sports line

Euroathletic KLS

Euroathletic sport tape

Eurosave

great athletes are cured like this



Euroathletic[®] KLS new kinesio lymphatic sport

Adhesive elastic bandage for the kinesio therapy and natural prevention of trauma and muscular and joint inflammation.

Relieves pain. Corrects muscle and joint functions. Promotes lymphatic circulation. Helps rehabilitation. Treats minor swelling and subcutaneous bruising.



item no.	color/size	cont.	pkg.	code
520325	blue m 5 x cm 5	1 pcs.	24	8 057018 291489
520355	pink m 5 x cm 5	1 pcs.	24	8 057018 291519
520335	black m 5 x cm 5	1 pcs.	24	8 057018 291496
520305	skin m 5 x cm 5	1 pcs.	24	8 057018 291458

Euroathletic sport tape new

Rigid strapping zinc oxide tape for functional and sports bandaging in 100% viscose with hypoallergenic zinc oxide adhesive. It provides strong and efficient stabilisation of joint structures. It is hand tearable for quick and easy application. Indicated for functional bandaging of muscles ligaments and joints injuries.



item no.	color/size	cont.	pkg.	code
----------	------------	-------	------	------

856023	m 10 x cm 3,8	1 pcs.	12	
--------	---------------	--------	----	--



856035	m 14 x cm 5	1 pcs.	12	
--------	-------------	--------	----	--



Latex free



Ideal for all sports



item no.	color/size	cont.	pkg.	code
746032	m 10 x cm 2,5	24 pcs.	2	8 057018 291298
746033	m 10 x cm 3,8	18 pcs.	2	8 057018 291304
746035	m 10 x cm 5	12 pcs.	2	8 057018 291311
746036	m 14 x cm 2,5	48 pcs.	2	8 057018 291328
746037	m 14 x cm 3,8	32 pcs.	2	8 057018 291335
746038	m 14 x cm 5	24 pcs.	2	8 057018 291342
746039	m 20 x cm 2,5	30 pcs.	2	8 057018 291359
746040	m 20 x cm 3,8	20 pcs.	2	8 057018 291366
746041	m 20 x cm 5	15 pcs.	2	8 057018 291373

Eurosave underwrap

Thin elastic polyurethane foam bandage, resistant and highly porous. Eurosave is the best solution for maximum comfort of the athletes, to be used under Eurotape or similar products for their comfortable and painless removal during competition.



item no.	size	cont.	pkg.	code
814002	m 27 x cm 7	1 pcs.	48	8 057681 908073



Latex free



bandages

Eurelasto L

Eurelasto

Eurelasto F

Eurozinc

Eurozinc elast

Eurozinc alginate

Eurozinc bielast

Flebimed short

Flebimed medium

Flebimed strong

Flebimed bielast

Eurobandage

Eurocollar

bandages

Eurelasto L

Cohesive elastic bandage in cotton, viscose and polyamide.
Suitable for applications in difficult parts of the body; avoids the use of clips.
Adheres to itself but not to skin or other fabrics.



item no.	size	cont.	pkg.	code
516004	m 4 x cm 4	1 pcs.	12	8 059616 061143
516006	m 4 x cm 6	1 pcs.	12	8 059616 061150
516008	m 4 x cm 8	1 pcs.	12	8 059616 061167
516010	m 4 x cm 10	1 pcs.	12	8 059616 061174
516012	m 4 x cm 12	1 pcs.	12	8 059616 061181
516104	m 20 x cm 4	1 pcs.	1	8 059616 061198
516106	m 20 x cm 6	1 pcs.	1	8 059616 061204
516108	m 20 x cm 8	1 pcs.	1	8 059616 061211
516110	m 20 x cm 10	1 pcs.	1	8 059616 061228
516112	m 20 x cm 12	1 pcs.	1	8 059616 061235



Soft



Conformable

Eurelasto

100% white cotton elastic adhesive bandage with a coloured guideline along middle. Coated with zinc oxide hypoallergenic adhesive and natural resins. Especially suitable as compressive bandage for the treatment of the vascular diseases and in the prophylaxis of thrombosis. Used as support and relief bandaging in case of muscular and tendinous injuries, distortion, dislocations and sprains.



item no.	size	cont.	pkg.	code
508006	mt 4,5 x cm 6	1 pcs.	16	8 059616 060474
508008	mt 4,5 x cm 8	1 pcs.	16	8 059616 060481
508010	mt 4,5 x cm 10	1 pcs.	16	8 059616 060498

Latex free



Conformable



Eurelasto F

100% Extra Strong white cotton elastic adhesive bandage with a coloured guideline along the middle. Coated with zinc oxide hypoallergenic adhesive and natural resins. Especially suitable as compressive bandage for the treatment of the vascular diseases and in the prophylaxis of thrombosis. Used as support and relief bandaging in case of muscular and tendinous injuries, distortion, dislocations and sprains. Specially designed for contact sport (rugby, martial arts)



item no.	size	cont.	pkg.	code
508018	mt 4,5 x cm 8	1 pcs.	16	8 057681 909032

Latex free



Conformable



Eurozinc

Rigid bandage impregnated with a zinc oxide ointment with a strong anti-inflammatory action.

Suitable for the treatment of leg ulcers caused by venous insufficiency (varicose veins, post-phlebitis syndrome and venous oedema), and for the treatment of sprains after the removal of the plaster cast.



item no.	size	cont.	pkg.	code
507005	mt 5 x cm 10	1 pcs.	10	8 059616 060436
507007	mt 7 x cm 10	1 pcs.	10	8 059616 060443
507010	mt 10 x cm 10	1 pcs.	10	8 059616 060450

Latex free



Eurozinc elast

Elastic bandage impregnated with a zinc oxide ointment with a strong anti-inflammatory action.

Suitable for the treatment of leg ulcers caused by venous insufficiency (varicose veins, post-phlebitis syndrome and venous oedema), and for the treatment of sprains after the removal of the plaster cast.



item no.	size	cont.	pkg.	code
506005	mt 5 x cm 10	1 pcs.	10	8 059616 060405
506007	mt 7 x cm 10	1 pcs.	10	8 059616 060412
506010	mt 10 x cm 10	1 pcs.	10	8 059616 060429

new

Eurozinc alginate



Latex free

Elastic alginate is a semi-rigid bandage impregnated with zinc oxide paste and calcium alginate. coolant and supple. Useful for the treatment of the acute phase of joint injuries, muscle, ligaments and tendons.

Very easy to cast and adaptable to every type of body contour.

Eurozinc alginate is also indicated for the treatment of leg ulcers caused by venous insufficiency (varicose veins, post-phlebitis syndrome and venous edema) or as a support bandage.

Eurozinc alginate is also indicated as first aid in sports injuries.

The hydrocolloidal properties ensures a refreshing effect for about 4 hours.

Once the bandage is dry, the cooling effect can be increased by conserving the bandage in fridge

item no.	size	cont.	pkg.	code
506035	m 5 x cm 8	1 pcs.	10	8 057681 909124
506027	m 7 x cm 10	1 pcs.	10	8 057681 909117

new

Eurozinc bielast



Latex free

Eurozinc bielast is an elastic medicated bandage in both direction lengthwise and crosswise ready to use. Impregnated with zinc oxide paste with extraordinary anti-inflammatory properties. It is able to guarantee a moist environment and will not dry during use.


Easy to apply and well tolerated by the skin. It has a skin refreshing effect after application. Very easy to cast and adaptable to every type of body contour.

Indicated for the treatment of ulcers of the lower limbs caused by venous insufficiency (varicose veins, post-phlebitis syndrome and venous edema) also commonly used for the treatment of distortions or as a support bandage.

item no.	size	cont.	pkg.	code
506105	m 5 x cm 10	1 pcs.	10	8 057018 290536
506107	m 7 x cm 10	1 pcs.	10	8 057018 291601
506110	m 10 x cm 10	1 pcs.	10	8 057018 290543

Flebimed short

Non-adhesive short stretch (35%) elastic compression bandage.
Made of: cotton 80% - nylon 20%. Can be sterilised, comes with clips. Flesh colour.
To be used for support and relief bandages in cases of contusions, sprains and dislocations; in case of tendinopathy, for the immobilisation of joints, protection against haematoma in case of muscular wounds. Also suitable for phlebology uses (oedema, lymphoedema, ulcus cruris, symptomatic varicose system).

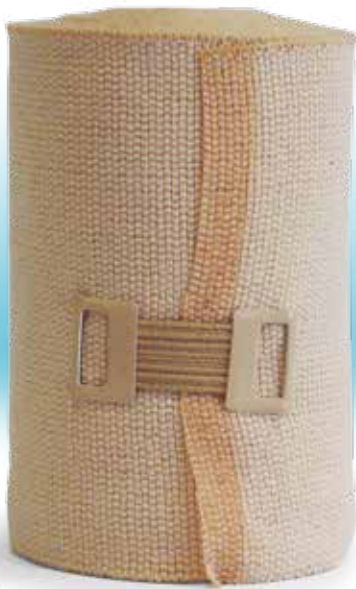

item no.	size	cont.	pkg.	code
527004	mt 5 x cm 4	1 pcs.	10	8 057681 909216
527006	mt 5 x cm 6	1 pcs.	10	8 057681 909223
527008	mt 5 x cm 8	1 pcs.	10	8 057681 909230
527010	mt 5 x cm 10	1 pcs.	10	8 057681 909247
527012	mt 5 x cm 12	1 pcs.	10	8 057681 909254

Comfortable




Washable



Flebimed media

Made of 100% cotton. Can be sterilised, comes with clips. Flesh colour.
Can be used for the treatment of chronic venous insufficiency, in cases of primary and secondary varicose veins, leg ulcers, oedema, thrombophlebitis and after vaso-sclerotherapy.

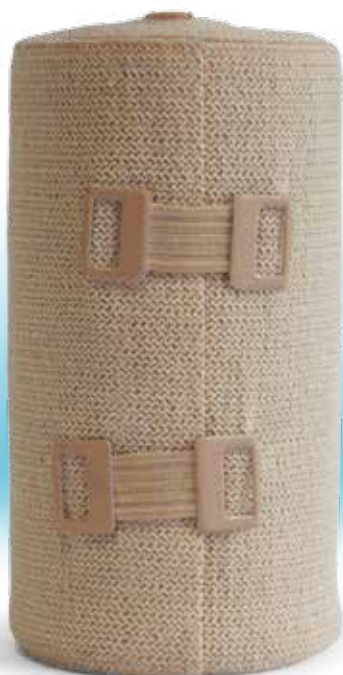

item no.	size	cont.	pkg.	code
527106	mt 5 x cm 6	1 pcs.	10	8 057681 909261
527108	mt 5 x cm 8	1 pcs.	10	8 057681 909278
527110	mt 5 x cm 10	1 pcs.	10	8 057681 909285
527112	mt 5 x cm 12	1 pcs.	10	8 057681 909292

Comfortable







Washable



Flebimed forte

Non-adhesive elastic compression band with extra extendability and strong compression: 170%. Made of: cotton 92%, polyurethane 8%. Can be sterilised, comes with clips. Flesh colour. To be used as phlebological compression bandages, for the treatment of the symptomatic varicose system, for the prevention and cure of sports- or work-related trauma, for the support and relief of sprains, dislocations, contusions and slight fractures.



item no.	size	cont.	pkg.	code
527206	mt 7 x cm 6	1 pcs.	10	8  057681 909308
527208	mt 7 x cm 8	1 pcs.	10	8  057681 909315
527210	mt 7 x cm 10	1 pcs.	10	8  057681 909322
527212	mt 7 x cm 12	1 pcs.	10	8  057681 909339

Comfortable



Washable







Flebimed bielast

Non-adherent bi-elastic compression bandage with high elasticity: 160% longitudinally, 80% transversally.

Made of: cotton 70%, nylon 19%, polyurethane 11%. Can be sterilised, comes with clips. Flesh colour. To be used as compression bandages after surgery, for the treatment of the symptomatic varicose system, for the prevention and treatment of sports- and work-related trauma, and for the support and relief of sprains and dislocations.



item no.	size	cont.	pkg.	code
527306	mt 7 x cm 6	1 pcs.	10	8  057681 909346
527308	mt 7 x cm 8	1 pcs.	10	8  057681 909353
527310	mt 7 x cm 10	1 pcs.	10	8  057681 909360
527312	mt 7 x cm 12	1 pcs.	10	8  057681 909377

Comfortable



Washable



Eurobandage

Elastic bandage (not self-adherent) in polyester and cotton, with attachment clips. Suitable for use as low compression bandages to treat the varicose system and thrombosis prevention treatment, also for the support and relief of sprains, dislocations, contusions. Washable, reusable and durable, even in the presence of oils and ointments.

item no.	size	cont.	pkg.	code
519005	mt 4,5 x cm 5	1 pcs.	12	8 059616 061310
519007	mt 4,5 x cm 7,5	1 pcs.	12	8 059616 061327
519010	mt 4,5 x cm 10	1 pcs.	12	8 059616 061334
519015	mt 4,5 x cm 15	1 pcs.	12	8 059616 061341



Washable



Eurocollar

Orthopaedic cervical collar. Suitable for cases in which the uncontrolled movement of the cervical vertebrae causes pain. For collision trauma, stiff neck of rheumatic origin and sports trauma.

item no.	size	cont.	pkg.	code
503002	small	1 pcs.	10	8 059616 060290
503003	medium	1 pcs.	10	8 059616 060306
503004	farge	1 pcs.	10	8 059616 060313



Latex free







Washable




product legend

Wound bed preparation







	Chronic wounds	Acute wounds	I. degree burns	II. degree burns	III. degree burns	IV. degree burns	Infected wounds	
Vulnomed 	✓	✓	✓	✓	✓	✓	✓	<ul style="list-style-type: none">→ Sterile Saline Solution→ Prevents bacterial formation→ Well-Know substances with low allergenic potential→ Compatible with comonly used wound dressings→ Opening does not affect shelf life
Eurodressing 	✓	✓	✓	✓	✓	✓	✓	<ul style="list-style-type: none">→ Fast absorbency→ Can be use it to remove exudate and necrotic tissue→ Can use instead of gauze→ May also be used as the primary wound contact layer for more heavily exuding wounds, if backed by a second absorbent dressing




Necrotic to fibrinous wounds

	Dry necrotic wounds	Moderately exuding necrotic wounds	Sloughy necrotic wounds	
<p>Hydromed Gel</p> 	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<p>→ Special Cannula to reach deeper wounds and sinuses</p> <p>→ Outstanding hydration properties</p> <p>→ Cristal clear for accurate wound assesment</p> <p>→ Exceptional stability; remains in place after application</p>
<p>Eurosorb</p> 	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<p>→ Optimal conformability, absorbency</p> <p>→ Vertical absorption</p> <p>→ No maceration of wound edges</p> <p>→ Comes off in one piece</p> <p>→ Atraumatic, pain-free dressing removal</p>
<p>Eurocell hydro</p> 	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<p>→ Optimal conformability, absorbency and gel formation</p> <p>→ Vertical absorption</p> <p>→ No maceration of wound edges</p> <p>→ Comes off in one piece</p> <p>→ Atraumatic, pain-free dressing removal</p>
<p>Euroderm foam</p> 	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<p>→ High, long lasting absorption capacity</p> <p>→ High retention</p> <p>→ Can be use with compression therapy</p> <p>→ No maceration of wound edges</p> <p>→ Soft and flexible even when saturated</p>



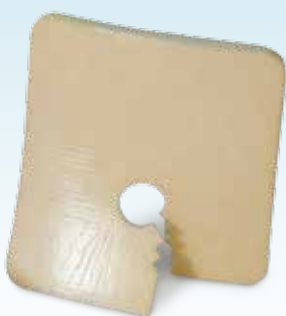







 low exuding wounds
 moderately exuding wounds
 heavily exuding wounds

Fibrinous to granulating wounds

	Pressure ulcers	Arterial and venous leg ulcers	Diabetic foot ulcers/surgical wounds	Surgical wounds	Traumatic wounds	II. degree burns	Tracheostomies/drainage sites	
Euroderm foam 	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	 → Excellent fluid handling characteristics → Cost efficient: no need frequent dressing changes → Soft and Comfortable → Sealed edges: low risk of leaving pressure marks → Atraumatic. Painless dressing changes → Visual control of exudate absorption
Euroderm foam heel 	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	 → Ready to use, no fixation device needed → Innovative anatomical design covering the malleolus → Pressure relieving foam: helps prevent stage I pressure ulcers → Good protection against friction and shear stress
Euroderm foam tracheo 	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	 → Pre-cut opening to fit neatly around tracheostomies and drainage sites → Can be used around any cannula or stoma




 low exuding wounds
 moderately exuding wounds
 heavily exuding wounds

Granulating to epithelializing wounds

	Pressure ulcers	Arterial and venous leg ulcers	Diabetic foot ulcers	Surgical wounds	Traumatic wounds	II. degree burns	Superficial acute wounds	Skin graft	
Euroderm foam plus concave 	✓	✓	✓	✓	✓	✓	✓	☐	 <ul style="list-style-type: none"> → Rapid transparent properties → Excellent fluid management → Safe adhesion to surrounding skin → Safe fixation due to adhesive border → Atraumatic, painless removal → Soft and comfortable
Euroderm foam plus 	✓	✓	✓	✓	✓	✓	✓	☐	 <ul style="list-style-type: none"> → Easy to apply, no secondary fixation needed → Safe fixation due to adhesive border → Adaptable to body contours → Stay in place during showering
Euroderm foam plus sacral 	✓	☐	☐	✓	✓	✓	✓	☐	 <ul style="list-style-type: none"> → Anatomic shape adapted to sacrum area → Good protection against pressure ulcers and maceration → Stays in place during showering
Hydromed 	✓	✓	✓	✓	✓	☐	☐	☐	 <p>Hydrogel Technology</p> <ul style="list-style-type: none"> → Strong and long lasting absorption capacity → High cohesive strength of the gel → Less dressing residues in the wound bed → Low risk of dressing melt down and leakage → Reduce odours
Euroderm plus 	✓	✓	☐	✓	✓	✓	☐	☐	 <ul style="list-style-type: none"> → Efficient absorption of residual exudate → Thin dressing that fits any body contours → Transparent, allows visual control of the wound → Easy to apply → Stay in place during showering

Infected wounds ○

Pressure ulcers
 Arterial and venous leg ulcers
 Diabetic foot ulcers
 II. degree burns
 Donor sites
 Skin graft

 low exuding wounds
 moderately exuding wounds
 heavily exuding wounds

Silvermed sb



-
-
-
-
-
-

Silver Ions Technology

- Conformable, sterile wound dressing
- Thick, absorbent layer made of non woven viscose-rayon, and an activated Silver ions central pad for bacteria and malodorous absorption
- No activation needed, ionic silver readily available
- Silvermed has limited absorption capacity: it can be used alone on low exuding malodorous wound, or in combination with other absorbing dressings for moderately to heavily exuding malodorous wounds
- Efficient odour absorption
- Can be cut

Silvermed






-
-
-
-
-
-

Silver Ions Technology

- Conformable, sterile wound dressing
- Thick, absorbent layer made of non woven viscose-rayon, and an activated Silver ions central pad for bacteria and malodorous absorption
- No activation needed, ionic silver readily available
- Silvermed has limited absorption capacity: it can be used alone on low exuding malodorous wound, or in combination with other absorbing dressings for moderately to heavily exuding malodorous wounds
- Efficient odour absorption
- Adhesive border

Post-operative dressings • Minor wounds

Primary wound contact layer backed by secondary dressing
 Control the microenvironment
 Small burns
 post-operative island dressing
 Small ulcer of the low parts of the body

 low exuding wounds
 moderately exuding wounds
 heavily exuding wounds

	Primary wound contact layer backed by secondary dressing	Control the microenvironment	Small burns	post-operative island dressing	Small ulcer of the low parts of the body	
Eurodressing 	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	 → Fast absorbency → Low adherence → Ideal for lightly exuding wounds → Can be use it to remove exudate and necrotic tissue → Can use instead of gauze → May also be used as the primary wound contact layer for more heavily exuding wounds, if backed and covered by a second absorbent dressing
Eurodressing metal 	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	 → Fast absorbency → Low adherence → Ideal for lightly exuding wounds where Antibacterial effect in required → May also be used as the primary wound contact layer for more heavily exuding wounds, if backed and covered by a second absorbent dressing
Euromed 	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	 → Low absorbency → Low adherence → Ideal for lightly exuding wounds/scars → Minor bleeding cuts → To be used as mechanical barrier → To be used to control the microenvironment of the wound, ulcer of the low parts of the body, burns and skin abrasions in general
Eurometal 	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	 → Low absorbency → Low adherence → Ideal for lightly exuding wounds where Antibacterial effect in required → Small wounds/scars → Minor bleeding wounds → To be used as mechanical barrier → To be used to control the microenvironment of the wound, ulcer of the low parts of the body, burns and skin abrasions in general







eurofarm®
ADVANCED MEDICAL DEVICES