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The solution is a spray

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VULNOMED®

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Spray detergent to be applied on wounds,
sores, burns and rashes

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VULNOMED is a sterile isotonic saline spray solution based
on Silver chloride and Aloe vera for the cleansing,
irrigation and hydration of the skin.

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Aloe Vera is known for its healing, restorative
and antiseptic properties.

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Silver Chloride prevents microbial contamination
of the dressing and keeps the wound environment free
from exogenous bacteria.

produced by

eurofarm®
ADVANCED MEDICAL DEVICES

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VULNOMED is a sterile isotonic saline spray solution based on Silver Chloride and Aloe Vera for the cleansing, irrigation and hydration of the skin. It is indicated for acute and chronic wounds, dermatological lesions of varying etiology, intact or injured skin surfaces. It allows a complete and thorough cleansing of the damaged skin and the neighboring area. It may be useful in the debridement of the crusty necrotic tissue.

ADVANTAGES

The presence of a mild surfactant and the mechanical action of the spray allow a complete cleansing of the injured skin without damaging the newly formed tissue. Silver Chloride prevents microbial contamination of the dressing and keeps the wound environment free from exogenous bacteria.

Aloe Vera is known for its healing, restorative and antiseptic properties. The special dispenser allows the application of the product in any position. The product contains no preservatives or propellants harmful to the ozone.

PRESENTATION

Pressurized aluminum canister containing 125 ml of spray solution.

FORMULATION

Decyl Glucoside, Sodium Chloride, extract of Aloe Vera, Silver Chloride, purified water.

HOW TO USE

Spray the solution directly on the area to be treated and remove any impurities with sterile gauze. Repeat the treatments, if required, two/three times daily, before each dressing. The product can be applied in any position (360°).

WARNINGS

Pressurized container. Do not burn or pierce even after use. Keep container in a cool, well-ventilated area away from sources of ignition.

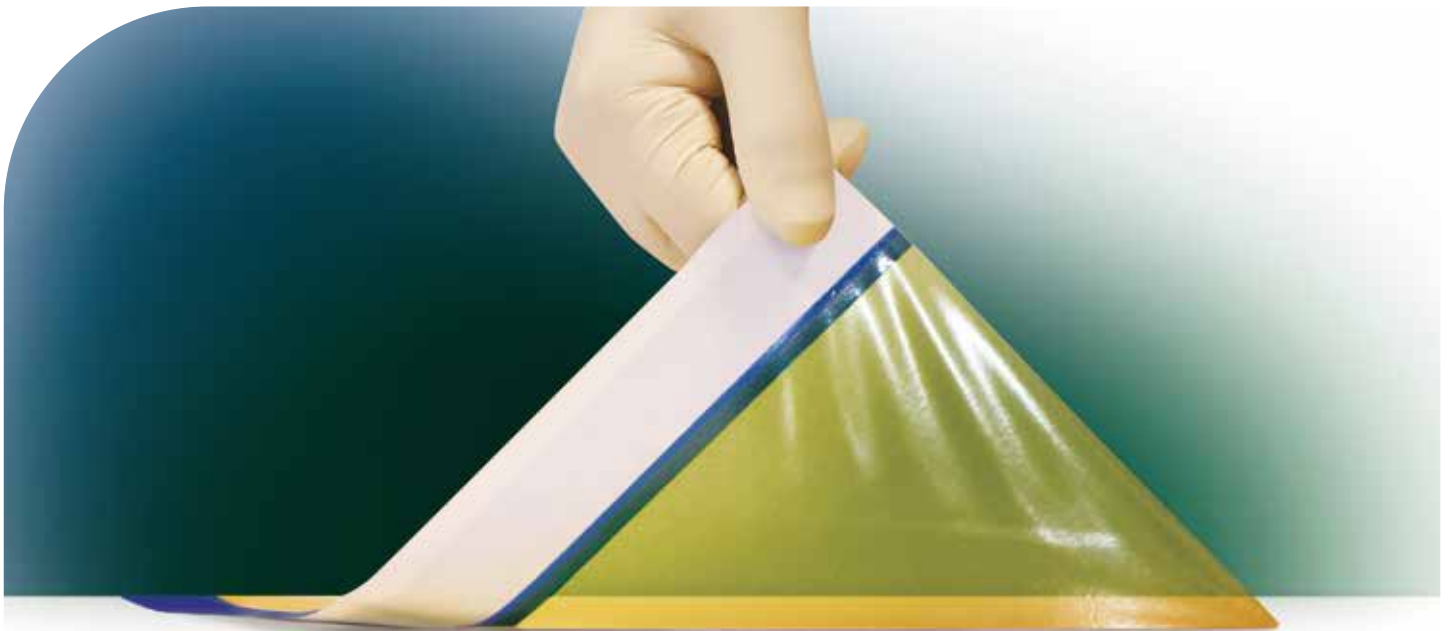
Keep away from children. Avoid contact with eyes. Do not swallow. If swallowed, seek medical advice. Do not mix with other liquids.

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produced by
eurofarm®
ADVANCED MEDICAL DEVICES

EUROFARM S.p.A.
Zona Industriale - Piano Tavola - 95032 - Belpasso - CT - ITALY
www.eurofarm-spa.com - tel. +39 095 391346

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The surgical incise drape Euroderm Incise is a transparent film of sterile polyurethane adhesive, anti-static, non-reflective which makes clearly visible the incision area. Euroderm Incise is latex free.

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Features

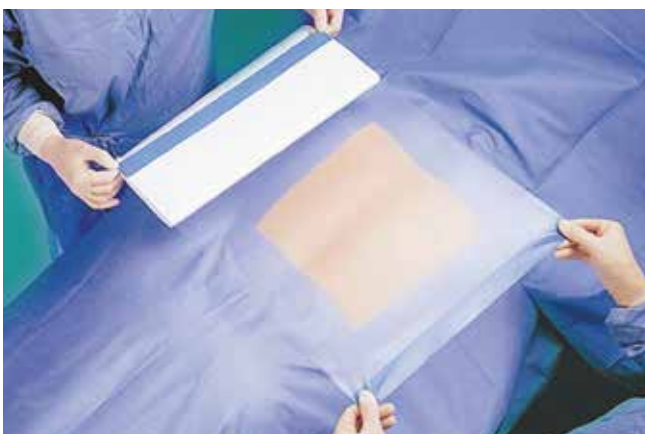
- Conformable, flexible and elastic: the flexibility and conformability of the film makes it suitable for the most difficult areas of intervention.
- Perfect adhesion: the drape adheres firmly to the skin surrounding the incision area for the whole duration of the intervention, thus avoiding the use of hooks.
- Thin but strong: the film is very strong and tear-resistant but can be clearly incised with a surgical scalpel.
- The film creates a sterile incision area: the drape guarantees a sterile surgical area and protects it from contamination.

Benefits

- Adheres to the edge of the wound.
- Avoid lateral migration of bacteria into the wound.
- Non-reflective surface that provides perfect visibility of the incision area.
- Ideal in cases where the manipulation is required during surgery (hip prosthesis).
- Liquids and bacteria proof.
- Wide range of sizes.

Indications

- Recommended for all surgeries: Orthopaedics • Neurosurgery • Abdominal surgery • Plastic surgery • Cardiac and thoracic surgery • Ophthalmic surgery Paediatric surgery

articolo item no.	○ misure size	→ area adesiva adhesive area	○ conf. pkg.
717710	cm 10 x cm 24	cm 10 x cm 18	10 pz/pcs
717715	cm 15 x cm 26	cm 15 x cm 20	10 pz/pcs
717730	cm 30 x cm 26	cm 30 x cm 20	10 pz/pcs
717740	cm 40 x cm 34	cm 40 x cm 28	10 pz/pcs
717742	cm 42 x cm 42	cm 42 x cm 36	10 pz/pcs
717744	cm 44 x cm 26	cm 44 x cm 20	10 pz/pcs
717745	cm 45 x cm 55	cm 45 x cm 49	10 pz/pcs
717750	cm 50 x cm 90	cm 50 x cm 84	10 pz/pcs
717760	cm 60 x cm 52	cm 60 x cm 46	10 pz/pcs
717770	cm 70 x cm 80	cm 70 x cm 74	10 pz/pcs
717780	cm 80 x cm 60	cm 80 x cm 54	10 pz/pcs
717790	cm 90 x cm 80	cm 90 x cm 74	10 pz/pcs

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Euroderm Incise Iodine respects the same indications and has the same properties of Euroderm Incise. Euroderm Incise Iodine provides continuous antimicrobial broad-spectrum action by the gradual release of iodine molecules for over 7 hours of application. The film is liquid and bacteria proof (also in dry conditions) but it is permeable to gases and water vapour, protecting the area from contamination and leaves the skin to breathe, thereby preventing the accumulation of moisture under the drape. A critical aspect of all surgical interventions is the prevention of infections; Euroderm Incise Iodine reduces the entry of microorganisms into the wound and prevents lateral migration of bacteria into the wound itself. Studies on the effectiveness of Euroderm Incise Iodine on Staphylococcus Aureus (11000 CFU/ml) of Pseudomonas Aeruginosa (10200 CFU/ml) and Candida Albicans (11000 CFU/ml), showed a total reduction of bacteria. Euroderm Incise Iodine is latex free.

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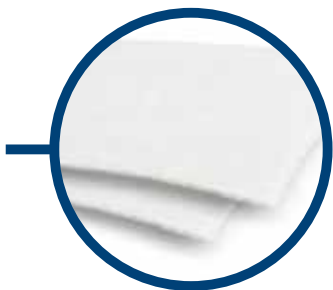
Sterile hydroactive dressing of carboxymethylcellulose fibers

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Sterile hydroactive ribbon dressing of carboxymethylcellulose fibers

Eurocell[®] hydro / Eurocell[®] hydro roll

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Eurocell Hydro is a highly absorbent, conformable and wet integral wound dressing incorporating with the Carboxymethyl Cellulose (CMC) super gelling technology.

On contact with the wound exudates, CMC forms a crystal clear gel which helps to keep a moist environment to support the wound healing process and aid in the removal of nonviable tissue from the wound, without damaging newly formed tissue.

Eurocell Hydro CMC dressing is designed for the management of moderate to heavily exuding partial and full-thickness chronic wounds and it is indicated for acute wounds, it controls minor bleeding in superficial wounds. Eurocell Hydro CMC Dressing can be left in place up to 7 days where clinically indicated (except for burns where it can be left in place up to 14 days).

Intended use:

- Leg ulcers, pressure ulcers (stage II-IV) and diabetic ulcers.
- Cavity wounds.
- Surgical wounds.
- Superficial and partial thickness burns.
- Traumatic wounds.
- Exudates absorption in oncology wounds (eg. Fungating cutaneous tumors, cutaneous metastases and Kaposi's sarcomas).

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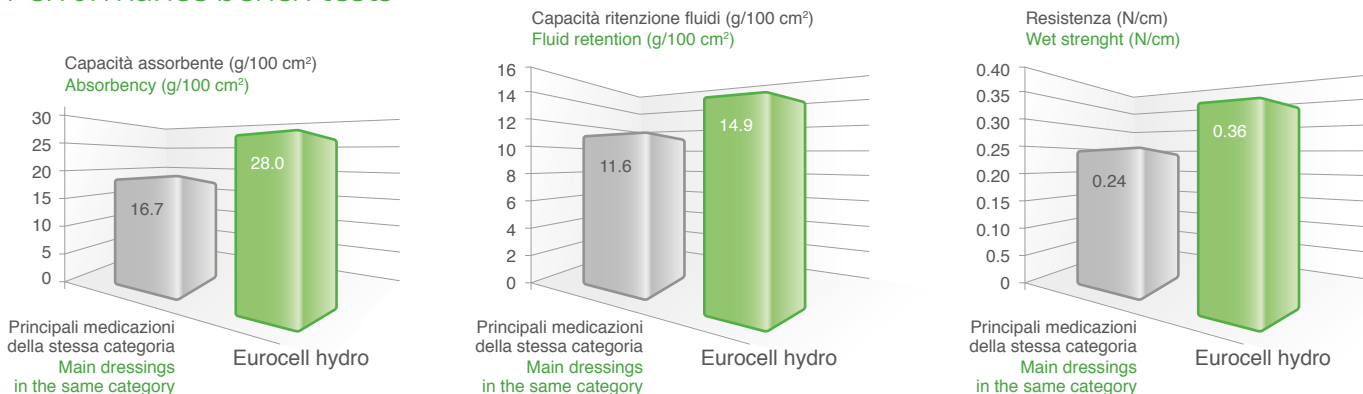
Comparison testing has demonstrated that Eurocell Hydro CMC wound dressing has an excellent performance over the leading gelling dressing in:

- **Absorbency, capacity to absorb high levels of exudates during use.**
- **Fluid Retention, capacity to retain exudate under compression.**
- **Wet Strength, to remain intact when being removed from the wound bed.**

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Eurocell Hydro ideal for superficial and partial thickness wounds

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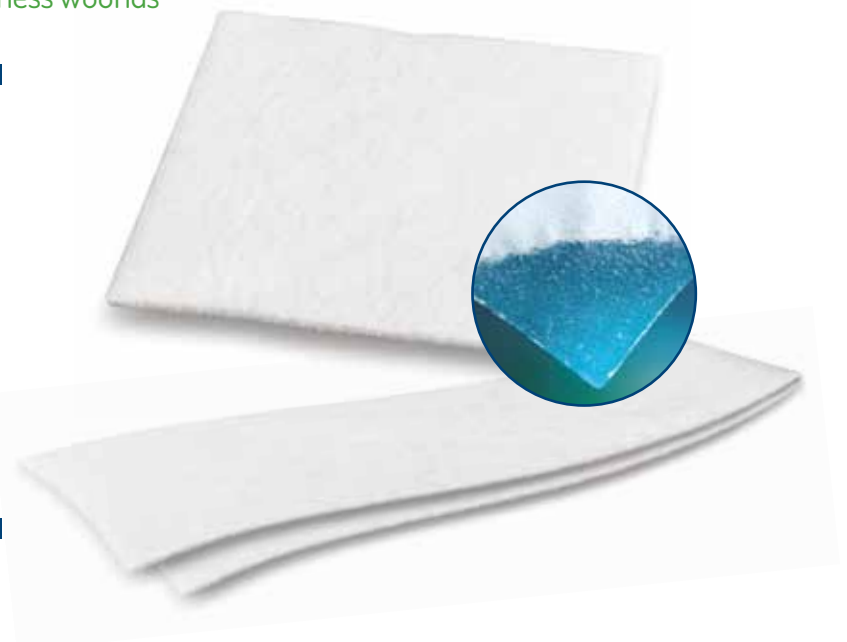
Eurocell Hydro roll CMC roll ideal for cavity wounds

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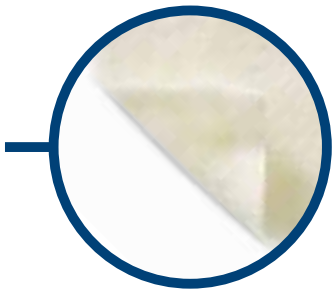
Eurosorb is a primary dressing in single swab made of 100% calcium alginate, sterile, soft and highly absorbent fibres. In contact with exudate, Eurosorb turns into a biocompatible gel, which maintains a moist wound healing environment. Promotes the healing process and the formation of newly formed tissue.

Eurosorb can absorb exudates up to 10 times its weight and can remain in situ up to 7 days, depending on the patient's condition and the amount of exudates.

Eurosorb is an integral alginate absorbing exudates between and within its fibres. The advantage is that it can be removed in one piece and the structure of Eurosorb reduces to the minimum the risk of release of alginate fibres into the wound and does not cause trauma to newly formed tissue, does not adhere to the wound bed and does not release exudates, preventing the risk of skin maceration. Eurosorb is latex free.

articolo	item no.	♀	misure	size	♀	conf.	pkg.
----------	----------	---	--------	------	---	-------	------

501505			cm 5 x cm 5			10 pz/pcs	
--------	--	--	-------------	--	--	-----------	--

501510			cm 10 x cm 10			10 pz/pcs	
--------	--	--	---------------	--	--	-----------	--

501515			cm 15 x cm 15			5 pz/pcs	
--------	--	--	---------------	--	--	----------	--

501520			cm 10 x cm 20			5 pz/pcs	
--------	--	--	---------------	--	--	----------	--

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Eurosorb roll is a primary dressing in ribbon made of 100% calcium alginate, sterile, soft and highly absorbent fibres. In contact with exudate, Eurosorb roll turns into a biocompatible gel, which maintains a moist wound healing environment. Promotes the healing process and the formation of newly formed tissue. Eurosorb roll can absorb exudates up to 10 times its weight and can remain in situ up to 7 days, depending on the patient's condition and the amount of exudates.

Eurosorb roll is an integral alginate absorbing exudates between and within its fibres. The advantage is that it can be removed in one piece and the structure of Eurosorb roll reduces to the minimum the risk of release of alginate fibres into the wound and does not cause trauma to newly formed tissue, does not adhere to the wound bed and does not release exudates, preventing the risk of skin maceration. Eurosorb roll is latex free.

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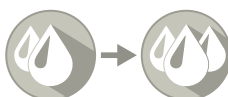
Indications

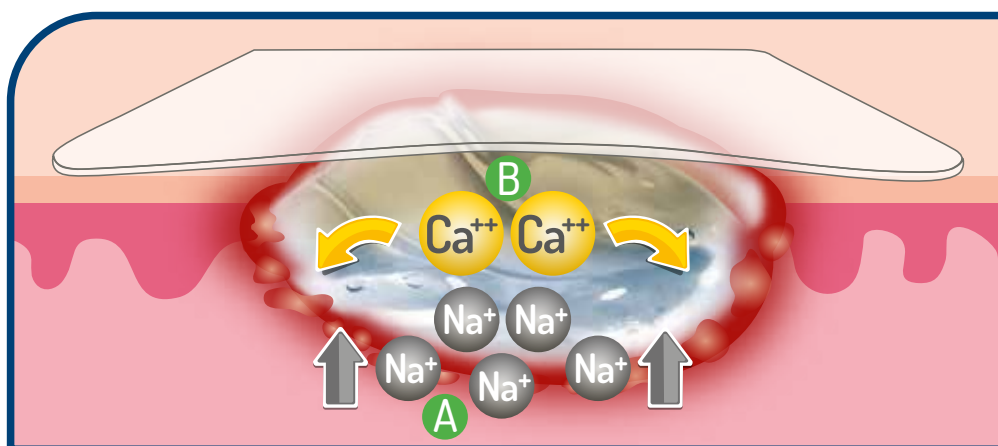
- Eurosoorb is indicated for the treatment of Pressure ulcers, Leg ulcers, Cavity wounds, Skin donor sites.
- Eurosoorb roll is indicated for cavity wounds.

Contraindications

- Do not use on Surgical implantation, Third degree burns, Dry wounds.

Property and benefits

- High Absorbency.
- High wet strength.
- Retains Exudate
- Turns into a gel that will not stick to the wound.
- Maintains a 'Moist Wound Healing' environment.
- Donates calcium to the wound.
- Longer wear time between dressings.
- One-piece removal.
- Minimal residues in the wound bed.
- Reduce risk of maceration.
- Reduce trauma during a dressing change.
- Reduce requirement for debridement.
- Optimum wound healing conditions.
- Helps to stop minor bleeding.

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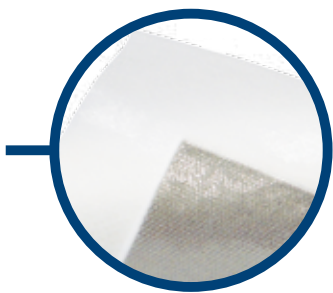
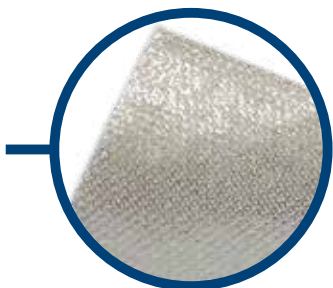
Hydro-activated micronized silver dressing
with adhesive border

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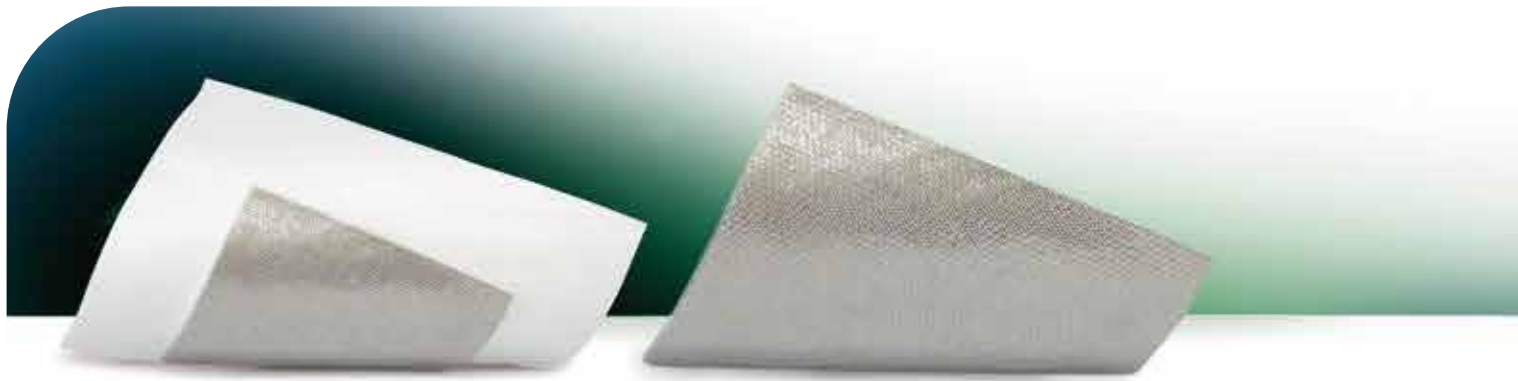
Hydro-activated silver micronized dressing
without adhesive border

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Silvermed is a ready for use sterile island dressing with a central absorbent and non-adherent pad with silver ions. The pad has an antibacterial effect that uses silver advanced technology to promote healing in a moist controlled environment. Silver is hydro-activated in a controlled manner by exudate. Silver ions are released in proportion to the amount of exudate and bacterial burden directly into the wound bed. It creates a moist environment ideal for a rapid healing and acts as a barrier to bacterial penetration, preventing infection to the wound; Thanks to its metallic silver micronized and non-adherent layer it does not stick to the wound, making dressing change atraumatic. Silvermed does not release silver residues. Silvermed is latex free.

articolo	item no.	♀	misure	size	♀	conf.	pkg.
----------	----------	---	--------	------	---	-------	------

892063			cm 10 x cm 10			5 pz/pcs	
--------	--	--	---------------	--	--	----------	--

892066			cm 10 x cm 20			3 pz/pcs	
--------	--	--	---------------	--	--	----------	--

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Indicated for

Infected wounds, dry to minimal drainage wounds, partial thickness pressure ulcers, first and second degree burns, infected wounds and ulcers, skin erosion caused by over-infection, granulating ulcers, skin donor areas, dermabrasions in general.

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Hydro-activated silver micronized dressing

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Sterile dressing, without adhesive border, absorbent and non-adherent pad with silver ions. The pad has an antibacterial effect that uses silver advanced technology to promote healing in a moist controlled environment. Silver is hydro-activated in a controlled manner by exudate. Silver ions are released in proportion to the amount of exudate and bacterial burden directly into the wound bed. It creates a moist environment ideal for a rapid healing and acts as a barrier to bacterial penetration, preventing infection to the wound; Thanks to its metallic silver micronized and non-adherent layer it does not stick to the wound, making dressing change atraumatic. Silvermed sb does not release silver residues. Silvermed sb è latex free.

articolo	item no.	♀ misure	size	♀ conf.	pkg.
----------	----------	----------	------	---------	------

892070		cm 5 x cm 5		25 pz/pcs	
--------	--	-------------	--	-----------	--

892073		cm 10 x cm 10		5 pz/pcs	
--------	--	---------------	--	----------	--

892076		cm 10 x cm 20		3 pz/pcs	
--------	--	---------------	--	----------	--

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Features

- Antimicrobial effect of silver reduces risk of infection.
- Prophylaxis against infection especially for wounds at high risk of infection.
- Biocompatible and complying to ISO 10993.

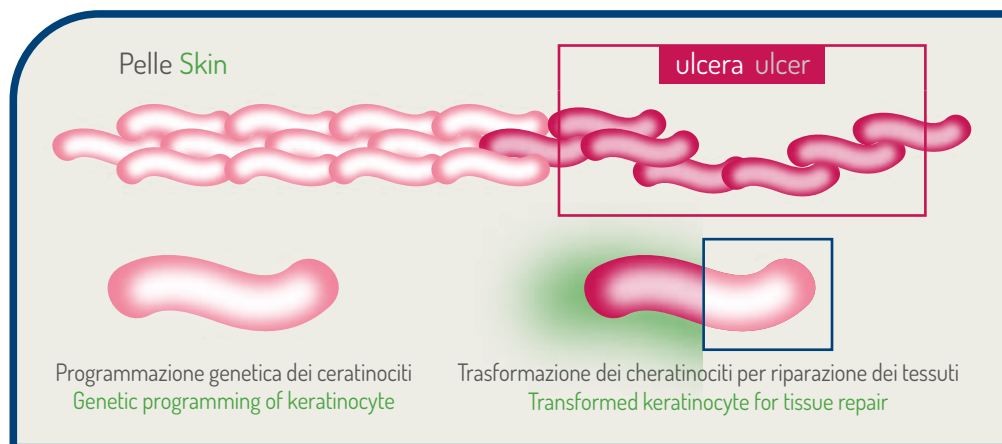
Property and benefits

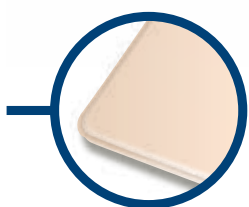
- Permeable to air and water vapor.
- Hypoallergenic.
- Soft.
- Atraumatic change.
- Dressing adapts to any anatomical part of the body.
- It is ready to use and does not require any activation.
- It stimulates the formation of granulation tissue and consequently re-epithelialization.
- It reduces the amount of devitalized tissue.

Contraindications

- There are no absolute contraindications for using Silvermed.
- Do not use for individuals with known sensitivity to any of the dressing components.
- Due to presence of antimicrobial silver ions, the experience with neonates/small babies is limited.
- Do not use on patients who are undergoing MRI.

Silvermed e Silvermed sb stimolano il processo di guarigione
Silvermed and Silvermed sb stimulates the healing process

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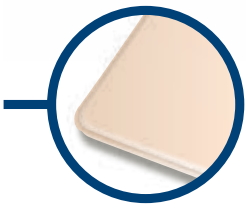
ipoallergenico

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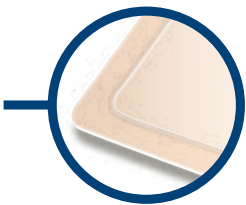
Euroderm foam

Multilayered hydrocellular polyurethane foam with full hydroactive area

Euroderm foam plus sacral

Multilayered hydrocellular shaped polyurethane foam for the sacral area

Euroderm foam plus

Multilayered hydrocellular polyurethane foam with adhesive border

Euroderm foam plus concave

Multilayered hydrocellular shaped polyurethane foam for anatomical areas

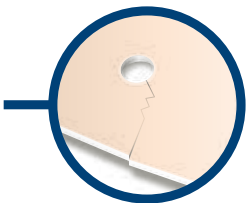
Euroderm foam heel

Multilayered hydrocellular shaped polyurethane foam for anatomical areas

Euroderm foam cavity

Non-adhesive polyurethane foam dressing for cavity wound

Euroderm foam tracheo

Key-hole apertured hydrocellular polyurethane foam for tracheostomy

Euroderm foam

Foam dressing consisting of an highly absorbent soft polyurethane foam and of a breathable polyurethane film which is water and bacteria impermeable but vapour permeable. Fluids are uniformly distributed into the foam dressing which, thus, perfectly conforms to the wound-bed maintaining the optimal moist wound environment for healing process. The high absorption of the foam dressing provides superior fluid management for a longer period of time. This minimized discomfort for the patient and results more cost effective.

The distinctive "flat-edged" design facilitates the applications of a secondary fixation dressing (tape, polyurethane film – Euroderm or Euroderm roll – or cohesive bandage) and provides an additional protection against leakage. The thickness of the foam ensures a cushioning effect.

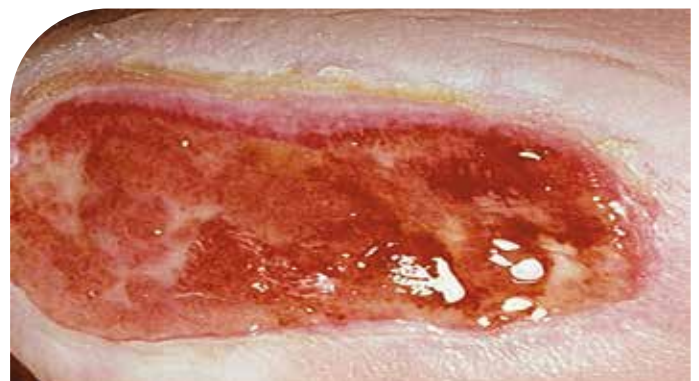
Euroderm foam reduces pressure over the wound area, thereby reducing friction and sheer stresses and giving excellent patient comfort and protection. For the treatment of necrotic or fibrinous wounds it is advisable to use it in combination with Hydromed gel.

The dressing can stay in place for up to 7 days. The frequency of change results from the clinical condition, the type of wound and the quantity of exudates.

Indications: Stage I-IV pressure ulcers, surgical wounds, vascular ulcers, 1st and 2nd degree burns, wounds on fragile skin, donor sites.

item no.	size	pkg.
893010	cm 10 x cm 10	10 pcs
893012	cm 10 x cm 20	5 pcs
893015	cm 15 x cm 15	5 pcs
893020	cm 20 x cm 20	3 pcs
892910	cm 10 x cm 10	5 pcs

latex free

hypo-allergenic

moderate exudate

heavy exudate

Multilayered hydrocellular polyurethane foam

Euroderm foam plus

Multilayered hydrocellular polyurethane foam combining the features of Euomed foam with an adhesive border which grants an effective fixation, even on difficult-to-dress areas. Foam dressing consisting of an hypoallergenic semi-permeable adhesive polyurethane top layer and of a soft highly absorbent foam pad.

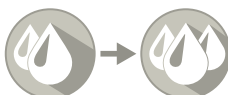
Fluids are uniformly distributed into polyurethane foam pad which, thus, perfectly conforms to the wound-bed maintaining the optimal moist wound environment for healing process. The high absorption of the foam dressing provides superior fluids management for a longer period of time. This minimizes discomfort for the patient and results more cost effective. The thickness of the foam ensures cushioning effect. Euroderm foam plus reduces pressure over the wound area, thereby reducing friction and sheer stresses and giving excellent patient comfort and protection. Do not use to dress wounds with a fragile periwound area. Euroderm foam plus can be used as a secondary dressing in combination with cavitory dressing (gauzes or alginates – Eurosorb roll), or in combination with compressive bandages for the treatment of arterial and venous stasis leg ulcers. For the treatment of necrotic or fibrinous wounds it is advisable to use it in combination with Hydromed gel. The dressing can stay in place for up to 7 days. The frequency of change results from the clinical conditions, the type of wound and the quantity of exudates.

Indications: Stage I-IV pressure ulcers, surgical wounds, vascular ulcers, 1st and 2nd degree burns and donor sites.

item no.	size	pkg.
893110	cm 10 x cm 10	10 pcs
893112	cm 10 x cm 20	5 pcs
893115	cm 15 x cm 15	5 pcs
893118	cm 18 x cm 18	3 pcs
893120	cm 20 x cm 20	3 pcs

moderate exudate

heavy exudate

hypo-allergenic

latex free

Euroderm foam heel

Multilayered hydrocellular shaped polyurethane foam for the management of wounds on the heel or any similar anatomical areas.

Foam dressing consisting of an highly absorbent soft polyurethane foam layer and of a breathable polyurethane film which is water and bacteria impermeable but vapour permeable.

Euroderm foam heel is non-adhesive. This allows an atraumatic and pain free dressing removal causing minimal distress to the patient, even on fragile tissues.

The high absorption of the foam dressing provides superior fluid management for a longer period of time. This minimized discomfort for the patient and results more cost effective.

The thickness of the foam ensures a cushioning effect.

Euroderm foam heel reduces pressure over the heel area, thereby reducing friction and sheer stresses and giving excellent patient comfort and protection. It needs a secondary fixation dressing (tape, polyurethane films – Euroderm or Euroderm roll- or cohesive bandage).

For the treatment of necrotic or fibrinous wounds it is advisable to use it in combination with Hydromed gel. The dressing can stay in place for up to 7 days. The frequency of change results from the clinical conditions, the type of wound and the quantity of exudates.

Indications: Diabetic foot ulcers, stage I-IV pressure ulcers, prevention of pressure ulcers on a patient with partial ability to move.

item no.	size	pkg.
893060	cm 12 x cm 13	3 pcs

latex free

hypo-allergenic

moderate exudate

heavy exudate

Euroderm foam tracheo

“Key-hole” apertured hydrocellular polyurethane foam designed to fit neatly around the tracheostomy or the drainage tube.

Foam dressing consisting of an highly absorbent soft polyurethane foam layer and of a breathable polyurethane film which is water and bacteria impermeable but vapour permeable.

Fluids are uniformly distributed into the foam dressing which, thus, perfectly conforms to the tracheostomy or drainage tube assuring superior fluid management for a longer period of time.

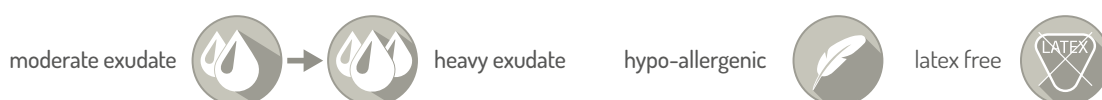
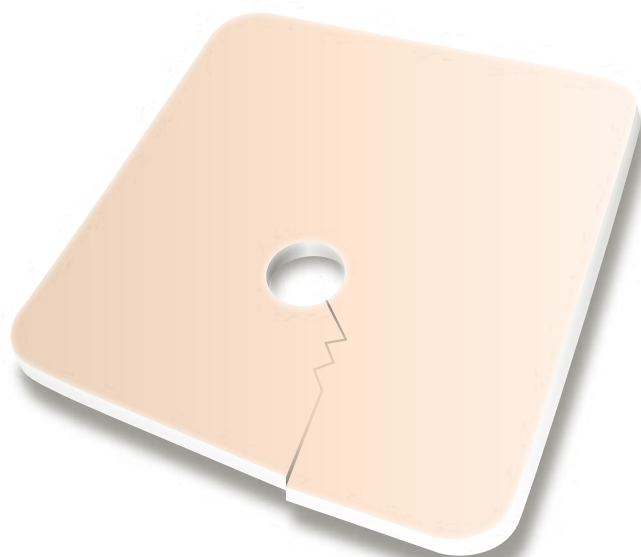
This minimized discomfort for the patient and results more cost effective

The dressing can stay in place for up to 7 days.

The frequency of change results from the clinical conditions, the type of wound and the quantity of exudates.

Indications: Fluids management around tracheostomy and drainage sites.

item no.	size	pkg.
893040	cm 8,5 x cm 8,5	10 pcs
893035	cm 6,5 x cm 6,5	10 pcs

Euroderm foam plus sacral

Multilayered hydrocellular shaped polyurethane foam with adhesive border combining the features of Euroderm foam with the ease of an anatomical design well suited for the sacral area. Foam dressing consisting of an hypoallergenic semi-permeable adhesive polyurethane top layer and of a soft highly absorbent shaped foam pad.

The adhesive polyurethane top layer acts as barrier against external contaminations and fluids, therefore making Euroderm foam plus sacral well suited for wounds of the sacral area where a waterproof covering is recommended.

The anatomical design prevents accidental removals of the dressing caused by friction' strengths and provides superior fluid management for a longer period of time. This minimized discomfort for the patient and results more cost effective. The thickness of the foam ensures a cushioning effect.

Euroderm foam plus sacral reduces pressure over the sacral area, thereby reducing friction and sheer stresses and giving excellent patient comfort and protection.

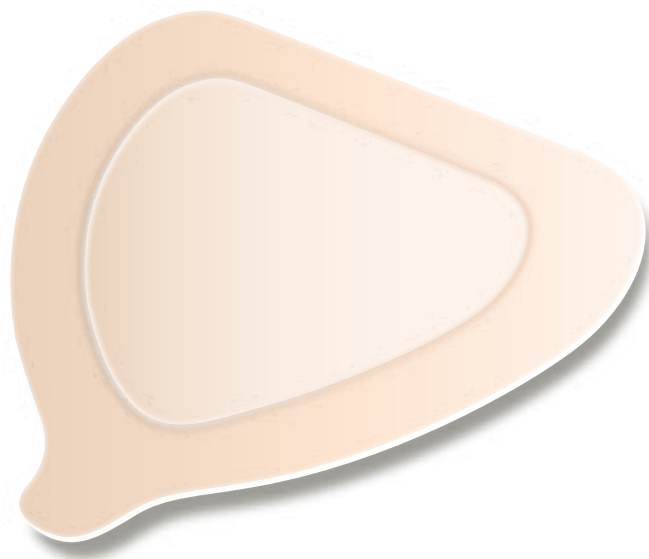
Euroderm foam plus sacral can be used as a secondary dressing in combination with cavitory dressing (gauzes or alginates – EuroSorb Roll) and for the treatment of necrotic or fibrinous wounds it is advisable to use it in combination with Hydromed gel.

The dressing can stay in place for up to 5 days; anyhow a frequent inspection of the wound is recommended. The frequency of change results from the clinical conditions, the type of wound and the quantity of exudates.

Indications: Pressure ulcers of the sacral area, stage I-IV pressure ulcers, surgical wounds, 1st and 2nd degree burns and donor sites.

item no.	size	pkg.
----------	------	------

893150	cm 22 x cm 22	3 pcs
--------	---------------	-------

moderate exudate

heavy exudate

hypo-allergenic

latex free

Multilayered hydrocellular polyurethane foam

Euroderm foam plus concave

Multilayered hydrocellular shaped polyurethane foam with adhesive border combining the features of Euroderm foam with the ease of an anatomical design well suited for the management of wounds on the heel or on any similar anatomical areas. Foam dressing consisting of an hypoallergenic semi-permeable adhesive polyurethane top layer and of a soft highly absorbent shaped foam pad. The anatomical design prevents accidental removals of the dressing caused by friction' strengths and provides superior fluid management for a longer period of time. This minimized discomfort for the patient and results more cost effective. The thickness of the foam ensures cushioning effect.

Euroderm foam plus concave reduces pressure over the wound area, thereby reducing friction and shear stresses and giving excellent patient comfort and protection. Do not use to dress wounds with a fragile periwound area. Euroderm foam plus concave can be used as a secondary dressing in combination with cavitory dressing (gauzes or alginates – EuroSorb Roll), or in combination with compressive bandages for the treatment of arterial and venous stasis leg ulcers.

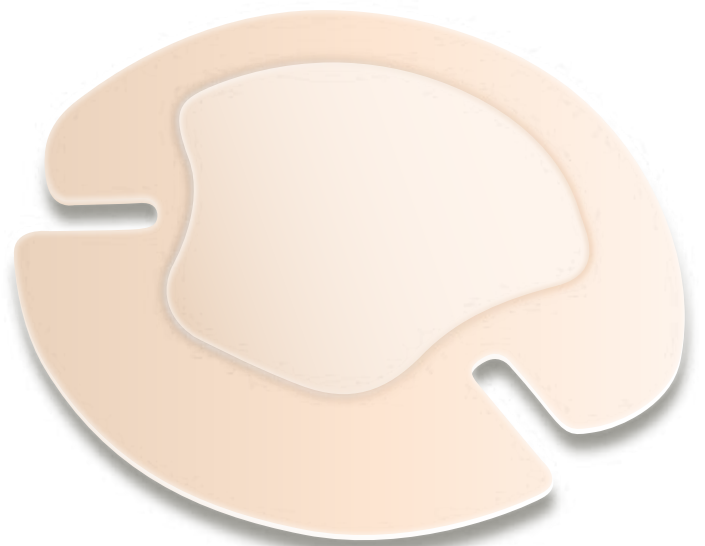
For the treatment of necrotic or fibrinous wounds it is advisable to use it in combination with Hydromed gel. The dressing can stay in place for up to 7 days.

The frequency of change results from the clinical conditions, the type of wound and the quantity of exudates.

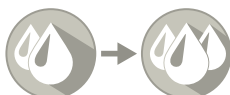
Indications: Diabetic foot ulcers, stage I-IV pressure ulcers, prevention of pressure ulcers on a patient with partial ability to move, surgical wounds, 1st and 2nd degree burns and donor sites.

item no.	size	pkg.
----------	------	------

893140	cm 18 x cm 16,5	3 pcs
--------	-----------------	-------

moderate exudate

heavy exudate

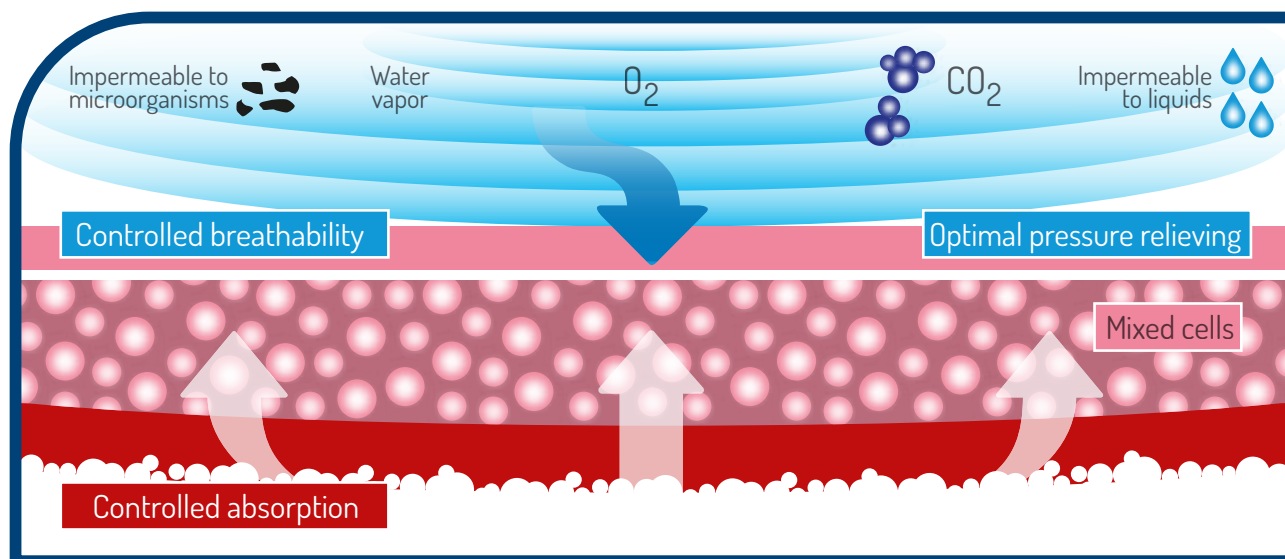
hypo-allergenic

latex free

The best method for management of wounds from light high exudate

Euroderm foam cavity

Sterile, non-adhesive polyurethane foam dressing composed by an hydroactive surface.

It is highly absorbent, for medium or high level of exudate.

Euroderm cavity foam conforms perfectly to the cavity wound and to the different anatomical part of the body with excellent comfort for the patient, it does not adhere to the wound surface.

The product has a good conformability and softness.

Furthermore, the thickness of the foam creates the bearing effect, which reduces the pressure on the lesion and avoid friction.

Euroderm foam cavity is latex free.

Indicazioni: To be used for the treatment of cavity wounds such as skin donor sites, pressure ulcers, wounds after excision of fistulas and abscesses, venous ulcers, diabetic ulcers, trauma and postoperative wounds, dehiscent wounds. Euroderm foam cavity can be applied using a secondary fixation dressing.

In the occurrence of necrotic or fibrinous tissue injury, it is recommended to use in combination with Hydromed gel.

item no.	size	pkg.
893205	Ø cm 5	10 pcs
893210	Ø cm 10	5 pcs
893212	cm 7,5 x cm 10	3 pcs

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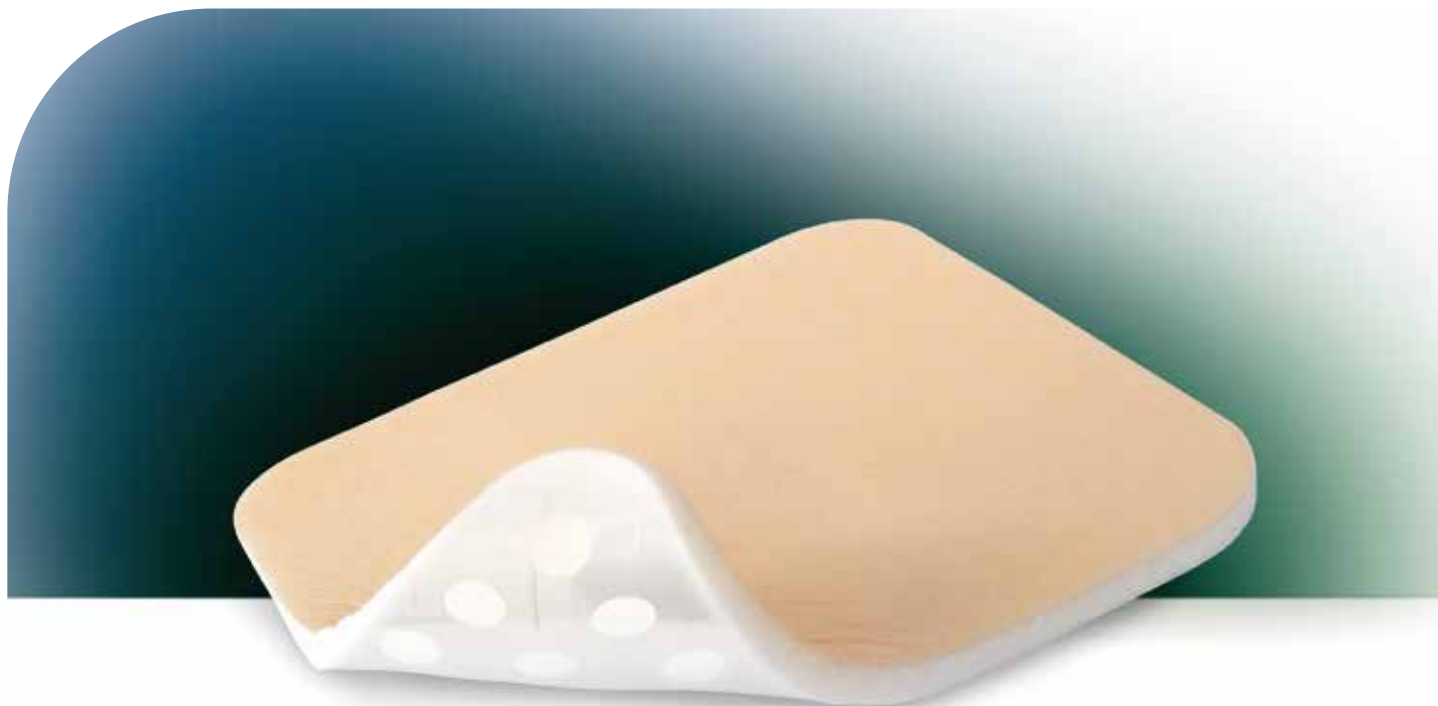
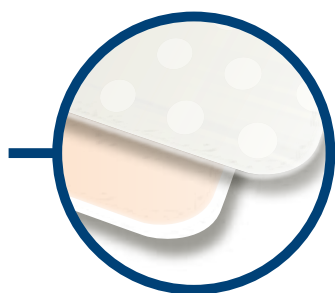
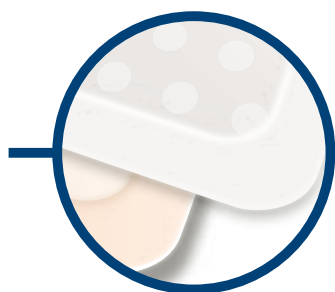
Highly absorbent dressing made of polyurethane foam and soft silicone gel

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Highly absorbent dressing made of polyurethane foam and soft silicone gel with adhesive border

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Euroderm foam silicone is an highly absorbent dressing made of polyurethane foam with polyurethane top sheet and perforated soft silicone gel wound contact layer. The dressing does not adheres to wound bed thereby minimizing disruption of healing tissue. The outer surface is made of breathable polyurethane film, which is waterproof, bacteria proof and viral barrier, while the wound contact layer is perforated soft silicone gel layer to prevent adherence to the wound, allow painless and atraumatic removal allowing the passage of exudates to the absorbent foam layer.

Advantages

- Perforated silicone film, allow free passage of wound exudate to the foam so prevents skin maceration.
- Excellent Absorption capacity
- Gentle soft Silicone gel allows painless removal.
- Provides a cushioning effect.
- Maintains moist wound Optimal MVTR (Moisture Vapour Transmission Rate)
- Long wear time and minimal patient trauma.
- High fluid handling capacity.
- Cost effective

Indications:

For the management of moderate to heavily exuding partial to full thickness wounds including:

- Suitable for use on fragile skin.
- Stage I - IV pressure ulcers, neuropathic (diabetic), Venous and Arterial leg ulcers.
- Diabetic foot ulcers.
- Suitable for use with compression.
- First/Second degree burns.
- Skin tears.
- Surgical wounds.
- Donor sites.

Contraindications

Do not use Euroderm foam silicone dressings with oxidizing agents such as hypochlorite solutions or hydrogen peroxide, as these can break down the absorbent polyurethane component of the dressing.

Precaution

In case of infection, Euroderm foam silicone still can be used but under medical supervision and with antimicrobial conjunctive therapy.

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Euroderm foam plus silicone is an absorbent, polyurethane foam dressing laminated with perforated soft silicone gel wound contact layer. This island adhesive dressing allows easy fixation without the use of traditional adhesive tapes or fixation dressing and provides an atraumatic-dressing removal. The outermost layer of the dressing is made of a bi-laminate polyurethane, which is waterproof, bacteria proof and virus proof.

Features

- The soft Silicone gel layer allows atraumatic dressing removal.
- The perforated silicone film allows free passage of wound exudate to the foam so prevents skin maceration.
- High absorption capacity polyurethane foam also provides a cushioning effect.
- Highly breathable top sheet allows evaporation of absorbed exudates, thus enhances fluid handling capacity of the dressing.
- Being waterproof, the top layer prevents strike through and allows showering while the dressing is in place.

Indications:

Euroderm foam plus silicone is suitable for dressing many types of exuding wounds including leg and pressure ulcers, surgical incisions and traumatic wounds resulting in skin loss, also may be used under compression bandaging.

Contraindications

Do not use Euroderm foam plus silicone dressings with oxidizing agents such as hypochlorite solutions or hydrogen peroxide, as these can break down the absorbent polyurethane component of the dressing.

Precaution

In case of infection, Euroderm foam plus silicone still can be used but under medical supervision and with antimicrobial conjunctive therapy.

Frequency of Change

The interval between dressing changes will normally be determined by the degree of exudate produced but the dressing may be left undisturbed for several days on clean lightly exuding wounds.

How we supply it

Euroderm foam plus silicone is supplied individually wrapped in heat sealable pouch, sterilized by ethylene oxide.

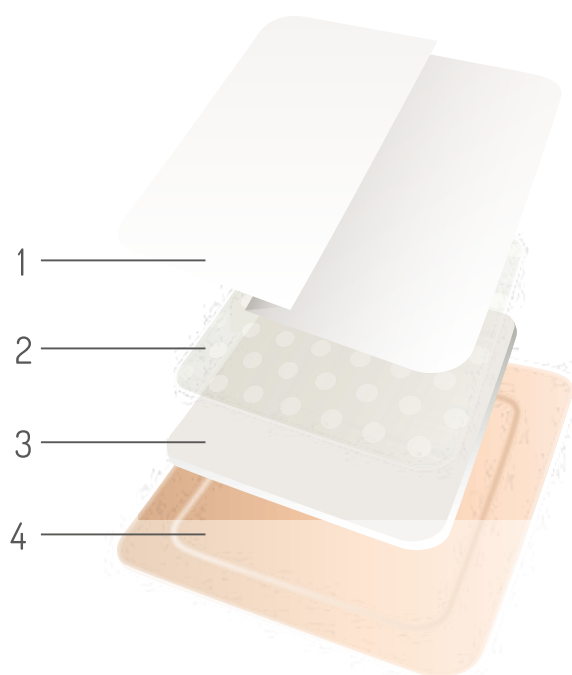
articolo	item no.	♀ misure size	♀ conf.pkg.
----------	----------	---------------	-------------

893410		cm 10 x cm 10	10 pz/pcs
--------	--	---------------	-----------

893412		cm 10 x cm 20	5 pz/pcs
--------	--	---------------	----------

893415		cm 15 x cm 15	5 pz/pcs
--------	--	---------------	----------

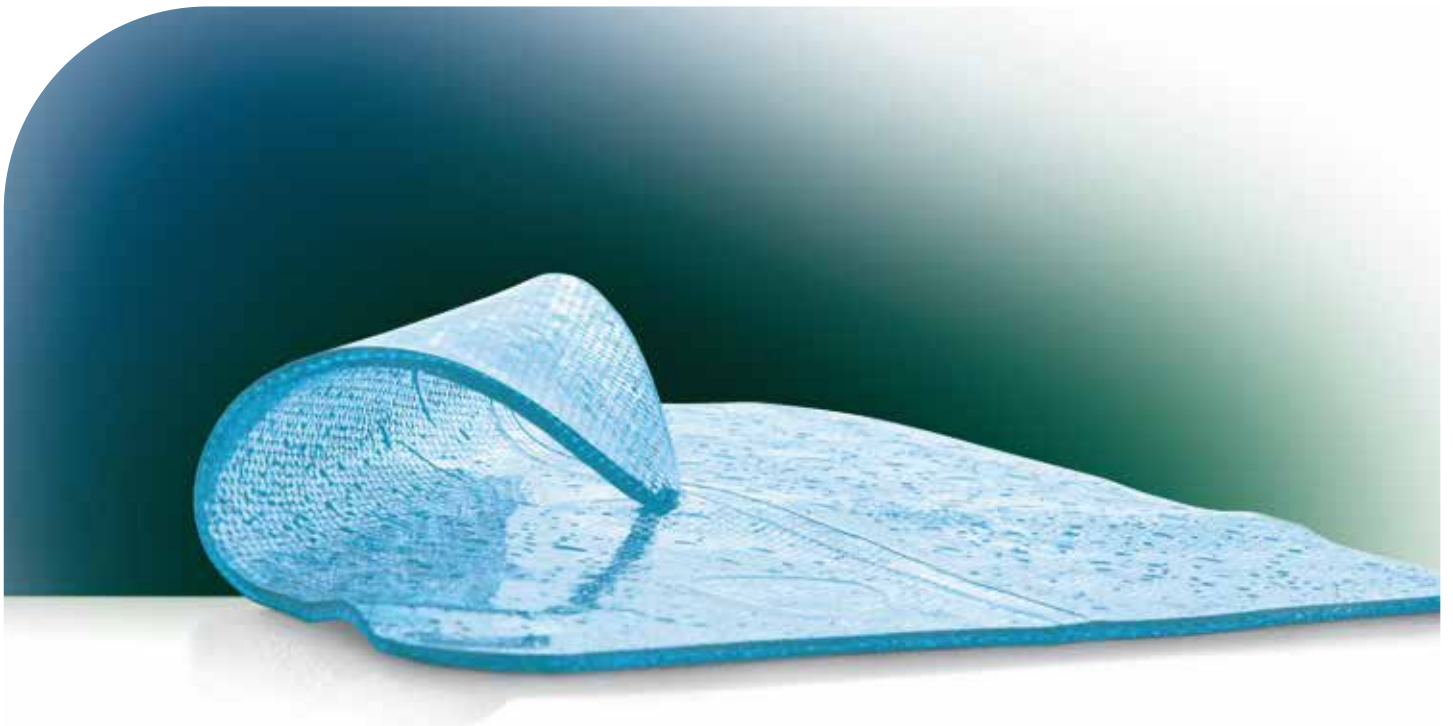
893420		cm 20 x cm 20	3 pz/pcs
--------	--	---------------	----------

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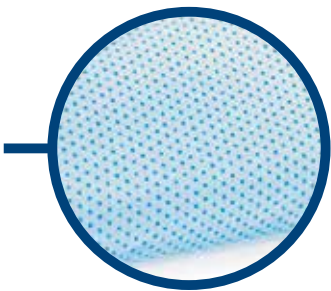
Sterile dressing with high water content gel

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Sterile transparent amorphous gel suitable for the removal of necrotic tissue

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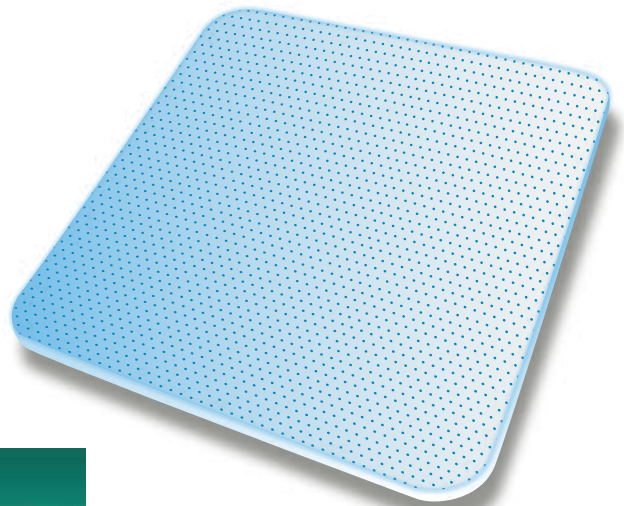
Hydrogel dressing designed to manage pain, moisture and protease activity in low exudate or sloughy wounds as absorb quickly and locally. This reduces the risk of periwound maceration. Suitable for: 1st and 2nd degree burns, abrasions,dermabrasions,radiodermatitis,tirpid wounds and ulcers with fibrinous or necrotic bases. Refreshing. Unlike typical sheet hydrogels, hydromed absorb quicly and locally. This reduces the risk of periwound maceration. Hydromed is latex free.

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Sterile transparent amorphous gel suitable for the removal of necrotic tissue.

The gel creates a moist wound environment, preventing eschar formation and encouraging healing. By keeping fragile granulation tissue moist, the gel helps to create a moist wound healing environment, allowing the migration of epithelializing cells.

Hydromed gel is latex free.

Features

- Debriding action: Hydromed gel rehydrates necrotic tissue and aids debridement.
- Desloughing action: Hydromed gel absorbs slough and exudate without damaging fragile granulation tissue.
- Creates moist wound healing environment: Hydromed gel helps to create an optimal moist wound management environment, which promotes healing, reduces the likelihood of scarring and helps soothe pain.
- Stable and counteract leakage and maceration.
- Easy to remove. The gel is easy to remove due to high cohesion even after absorption of debris and exudate.
- Good donation properties.
- High viscosity.

Indications

- Hydromed gel is primarily intended for the treatment of necrotic and sloughy wounds, e.g. leg ulcers, pressure ulcers, cavity wounds, leg ulcers, and non-infected diabetic foot ulcers.
- Hydromed gel may also be used on 1st and 2nd degree burns.

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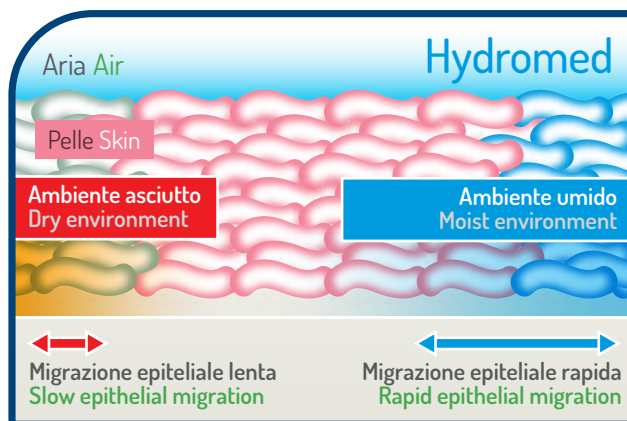
Indications

- Suitable for treatment of chronic and slow healing wounds in granulation epitheliazation phase.
- Due to a three dimensional structure and high water content Hydromed is suitable for use in debriding and for secondo-degree burns.

Contraindications

- Hydromed should not be used on infect wounds nor on third degree burns.
- Also they should not be used on highly exuding wounds nor on exposed muscle, bones nor tendons.

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Benefit

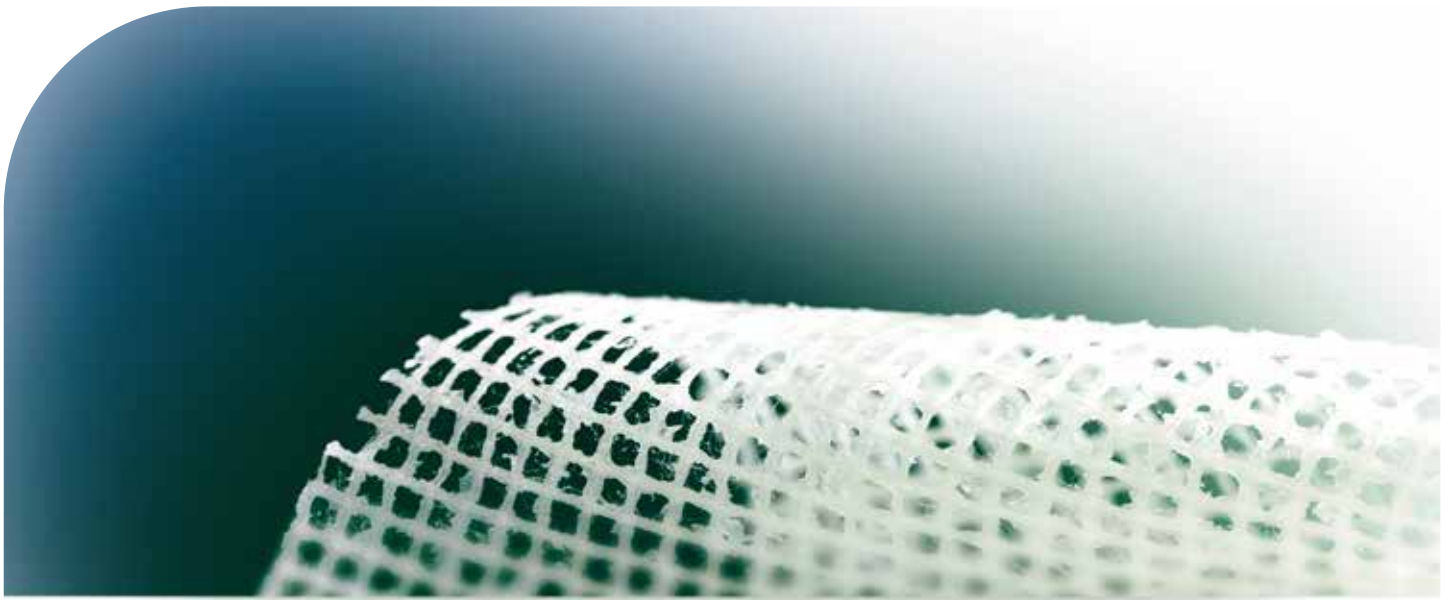
- Aids autolytic debridments facilitating faster healing rates.
- Minimises the need for sharp debridment.
- Easy to apply: remains in situ.

Precautions and warnings

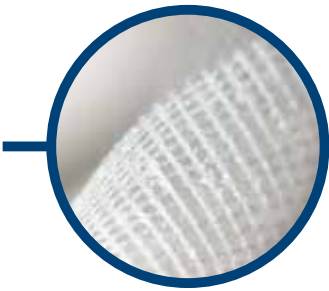
- May be used on patients with local and systemic infection under the discretion of healthcare professional.
- In case of allergic reactions, please discontinue use directly and consult your doctor.
- For external use only
- Do not reuse the product. Reuse may pose risk of infection.
- If packaging is broken do not use or re-sterilize product.

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Euronet Paraffin

is a 100 % absorbent cotton thread gauze, impregnated with white soft paraffin ointment. The gauze makes it a non-adherent and non-allergenic dressing. Euronet Paraffin is used as a primary wound contact layer in the treatment of burns, ulcers, skin grafts and a variety of traumatic injuries. The material is also used as a transfer medium for skin during grafting. Euronet Paraffin is latex free.

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Features:

- Individually packed in peel-able Aluminium pouches.
- Sterilized by gamma irradiation.
- Euronet Paraffin made of aperture leno cotton gauze that allows wound exudates to pass through.

Benefits:

- Euronet Paraffin is Impregnated with paraffin ointment which form a separate layer between wound and gauze preventing sticking with the wound and provide fast recovery.
- Protects newly formed tissue and minimize patient pain during changing the dressing.
- The leno-weave gauze allows the passage of the exudates to be absorbed by the secondary dressing, so the exudates do not dry on the wound bed.
- Can be cut to different sizes.

Indications:

- Euronet Paraffin is used as a primary wound contact layer in the treatment of burns, ulcers, skin grafts and a variety of traumatic injuries. The material is also used as a transfer medium for skin during grafting.
- Under the medical supervision of a healthcare professional, Euronet Paraffin can be used to manage infected wounds or wounds in which there is an increased risk of infection.
- Euronet Paraffin should be stored in a dry and cool place.

Contraindications:

- Although there are no contra-indications to the use of Euronet Paraffin, if the dressing is placed upon a heavily exuding wound its semi-occlusive nature may cause tissue maceration by preventing the free movement of exudates away from the surface of the wound.
- If Euronet Paraffin is left in position for prolonged periods of time, it can become adherent and cause tissue damage upon removal.

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Application:

- Open the pouch and take out the product.
- Remove the protector paper on both sides of the product and discard.
- Handle the impregnated gauze using an aseptic technique/using a forceps.
- Apply to the wound site.
- Cover with an absorbent secondary dressing e.g. Euroderm Foam.
- Fix using a fixation bandage or use Euroderm Foam Plus with adhesive border to eliminate the need for a fixation bandage.

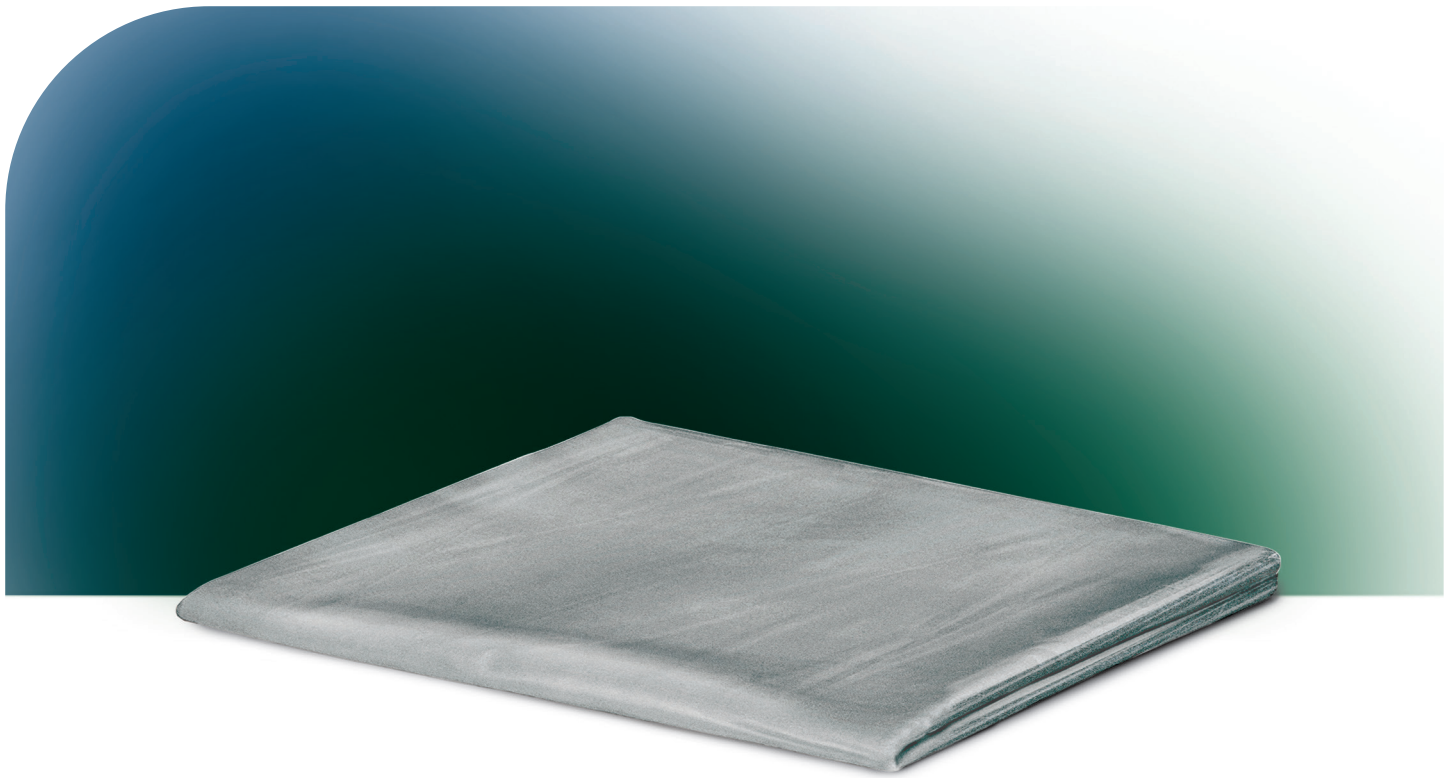
articolo	item no.	♀ misure size	♀ conf.pkg.	♀ astucci box
560001		cm 5 x cm 5	150 pz/pcs	16
560002		cm 5 x cm 10	100 pz/pcs	16
560003		cm 5 x cm 15	100 pz/pcs	16
560004		cm 5 x cm 20	50 pz/pcs	16
560010		cm 7,5 x cm 10	100 pz/pcs	16
560015		cm 7,5 x cm 15	50 pz/pcs	16
560020		cm 7,5 x cm 20	50 pz/pcs	16
560110		cm 10 x cm 10	100 pz/pcs	16
560115		cm 10 x cm 15	50 pz/pcs	16
560120		cm 10 x cm 20	50 pz/pcs	16
560140		cm 10 x cm 40	30 pz/pcs	16
560215		cm 20 x cm 15	50 pz/pcs	16
560220		cm 20 x cm 20	50 pz/pcs	16
560240		cm 20 x cm 40	30 pz/pcs	16
560315		cm 15 x cm 15	50 pz/pcs	16
560340		cm 15 x cm 40	30 pz/pcs	16
561005		cm 5 x cm 100	1 pz/pcs	48
561007		cm 7,5 x cm 100	1 pz/pcs	48
561010		cm 10 x cm 100	1 pz/pcs	48
561015		cm 15 x cm 100	1 pz/pcs	48
561020		cm 20 x cm 100	1 pz/pcs	48
562010		cm 10 x cm 200	1 pz/pcs	48
562015		cm 15 x cm 200	1 pz/pcs	48
562020		cm 20 x cm 200	1 pz/pcs	48
567007		cm 7,5 x cm 700	1 pz/pcs	24
567010		cm 10 x cm 700	1 pz/pcs	24
567015		cm 15 x cm 700	1 pz/pcs	24

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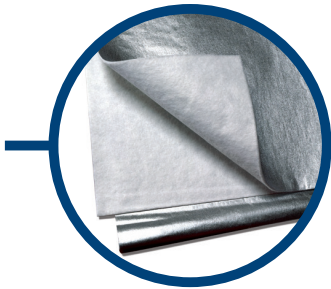
Eurodressing metal-KZ

Sterile dressing for large wounds and burns.

Eurodressing Metal - kz is a king size sterile dressing for large wounds and burns. Not adherent, hypoallergenic, breathable, antibacterial, excellent absorption capacity and soft. Composed by an Aluminium wound contact layer and an absorbent layer, greatly reduces the risk of infection.

Eurodressing metal - kz is latex free.

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Sterile dressing for large
wounds and burns

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Eurodressing metal-KZ

The aluminium layer, in direct contact with the wound or with the burn, allows the passage of exudate or blood without the aluminium layer sticking to the wound.

It promotes epithelialization and guarantees a rapid healing. Thanks to its fast and great absorption capacity, but not adherent, facilitates dressing change.

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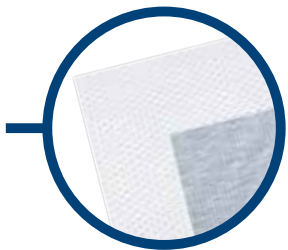
Euromed is a sterile post-operative island dressing with a absorbent central pad. The polyethylene wound contact layer enables the passage of blood or exudates into the absorbent pad while maintaining a good healing environment and minimal pain upon dressing removal.

Euromed is hypoallergenic, radiotranslucent, and breathable.

It consents a painless removal.

The high adaptability of the elastic backing material allows an excellent degree of fit to the skin and to irregular anatomical parts of the body.

To be used as mechanical barrier and to control the microenvironment of the wound, ulcer of the low parts of the body, burns and skin abrasions in general.

Euromed is supplied in a wide range of sizes with rounded edges, available in single envelope and ready to use.

Individually packed in a peel away pouch and sterilized by E.O. gas.

Euromed ensures gentle fixation without causing skin irritation or cutaneous alterations.

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Euromed I.V. is a sterile kit intended for a safe and functional management of insertion sites. It is composed by a sterile, radiotranslucent non-woven 100% polyester net with central pad, an extra pad anti-decubitus for superior comfort and an adhesive stripe in polyester net for the of the cannula that offer greater protection against accidental releases of the needle. The lengthwise cut allow a safe position of cannula and catheters. Euromed I.V. is latex free.

articolo	item no.	♀	misure	size	♀	conf.pkg.
717820			cm 9 X	cm 8		100 pz/pcs
718009			cm 6 X	cm 9		100 pz/pcs

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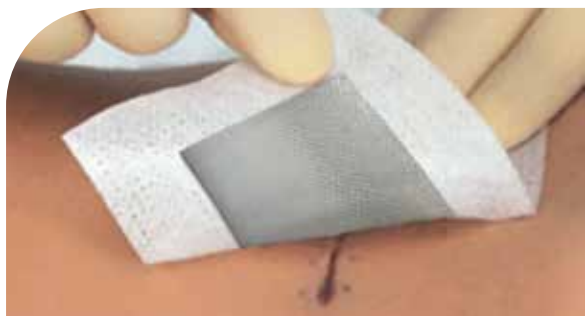
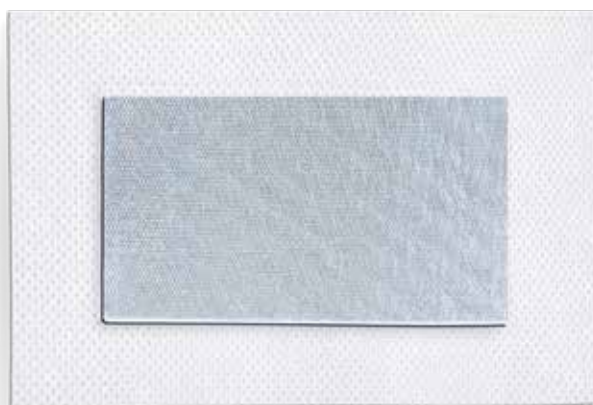
Eurometal is a sterile post-operative island dressing with a non-adherent highly absorbent aluminised central pad that does not stick to the wound thanks to a particular non-adherent layer. The aluminised central pad enables a rapid healing process. Eurometal is hypoallergenic, radiotranslucent, air-permeable and allows the transpiration of skin with a painless removal. The high adaptability of the elastic backing material allows an optimal fit of the plaster to the skin and to irregular anatomical parts of the body. To be used as mechanical barrier against bacteria and to control the microenvironment of the wound, ulcer of the low parts of the body, burns and skin abrasions in general. Eurometal is supplied in a wide range of sizes with rounded edges, available in single envelope and ready to use. Individually packed in a peel away pouch and sterilized by E.O. gas. Eurometal is latex free.

articolo	item no.	♀ misure	size	♀ conf.	pkg.
720000		cm 5 x cm 7,2		100 pz/pcs	
720001		cm 6 x cm 7		100 pz/pcs	
720002		cm 9 x cm 6		50 pz/pcs	
720003		cm 10 x cm 8		50 pz/pcs	
720004		cm 12 x cm 7		50 pz/pcs	
720005		cm 15 x cm 8		50 pz/pcs	
720006		cm 20 x cm 10		50 pz/pcs	
720007		cm 25 x cm 10		50 pz/pcs	
720008		cm 30 x cm 10		50 pz/pcs	

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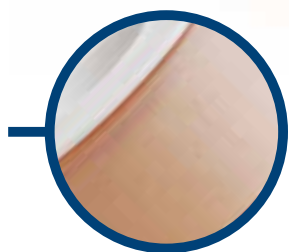
Benefits

- Radiotranslucent.
- Hypoallergenic.
- Easy to remove to avoid skin trauma. It does not release residues of glue.
- Easy to apply.
- Very soft and conformable.
- Does not affect skin movements.
- Breathable.
- Atraumatic dressing change.
- Highly absorbent.

Intended use

- Indicated for the absorption of exudates and to keep under control the wound microenvironment.
- Prophylaxis against infection especially for wounds at high risk of infection.
- Biocompatible and complying to ISO 10993.

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Rigid 100% cotton tape with pressure - sensitive zinc oxide adhesive mass for fixation

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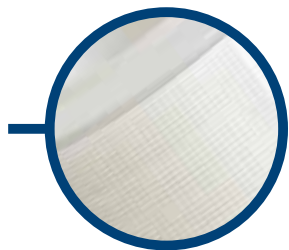
Rigid 100% acetate fabric with silky appearance, suitable for fixation

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White non-woven adhesive tape, suitable for fixation

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Transparent and micro-perforated polyethylene adhesive tape

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Euroreel is a rigid 100% cotton tape with pressure -sensitive zinc oxide adhesive mass for the fixation of cannulas, absorbent pads, bandages, venous catheters and probes. Euroreel is perfectly ventilated, hydrophobic and highly hypoallergenic which is well tolerated by the skin. The high degree of permeability to water vapour is guaranteed by the exclusive lined coating technic of the adhesive mass, which is uniformly coated in micrometrical spaced longitudinal lines. Radiotranslucent, tearable by hand in both direction, ease to remove from the skin avoiding skin trauma without the release of adhesive residues. It does not cause skin irritation. It has smooth edges.

Plastic spool guarantees hygiene. It does not adhere to latex gloves during use. Euroreel is also available in a 3 mm holes perforated version called Euroreel Perforato, which guarantees greater ventilation. Euroreel is also available in 100% acetate fabric version called Euroreel Acetato.

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Eurosilk is a rigid and 100% acetate fabric with silky appearance, suitable for the fixation of dressings, bandages, tubes, venous catheters and medical probes. Eurosilk uses a special polyacrylic adhesive mass without the addition of any chemical solvent. It is water-soluble, thus, ensuring the maximum level of security against allergic reactions. Eurosilk is highly hypoallergenic, has good adhesion strength, it is well tolerated by sensitive skin and very easy to tear by hand lengthwise and crosswise. It is also radiotranslucent, permeable to air and water vapour, thus, allowing the skin to breath. Painless removal without release of adhesive residues. It does not adhere to gloves during use. Plastic spool guarantees hygiene. Suitable for sensitive skin. It does not cause skin irritation. Eurosilk is latex free.

articolo	item no.	♀ misure	size	♂ conf.	pkg.
----------	----------	----------	------	---------	------

707001		cm	1,25 x m 5	36	pz/pcs
707002		cm	2,5 x m 5	24	pz/pcs
707005		cm	5 x m 5	12	pz/pcs
707007		cm	7,5 x m 5	6	pz/pcs
707010		cm	10 x m 5	6	pz/pcs
707011		cm	1,25 x m 9,14	36	pz/pcs
707012		cm	2,5 x m 9,14	24	pz/pcs
707015		cm	5 x m 9,14	12	pz/pcs
707017		cm	7,5 x m 9,14	6	pz/pcs
707020		cm	10 x m 9,14	6	pz/pcs

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Europor is a white non-woven adhesive tape, suitable for the fixation of dressings, bandages, tubes, venous catheters and medical probes. Europor uses a special polyacrylic adhesive mass without the addition of any chemical solvent. It is water-soluble, thus, ensuring the maximum level of security against allergic reactions. Europor is highly hypoallergenic, has good adhesion strength, suitable for sensitive skin and very easy to tear by hand. It is also radiotranslucent, permeable to air and water vapour, thus, allowing the skin to breath. Suitable for sensitive skin. It does not cause skin irritation. Painless removal without release of adhesive residues. It does not adhere to gloves during use. Plastic spool guarantees hygiene. Europor is latex free.

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Eurotrasp is a transparent and micro-perforated polyethylene adhesive tape, suitable for the fixation of dressings, bandages, tubes, venous catheters and medical probes. The transparency allows a complete visibility of the covered surface. It is quick and easy to tear - even when using surgical gloves - both in transversal and longitudinal way, as well as in the middle. Eurotrasp is coated with a hypoallergenic polyacrylic adhesive mass. Permeable to air and to water vapour but impermeable to liquids.

Eurotrasp it has a very good adhesion strength. It is also radiotranslucent, highly permeable to air and water vapour, thus, allowing the skin to breath. Painless removal without release of adhesive residues.

It does not adhere to gloves during use. Plastic spool guarantees hygiene.

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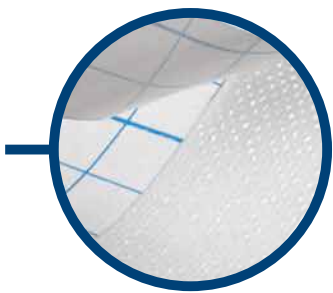
Elastic adhesive fixation plaster

Eurofix is an elastic adhesive polyester net, highly breathable.

It is very soft, skin respectful and highly hypoallergenic. Its crosswise elasticity allows the maximum adaptability of the plaster to the skin and to irregular anatomical parts of the body, ensuring the highest moisture and vapor permeability and correct dressing fixation. Water or secretions do not weaken the special adhesive mass of Eurofix and therefore the plaster keeps intact its original fixation characteristics.

The precut blue squared printed-paper protects the adhesive side.

Eurofix have been especially designed for an easy fixation of primary dressings helping an accurate cut of the plaster in one's favorite size.

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Benefits

- Easy to remove to avoid skin trauma. It does not release residues of glue.
- Highly hypoallergenic.
- Easy to apply on joints and difficult body contours, does not restrict or limit body movement.
- Radiotranslucent.
- Highly permeable to air and water vapor. Highly breathable.
- Gentle to the skin.

Intende use

- Ideally for the fixation of any type of medication, catheters and cannulas.

articolo **item no.** ♀ misure **size** ♀ conf. **pkg.**

715001	cm 5 x m 10	1 pz/pcs
715002	cm 10 x m 10	1 pz/pcs
715003	cm 15 x m 10	1 pz/pcs
715004	cm 20 x m 10	1 pz/pcs
715006	cm 30 x m 10	1 pz/pcs

latex free

conformabile
conformable

traspirante
breathable

pelli delicate
delicate skin

Eurofarm Spa Zona Industriale di Piano Tavola, 95032 Belpasso (CT) Italy
phone +39 095 391346 • fax +39 095 391507 eurofarm-spa.com

 **eurofarm**[®]
ADVANCED MEDICAL DEVICES

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Transparent polyurethane film roll, very flexible and impermeable to liquids and bacteria

Euroderm roll is an adhesive, transparent, waterproof and highly breathable Polyurethane film roll. Very flexible and conformable to irregular parts of the body, easy to cut in any required size.

Euroderm roll is impermeable to liquids and bacteria.

Designed for an easy fixation of primary dressings, it allows the skin to breath.

The pre-cut silicone paper provides protection to the adhesive side and facilitates removal. Euroderm roll is latex free.

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Benefits

- Easy to remove to avoid skin trauma. It does not release residues of glue.
- Reduces the risk of secondary infections by microorganisms.
- Vapour permeable.
- Impermeable to liquids and bacteria.
- Adaptable and flexible to irregular and difficult anatomical part of the body.
- The transparent support allows continuous monitoring of the wound, limiting the need of dressing change.
- Ensures optimal adhesion to all types of primary dressings.
- Highly breathable.
- Waterproof. It offers the patient the opportunity to perform regular and daily hygiene activities.

Intende use

- Primary dressing's fixation.
- Catheters and cannulas fixation.
- It grants the ideal skin protection when using drainage bags and ostomy bags.

Contraindications

- Euroderm roll should not be used directly on injured skin.

articolo	item no.	♀ misure	size	♂ conf.	pkg.
----------	----------	----------	------	---------	------

715600		cm 2,5 x m 10			408 pz/pcs
715601		cm 5 x m 10			204 pz/pcs
715602		cm 10 x m 10			102 pz/pcs
715603		cm 15 x m 10			72 pz/pcs
715604		cm 20 x m 10			52 pz/pcs

latex free

conformabile
conformable

traspirante
breathable

Eurofarm Spa Zona Industriale di Piano Tavola, 95032 Belpasso (CT) Italy
phone +39 095 391346 • fax +39 095 391507 eurofarm-spa.com

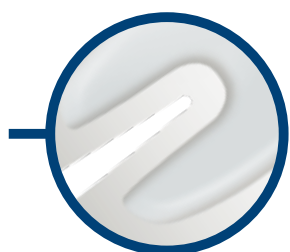
 **eurofarm**[®]
ADVANCED MEDICAL DEVICES

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Transparent sterile adhesive dressing made of a highly resistant thin polyurethane film

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Polyurethane transparent film dressing for cannula fixation

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Highly flexible and resistant polyurethane film reinforced with a non-woven polyester fabric support

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Transparent sterile adhesive dressing made of a highly resistant thin polyurethane film. Stretchable, hypoallergenic and radiotranslucent. Euroderm is Water and bacteria proof but permeable to water vapour and oxygen, guarantees an excellent moisture vapour transmission rate, creating an ideal environment for wound healing and reduces the risk of skin maceration. It is easy to remove in a non-traumatic manner; it does not release residues of adhesive on the skin. The high conformability of the support enables an easy application on joints and difficult body contours. Its transparency allows continuous monitoring of the wound without having to remove the dressing. Euroderm's waterproof polyurethane support allows patients the opportunity to do regular shower and bath. Euroderm can be left in place for periods greater than average dressing. Euroderm is latex free.

Intended use: To be used for the protection of insertion site for venous and arterial catheter or other percutaneous devices. I and II degree burns. Protect the skin at risk of decubitus or subject to redness sites for prolonged immobilization. Secondary fixation of other dressings.

articolo	item no.	♀ misure	size	♀ conf.	pkg.
----------	----------	----------	------	---------	------

717505		cm 4 x cm 5		100 pz/pcs	
717507		cm 6 x cm 7		100 pz/pcs	
717510		cm 10 x cm 10		50 pz/pcs	
717512		cm 10 x cm 12		50 pz/pcs	
717520		cm 15 x cm 20		10 pz/pcs	
717521		cm 20 x cm 25		100 pz/pcs	
717522		cm 20 x cm 30		100 pz/pcs	
717525		cm 10 x cm 25		10 pz/pcs	

latex free

barriera contro batteri, impermeabile
acts as bacteria barrier and waterproof

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Euroderm I.V. is a polyurethane transparent film dressing for cannula fixation. Water and bacteria proof but permeable to water vapour and oxygen, guarantees an excellent moisture vapour transmission rate reduces the risk of skin maceration. Its transparency enables the observation of the puncture site without having to remove it. Highly flexible and resistant polyurethane film provides greater comfort to patient. The longitudinal cut makes the application easy for the fixation and protection of the insertion of cannula and venous catheters. Waterproof polyurethane support allows patient to perform regular hygiene activities. The dressing can be removed in a non-traumatic manner and without the release of adhesive residues. Euroderm I.V. is latex free.

articolo	item no.	♀ misure size	♀ conf. pkg.
717550		cm 6 x cm 7	100 pz/pcs
717552		cm 8 x cm 9	100 pz/pcs
717560		cm 10 x cm 12	50 pz/pcs

traspirante
breathable

dermatologicamente testato
dermatologically tested

barriera contro batteri, impermeabile
acts as bacteria barrier and waterproof

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Highly flexible and resistant polyurethane film, reinforced with a non-woven polyester fabric support for extra strength ensuring reliable fixation for intravascular devices and cannulas. The reinforced non-woven polyester fabric provides an excellent fastening by distributing the power generated by the central and peripheral catheters from infusion sets and tubes on a highly resistant surface, thereby reducing the risk of outflow of devices.

Water and bacteria proof but permeable to water vapour and oxygen, guarantees an excellent moisture vapour transmission rate and reduces the risk of skin maceration. Being transparent, Euroderm I.V. R makes monitoring of the insertion site easier without having to remove the dressing. Euroderm I.V. R comes with 2 fastening strips useful for the transcription of the insertion date and to increase the anchorage of the device.

- Reliable and secure fixation for intravascular devices.
- Transparent for easy monitoring of insertion site.
- Excellent moisture vapour transmission rate lets skin moisture to evaporate.
- Easy to apply even when using gloves.

Euroderm I.V. R is latex free.

articolo	item no.	misure size	conf. pkg.
717570		cm 5 x cm 5,7	100 pz/pcs
717571		cm 8 x cm 9	100 pz/pcs
717572		cm 8,5 x cm 11,5	100 pz/pcs
717573		cm 10 x cm 12	100 pz/pcs

latex free

barriera contro batteri, impermeabile
acts as bacteria barrier and waterproof

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Post-operative island dressing in transparent
polyurethane film

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Sterile kit intended for a safe and functional
management of insertion sites

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Euroderm Plus is a post-operative island dressing in transparent polyurethane film with absorbent central pad. The absorbent and non-adherent pad absorbs exudates, minimizing the risk of skin maceration, and allows a painless removal of the dressing. The film transparency enables the observation of the exudates in the pad without having to remove the dressing. The thin film is highly resistant but flexible. It conforms to all body contours and guarantees an excellent moisture vapour transmission rate, creating an ideal environment for wound healing. Moreover, the film protects the wound from bacteria and virus contaminations.

Euroderm Plus is latex free.

articolo	item no.	♀ misure size	♀ conf. pkg.
717600		cm 5 x cm 7	100 pz/pcs
717601		cm 6 x cm 7	100 pz/pcs
717602		cm 6 x cm 9	50 pz/pcs
717603		cm 10 x cm 10	50 pz/pcs
717604		cm 12 x cm 7	50 pz/pcs
717605		cm 15 x cm 8	50 pz/pcs
717606		cm 10 x cm 20	50 pz/pcs
717607		cm 10 x cm 25	50 pz/pcs
717608		cm 10 x cm 30	50 pz/pcs
717609		cm 10 x cm 35	50 pz/pcs

latex free

bassa essudazione
low exudate

barriera contro batteri, impermeabile
acts as bacteria barrier and waterproof

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Euroderm I.V. Plus is a sterile kit intended for a safe and functional management of insertion sites. It is composed by a sterile polyurethane film dressing with an absorbent central pad, an extra pad antidecubitus and a stripe in polyester net for the fixation of tubing. The lengthwise cut allow a safe position of cannula and catheters. Euroderm I.V. Plus it is extensible, transparent, impermeable to liquids and bacteria but permeable to vapour, radiotranslucent, hypoallergenic, easy to remove from the skin in a non-traumatic manner. It does not release adhesive residues on the skin. The highly absorbent and non-adherent pad retaining exudate minimizing the risk of skin maceration. The pressure sore prevention pad can be placed under the tube, for greater patient comfort. The adhesive stripe in polyester non-woven fabric allows the fastening of the infusion set, in order to ensure greater safety against premature release of the needle. The dressing maintains an ideal moist wound microenvironment and promote healing. The polyurethane and transparent IV dressing allows keeping the insertion site visible and controlling the microenvironment of the wound.

Intended use: to be used for fixing cannulas and catheters. Euroderm I.V. Plus is latex free.

articolo **item no.** misure **size** conf. **pkg.**

717620 cm 9 x cm 8 100 pz/pcs

latex free

traspirante
breathable

dermatologicamente testato
dermatologically tested

barriera contro batteri, impermeabile
acts as bacteria barrier and waterproof

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Description

- Radiotranslucent and transparent protective wound dressing for light to moderately secreting wounds.
- Made of polyurethane bacteria proof, virus proof and waterproof film coated with hypoallergenic adhesive.
- The polyurethane film backing paper is kiss cut into an outer frame and central window for easy application.
- Ethylene oxide gas sterilized, supplied with a central non-adherent pad.

Benefits

- Maintains the undisturbed state of the wound.
- Very easy to apply.
- Provides moist wound environment to enhance healing and decrease pain.
- Patient can shower without risk.
- PU Frame style allows wounds to breathe and water vapour to escape protecting skin against maceration where dressing can be left in place for 5 days minimizing interference at wound site.

Indication

- Wound dressing for primary and secondary care of light to moderately secreting wounds.
- Fixation of I.V. catheters.
- Antibacterial and waterproof coverage of wounds.
- Post-operative dressings to protect clean closed wounds.

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Adhesive continuous length plaster with an Aloe Vera absorbent central pad.

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Polyethylene plasters with an Aloe Vera absorbent central pad.

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Adhesive continuous length plaster with an Aloe Vera absorbent central pad. Suitable for sensitive skins. Any minor wound needs a suitable plaster to help the healing process. Aloe Vera pad technology promotes healing and is capable of performing an exceptional epithelizing and antiseptic action.

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Polyethylene plasters with an Aloe Vera absorbent central pad. Suitable for sensitive skins. Any minor wound needs a suitable plaster to help the healing process. Aloe Vera pad technology promote healing and is capable of performing an exceptional epithelizing and antiseptic action. It is indicated for for first aid, medication, wellness, for the cure of small wounds and for fingers or other parts of the body.

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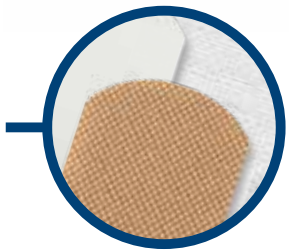
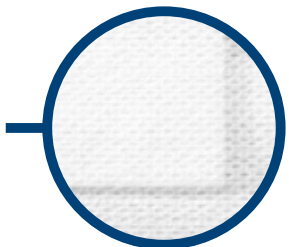
First aid adhesive plaster with central absorbent pad

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Dispenser with pre-cut first-aid strips in skin coloured non-woven fabric, easy to cut off

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Haemostatic adhesive strip with super-absorbent central pad

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Eurostrips is a vast family of first aid adhesive plasters with a central absorbent pad.

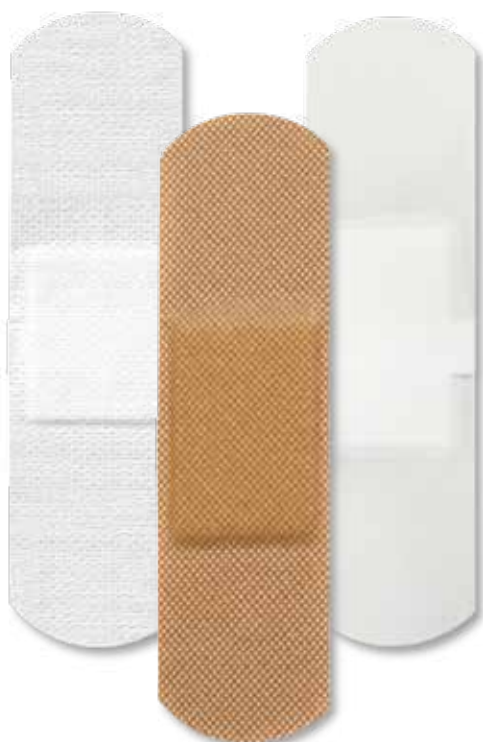
Sterile and non-sterile version are available.

Eurostrips are available different sizes and formats:

- Skin coloured Polyethylene.
- Transparent Polyethylene film.
- White non-woven Polyester Fabric.
- Skin colour non-woven paper fabric.

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Dispenser with 250 pre-cut first-aid strips in skin coloured non-woven fabric,
easy to cut off thanks to its pre-cut format.

Offer secure guarantees of hypoallergenicity avoiding skin irritation.

Non-adherent and absorbent central pad that does not stick to the wound thanks to the polyethylene
outer net layer allowing painless removal.

Eurostrips injection are skin friendly, radiotranslucent and air permeable.
Eurostrips injection are latex free.

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Eurostrips Emo is a family of haemostatic strips available in several sizes and formats.

Sterile packaging and not sterile are both available.

The central pad is composed by super-absorbent fibres that rapidly increase its volume immediately after blood absorption and should therefore effect a compressive action on the wound with fast stop of the bleeding.

articolo	item no.	♀ misure	size	♂ conf.	pkg.
724281		mm 19 x mm 72	(in PE)	100 pz/pcs	
724282		mm 25 x mm 72	(in PE)	100 pz/pcs	
724283		mm 31 x mm 72	(in PE)	100 pz/pcs	
724271		mm 40 x mm 40	(in PE)	100 pz/pcs	
731284		mm 40 x mm 40	(in Poliestere retinato)	100 pz/pcs	

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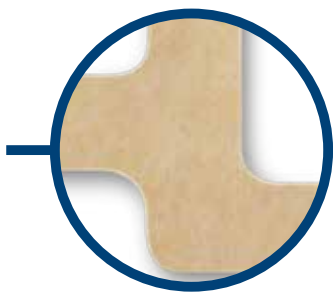
Nasogastric plasters for nose tube fixing

Enteralfix is a skin colour, non-woven nasogastric plasters for the fixation of nasogastric tubes for enteral supply.

The high elasticity allows easy application and high conformability.

Enteralfix has excellent adhesion properties resulting in long lasting wear time without causing pain on removal, from even the most delicate skin.

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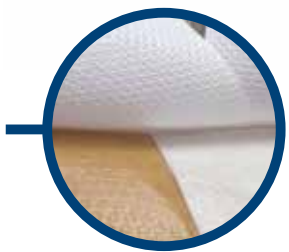
Intended use:

Fixation of nasogastric tubes for enteral supply.

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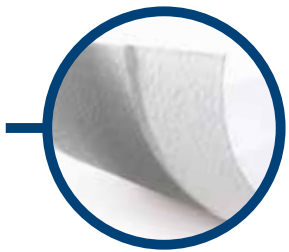
Long bend is a continuous length adhesive plaster with a central, non-adherent and highly absorbent wound pad. The excellent quality of the special fibers of viscose constituting the absorbent pad reduces the adherence of the dressing to the wound and permits a rapid absorption. Long bend is available in non-woven skin color with polyacrylic adhesive mass. 100% Rigid cotton in skin color with zinc oxide adhesive mass. Elastic cotton/polyamide with zinc oxide adhesive mass. Long bend is skin friendly and air permeable. Can be cut in one's favourite size. Long bend is available in a wide range of sizes. Long bend is latex free.

articolo item no.			misure size	conf.pkg.
non tessuto non woven	tessuto color pelle skin-colour fabric	tessuto elastico elastic fabric		
736001	738001	741001	4 cm x 0,5 m	36 pz/pcs
736002	738002	741002	6 cm x 0,5 m	36 pz/pcs
736003	738003	741003	8 cm x 0,5 m	36 pz/pcs
736004	738004	741004	4 cm x 1 m	36 pz/pcs
736005	738005	741005	6 cm x 1 m	36 pz/pcs
736006	738006	741006	8 cm x 1 m	36 pz/pcs
736007	738007	741007	4 cm x 5 m	132 pz/pcs
736008	738008	741008	6 cm x 5 m	90 pz/pcs
736009	738009	741009	8 cm x 5 m	72 pz/pcs

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Long-Fix is a continuous length adhesive plaster in white non-woven polyester net with a central, non-adherent and highly absorbent wound-pad.

Longfix is skin friendly and air permeable, allows a comfortable fit of the plaster to the skin and to the other particularly irregular anatomical parts of the body, ensuring the highest moisture permeability.

The plaster can be cut in one's favourite size.

Long Fix is suitable as barrier for minor wounds and general abrasions.

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Long-Fix Metal is a continuous length adhesive plaster in white non-woven with a central, non-adherent and highly absorbent wound-pad. The excellent quality of the aluminized layer in the pad reduces the adherence of the dressing to the wound, and permits good skin transpiration, a rapid absorption of blood and exudates into the dressing and consequently keeps dry the wound surface.

Longfix Metal, skin friendly and air permeable, it allows a comfortable fit of the plaster to the skin and to the other particularly irregular anatomical parts of the body, ensuring the highest moisture permeability.

The plaster, which can be cut in one's favourite size acts as bacteria barrier for minor wounds and general abrasions. Promotes healing.

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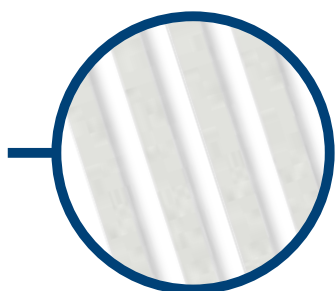
Sterile adhesive suture strips

Eurosuture are adhesive sterile strips for skin sutures, resisting to tensile stresses, conformable, made of reinforced non-woven fabric in polyester and polyamide. It is non-toxic highly breathable and hypoallergenic, excluding skin irritation.

They can be used to replace traditional suture, on all linear wounds subjected to low strain of the skin, thus, reduce the risk of infection, eliminate the unaesthetic signs of the suture punctures and minimizing the formation of scars.

Due to the unique rounded edges of the strip, Eurosuture will not curl or form crinkles avoiding premature detachment from the wound edges. Placed on a support paper for an easy application.

Eurosuture is latex free.

Cerotto sterile per suture cutanee Sterile adhesive suture strips

Eurosuture

articolo	item no.	♀ misure	size	♀ conf.	pkg.
717410		mm 3 x mm 75	(5 strips)	50 card	(250 strips)
717420		mm 6 x mm 38	(6 strips)	50 card	(300 strips)
717430		mm 6 x mm 75	(3 strips)	50 card	(150 strips)
717440		mm 6 x mm 100	(10 strips)	50 card	(500 strips)
717450		mm 13 x mm 100	(6 strips)	50 card	(300 strips)
717460		mm 25 x mm 100	(4 strips)	50 card	(200 strips)

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Property and benefits

- Breathable, porous, can be used on contoured body areas or where bending and movement occur.
- Faded colour for a cosmetically pleasing appearance.
- Eliminates puncture marks for better cosmetic result than sutures or staples.
- Fast and easy to apply to save time.
- Uses a hypoallergenic adhesive.
- Available in a variety of sizes to meet your needs.
- Rounded edges help prevent roll-up, curls or crinkles. Available in sealed pouches E.O sterilized.

Uses

- For use in highly contoured areas or body areas of musculoskeletal movements like joints.
- Providing wound support following early suture or staple removal.
- Useful for wound support post operatively and following suture removal.

For use in

- Emergency Rooms.
- Operating Rooms.
- Clinics and physician's offices.

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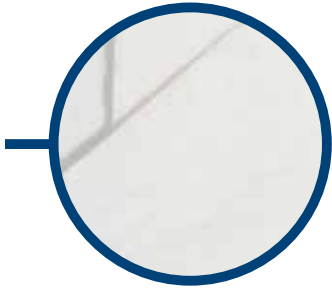
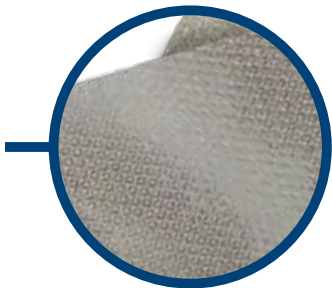
Eurodressing is a sterile and hypoallergenic absorbent non-woven dressing

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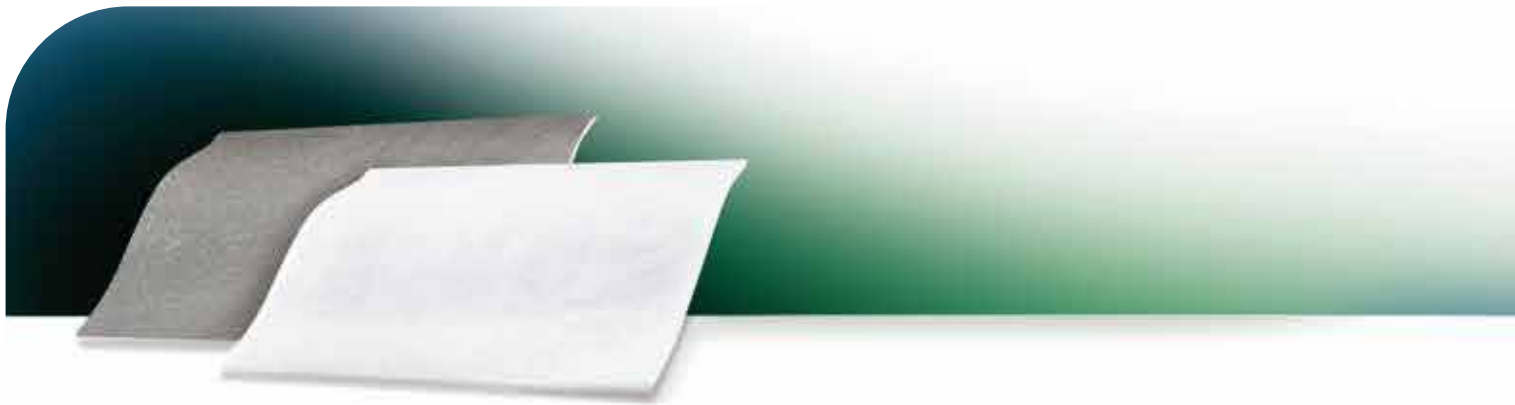
Sterile, hypoallergenic, aluminised and primary dressing with highly absorbent viscose fibre central layer

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Eurodressing is a sterile and hypoallergenic absorbent non-woven dressing.

Eurodressing is composed by 3 layers:

- 100% Polyethylene external layer applied on both sides: non-adherent open net for a fast absorption.
- 100% Viscose absorbent in the middle.
- Particularly used in surgery for the treatment of wounds, abrasions, ulcers as well as bedsores, and in every case of special medications instead of gauze.
- The dressing facilitates the absorption of blood and secretions, and it does not require a particular care when replaced, whereas the standard gauze requires a slow and gradual removal since they stick to the wound.
- Eurodressing is latex free.

articolo	item no.	♀	misure size	♀	conf. pkg.
----------	----------	---	-------------	---	------------

722001			cm 5 x cm 5		100 pz/pcs
--------	--	--	-------------	--	------------

722002			cm 5 x cm 7,5		100 pz/pcs
--------	--	--	---------------	--	------------

722003			cm 10 x cm 10		50 pz/pcs
--------	--	--	---------------	--	-----------

722004			cm 10 x cm 20		50 pz/pcs
--------	--	--	---------------	--	-----------

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Sterile, hypoallergenic, aluminised and primary dressing with highly absorbent viscose fibre central layer
Eurodressing Metal is a sterile primary dressing composed by 3 layers:

- 100% Polyethylene aluminised wound contact layer, non-adherent open net for a fast absorption.
- 100 % Polyethylene external layer applied.
- 100% Viscose absorbent fibres in the middle.
- Eurodressing metal does not stick to the wound thanks to a particular non-adherent layer. The aluminised central pad enables a rapid healing process and acts as bacteria barrier.
- Eurodressing metal is hypoallergenic, radiotranslucent, air-permeable and allows the transpiration of skin with a painless removal.
- To be used as mechanical barrier against bacteria and to control the microenvironment of the wound, ulcer of the low parts of the body, burns and skin abrasions in general.
- Eurodressing metal is supplied in a wide range of sizes with rounded edges, available in single envelope and ready to use.
- Individually packed in a peel away pouch and sterilized by E.O. gas
- Eurodressing metal is Latex Free.

articolo	item no.	♀ misure size	♀ conf. pkg.
722300		cm 5 x cm 5	100 pz/pcs
722301		cm 5 x cm 7,5	100 pz/pcs
722302		cm 10 x cm 8	50 pz/pcs
722303		cm 10 x cm 10	50 pz/pcs
722304		cm 10 x cm 12	50 pz/pcs
722305		cm 10 x cm 20	50 pz/pcs

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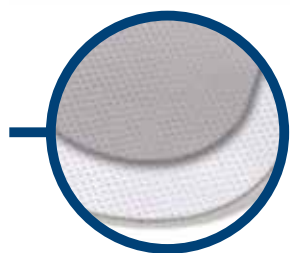
Non-woven sterile eye plaster, ready for use,
with an occlusive pad

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Non-woven sterile eye plaster with a white,
absorbent, non-occlusive pad

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Sterile and non-adhesive anatomic eye pad

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Europtical is a non-woven sterile eye plaster, ready for use, with an occlusive pad, ideal for the correction of strabismus and for the prophylaxis and treatment of amblyopia. The highly porous non-woven backing material preserves the physiological functions of the skin and reduces the risk of maceration. Besides, the high permeability both to air and to water vapour avoids heat congestion in the eye socket. Its anatomical shape enables easy and accurate applications, even when worn beneath glasses. The pad in the middle of the adhesive area is composed by especial cellulose fibres and from a black occlusive pad making Europtical Occlusivo impenetrable to light and allowing an appropriate orthoptist therapy. Highly tolerable by the patient without sacrificing the aesthetic element. Europtical occlusive is latex free.

articolo	item no.	♀	misure	size	♀	conf.	pkg.
----------	----------	---	--------	------	---	-------	------

717200			cm 5,2 x cm 6,4			50 pz/pcs	
--------	--	--	-----------------	--	--	-----------	--

717201			cm 5,7 x cm 8,2			50 pz/pcs	
--------	--	--	-----------------	--	--	-----------	--

717202			cm 8,5 x cm 11			50 pz/pcs	
--------	--	--	----------------	--	--	-----------	--

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Non-woven sterile eye plaster with a white, absorbent, non-occlusive pad. It is ideal for the post-operative protection of the eye.

The highly porous non-woven backing material preserves the physiological functions of the skin and reduces the risk of maceration. Besides, the high permeability both to air and to water vapour avoids heat congestion in the eye socket. Its anatomical shape enables easy and accurate applications, even when worn beneath glasses. It ensures atraumatic removal.

Euoptical non-occlusive is latex free.

articolo	item no.	♀ misure	size	♂ conf.	pkg.
----------	----------	----------	------	---------	------

717210		cm 5,2 x cm 6,4		50 pz/pcs	
--------	--	-----------------	--	-----------	--

717211		cm 5,7 x cm 8,2		50 pz/pcs	
--------	--	-----------------	--	-----------	--

717212		cm 8,5 x cm 11		50 pz/pcs	
--------	--	----------------	--	-----------	--

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Sterile and non-adhesive eye pad, particularly suitable for the prophylaxis and eye treatment. The anatomical shape allows an easy and correct application. The pad, made of special cellulose fibres, offers protection, high absorption and comfort for the patient. Prevents the formation sweating in the wound allowing skin perspiration. The edge is welded along the whole circumference of the pad, thus avoiding the leakage of exudate. Tampone oculare is indicated for the eye dressing for post-operative interventions, for corneal wounds or medication of all kinds. Europtical tampone oculare is latex free.

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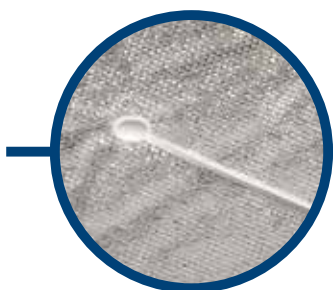
Aluminised non-woven dressing especially designed for use around tracheotomies

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Aluminised non-woven dressing especially designed for infusion catheters

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Tracheopad is an aluminised non-woven dressing, especially designed for use around tracheotomies, nephrostomies and large calibre drains. The pre-cut and minor slits radiating from the central hole (Ø from 12 mm to 20 mm) allows a perfect fit around the tracheotomy tube assuring the maximum comfort to the patient. The excellent quality of the aluminised wound contact layer performs a healing function and it reduces the adherence of the dressing either to the peristome skin or to the tracheotomy tube, minimizing trauma for the patient during the removal. Furthermore, the high porosity of this layer permits good skin transpiration, allows a rapid absorption of blood and exudates into the dressing and consequently keeps the wound surface dry. Tracheopad acts as a reliable barrier against bacterial contamination. Tracheopad is latex free.

articolo	item no.	♀ misure	size	♀ conf.	pkg.
717310		cm 8 x cm 9,3		100	pz/pcs
717320		cm 8 x cm 9,3		50	pz/pcs

latex free

barriera contro batteri, impermeabile
acts as bacteria barrier and waterproof

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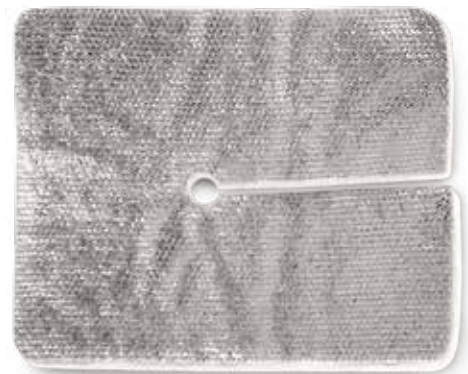
Drenapad is a non-woven aluminised dressing especially designed for use on wounds produced by small and medium calibre drains and infusion catheters, or around suture. The pre-cut and minor slits radiating from the central hole (Ø from 5 mm to 12 mm) allow a perfect fit around the drain assuring the maximum comfort to the patient. The excellent quality of the aluminised wound contact layer reduces the adherence of the dressing either to the peristome skin or to the drain. It minimises the trauma for the patient during the removal. Furthermore, the high porosity of this layer permits good skin transpiration, allows a rapid absorption of blood and exudates into the dressing and consequently keeps the wound surface dry. Drenapad acts as a reliable barrier against bacterial contamination. Drenapad is latex free.

articolo item no.	♀ misure size	♀ conf. pkg.
717300	cm 6 x cm 7	50 pz/pkg

bassa essudazione
low exudate

barriera contro batteri, impermeabile
acts as bacteria barrier and waterproof

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Features and benefits

- Permeable to air and water vapour.
- Good drainage.
- Soft and adaptable.
- Ensures patient comfort.
- Rapid absorption of exudate.
- Easy to apply and remove.
- Does not stick to the wound, facilitating atraumatic removal of the dressing.

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Rigid strapping zinc oxide tape for functional and sports bandaging

Rigid strapping zinc oxide tape for functional and sports bandaging in 100% viscose with hypoallergenic zinc oxide adhesive. It provides strong and efficient stabilisation of joint structures. It is hand tearable for quick and easy application. Indicated for functional bandaging of muscles ligaments and joints injuries. Euroathletic sport tape is latex free.

Cerotto per bendaggi funzionali
Rigid strapping zinc oxide tape for
functional and sports bandaging

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Property and advantages

- Water repellent.
- Strong adhesion for longer application of the bandage under high stress conditions.
- Easy to apply.
- Easy to tear by hand in both directions.

Intended use

- Functional bandaging to reduce the movements of the muscles, ligaments and joints.
- Injury ligaments and joints.
- Support bandaging in case of sprains, dislocations or joint injuries.
- For orthopaedic immobilization and for the fixation of splint plasters.
- Commonly used for taping in sports medicine.

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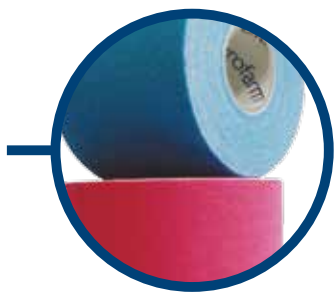
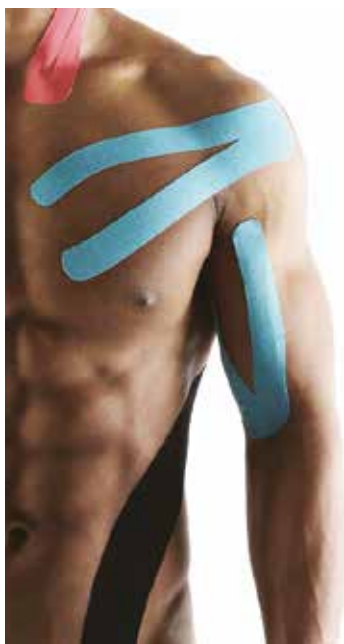
Elastic adhesive tape for the treatment and natural prevention of trauma as well as joint and muscle inflammation

Kinesiology taping is used by sports doctors and physiotherapists to treat muscle and joint pain and similar disorders. Muscles stretch and contract continuously within certain limits. However, when these limits are exceeded, the muscles under stress become inflamed. When a muscle is inflamed, swollen or stiff because fatigued, the space between the skin and the muscle is "compressed" and the result is a restriction on the movement of lymphatic fluid. This compression involves even the pressure on pain receptors located under the skin, resulting in pain. This type of pain is called "myalgia" or simply "muscle pain". When properly applied, Euroathletic kls for kinesiology taping relieves pain and facilitates lymphatic drainage through an action on the skin that increases the interstitial space, reducing the pressure on lymph flow and pain receptors, due to the formation of particular convolutions on the skin.

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Relieves pain • Corrects muscle and joint function • Promotes lymphatic circulation
• Helps rehabilitation • Heals small subcutaneous edemas and hematomas

ADVANTAGES: Euroathletic kls for kinesiology taping adapts well to the contours of the body and does not hinder the movements of the joints, thus ensuring freedom of movement. It is radiolucent, dermatophyte, highly hypoallergenic, permeable to air and water vapor, due its special wave coating. It contains no drugs. It promotes lymphatic drainage, facilitates joint realignment and normalizes muscle functions. Can also be used in the pool, as it is water resistant.

WARNINGS: Disposable • Do not use on injured skin • The patch may cause a slight reddening of the area at the time of removal • In case of intolerance to the patch, remove it and wash the application area.

HOW TO USE: To be used under the guidance of a doctor specializing in sports medicine or a physiotherapist who know how to apply it.

FREQUENCY OF REPLACEMENT: It can remain in situ for several days from the time of application, given the high permeability and total freedom of movement. The frequency of replacement should be established based on the opinion of the specialist.

articolo	item no.	♀ misure size	♀ conf. pkg.
520305	Pelle/Skin	cm 5 x m 5	1 pz/pcs
520325	Blu/Blue	cm 5 x m 5	1 pz/pcs
520335	Nero/Black	cm 5 x m 5	1 pz/pcs
520355	Rosa/Pink	cm 5 x m 5	1 pz/pcs

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Adhesive bandage of durable elasticity
with zinc oxide

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Cohesive elastic bandage of durable elasticity
with and without latex

Eurobandage

Benda elastica ad uso
universale con ganci

Elastic bandage for universal
use with clips

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Lengthwise elastic adhesive bandage of medium stretchability. Made of 100% cotton with zinc oxide hypoallergenic adhesive paste, porously coated in order to guarantee maximum breathability and skin tolerability.

Off-white (with red middle-thread). Soft and smooth uncoated edges to avoid skin cut. Particularly suitable for permanent compression for the treatment of complex varicose and thrombosis prophylaxis. Also used as functional dressings in the therapy of muscle and tendon injuries, sprains and dislocations.

Eurelasto is also available in:

Eurelasto H (hospital) version (82% cotton and 12% polyamide) characterized by a greater elasticity.

Eurelasto F plaster bandage with extra strong hypoallergenic, synthetic adhesive containing zinc oxide, for permanent compression and functional dressings, particularly suitable for contact sport, mixed martial arts, rugby among others.

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Cohesive bandage of durable elasticity with a short stretch for fixation, support and relief dressings. Euroelasto L comes impregnated with latex emulsion, stretchable in the longitudinal direction, without adhesive, capable of fixating on itself without the use of clips. It does not stick on the hair or the skin. Highly hypoallergenic, resistant to sliding, porous, permeable to air and breathable avoiding the risk of maceration and skin irritation. Particularly used for fastenings were applications in difficult parts of the body (head, hands, arms, legs and ankles) is required. Also used for post-traumatic compression dressing, under compression bandage or as post-medication dressing. Its ventilated structure ensures an optimal thermal and moisture exchange. **Eurelasto L is also available in latex free version, Eurelasto LF.**

	articolo	item no.	misure in tensione	size	conf. pkg.
Eurelasto L  conformabile conformable	516002		cm 2,5 x m 4		12 pz/pcs
	516004		cm 4 x m 4		12 pz/pcs
	516006		cm 6 x m 4		12 pz/pcs
	516008		cm 8 x m 4		12 pz/pcs
	516010		cm 10 x m 4		12 pz/pcs
	516012		cm 12 x m 4		12 pz/pcs
	516104		cm 4 x m 20		1 pz/pcs
	516106		cm 6 x m 20		1 pz/pcs
	516108		cm 8 x m 20		1 pz/pcs
	516110		cm 10 x m 20		1 pz/pcs
	516112		cm 12 x m 20		1 pz/pcs
	Eurelasto LF  latex free	516202		cm 2,5 x m 4	
516204			cm 4 x m 4		10 pz/pcs
516206			cm 6 x m 4		10 pz/pcs
516208			cm 8 x m 4		10 pz/pcs
516210			cm 10 x m 4		10 pz/pcs
516212			cm 12 x m 4		10 pz/pcs
516304			cm 4 x m 20		1 pz/pcs
516306			cm 6 x m 20		1 pz/pcs
516308			cm 8 x m 20		1 pz/pcs
516310			cm 10 x m 20		1 pz/pcs
516312			cm 12 x m 20		1 pz/pcs

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Polyester and cotton elastic bandage for universal use in white or skin colour, hypoallergenic and breathable. Does not slip after application. It is reusable and resistant to repeated washings and tractions.

To use as application of dressings and skin protection, as a functional bandage in the treatment of muscle injuries and tendon, sprains, bruises, and for the treatment where moderate compression required.

Soft texture that guarantees skin transpiration. It does not causes skin irritation. Supplied with two clips.

Resistant to oils and ointments. Eurobandage is latex free.

articolo	item no.	misure in tensione	size	conf. pkg.
	bianco white		rosa pink	
519005	519105	cm 5 x m 4,5		12 pz/pcs
519007	519107	cm 7,5 x m 4,5		12 pz/pcs
519010	519110	cm 10 x m 4,5		12 pz/pcs
519012		cm 12 x m 4,5		12 pz/pcs
519015	519115	cm 15 x m 4,5		12 pz/pcs
519020	519120	cm 20 x m 4,5		12 pz/pcs

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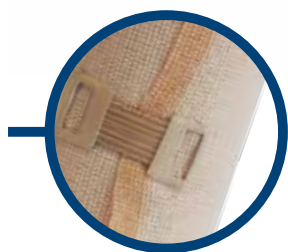
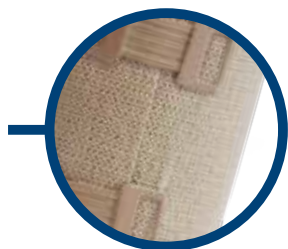
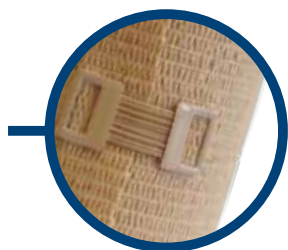
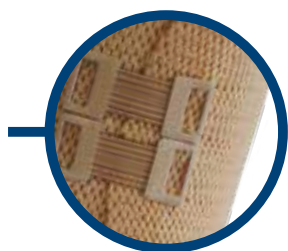
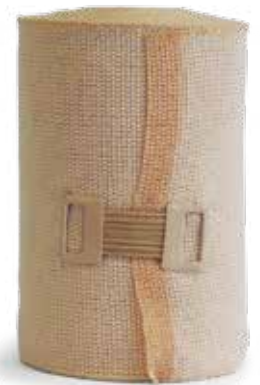
Non-adhesive elastic compression bandage with **short stretch (<40%)** in cotton and polyamide, skin coloured, non-allergenic and breathable, can be re-used, resists repeated stretching and washing, sterilisable. With fasteners. Non-deformable and non-tear, it unrolls easily without creasing. The soft weave fabric guarantees breathability and does not cause skin irritation. Resistant to oils and ointments. Latex free.

Intended use: • For use as support and relief bandages for contusions, sprains, dislocations and in the case of tendon problems, to immobilise joints, to protect against haematoma in the case of muscle injuries.

• Also suitable for phlebologic use (oedema, lymphoedema, ulcus cruris, varicosis).

articolo	item no.	♀ misure size	♀ conf.pkg.
527004		mt 5 x cm 4	1 pz/pcs
527006		mt 5 x cm 6	1 pz/pcs
527008		mt 5 x cm 8	1 pz/pcs
527010		mt 5 x cm 10	1 pz/pcs
527012		mt 5 x cm 12	1 pz/pcs

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Non-adhesive elastic compression bandage with **medium stretch**. Made of pure cotton, skin coloured, non-allergenic and breathable, can be re-used, resists repeated stretching and washing, sterilisable. With fasteners. Easy to apply, it is non-deformable and non-tear. Allows perfect adhesion to the various anatomical forms. It unrolls easily without forming creases. The soft weave fabric guarantees breathability and does not cause skin irritation. Resistant to oils and ointments. Latex free.

Intended use: • To be used for treatment of chronic venous insufficiency, primary and secondary varicosis, leg ulcers, oedema, thrombophlebitis, phlebothrombosis, after vein surgery, after sclerotherapy.

articolo	item no.	♀ misure size	♀ conf.pkg.
527106		mt 5 x cm 6	1 pz/pcs
527108		mt 5 x cm 8	1 pz/pcs
527110		mt 5 x cm 10	1 pz/pcs
527112		mt 5 x cm 12	1 pz/pcs

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Non-adhesive single-elastic compression bandage with long stretch (170%)

in cotton and polyurethane, skin coloured, non-allergenic and breathable, can be re-used, resists repeated stretching and washing, sterilisable.

With fasteners. Easy to apply, it is permanently elastic, non-deformable, non-tear, unrolls easily without forming creases.

The soft weave fabric guarantees breathability and does not cause skin irritation. Resistant to oils and ointments. Latex free.

Intended use: • Support bandage with strong, constant compression, ideal for the treatment of varicosis in general.

- Can also be used for the prevention and treatment of sports and work injuries, and to support and sustain sprains, dislocations, contusions and light fractures.

articolo	item no.	♀ misure size	♀ conf. pkg.
----------	----------	---------------	--------------

527206		mt 7 x cm 6	1 pz/pcs
--------	--	-------------	----------

527208		mt 7 x cm 8	1 pz/pcs
--------	--	-------------	----------

527210		mt 7 x cm 10	1 pz/pcs
--------	--	--------------	----------

527212		mt 7 x cm 12	1 pz/pcs
--------	--	--------------	----------

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Non-adhesive dual-elastic compression bandage with long stretch both lengthwise and widthwise.

Made of cotton, polyamide and polyurethane, skin coloured, non-allergenic and breathable, can be re-used, resists repeated stretching and washing, sterilisable. With fasteners. Non deformable and

non-tear, it unrolls easily without creasing. The soft weave fabric guarantees breathability and does not cause skin irritation. Resistant to oils and ointments.

Intended use: • Support bandage giving strong, constant compression, for postsurgical treatment bandages, ideal for treatment of varicosis.

- Can also be used for the prevention and treatment of sports and work injuries, and to support and sustain sprains and dis locations. The fact that it extends in all directions makes it particularly suitable as a bandage for joints.

articolo	item no.	♀ misure size	♀ conf. pkg.
----------	----------	---------------	--------------

527306		mt 7 x cm 6	1 pz/pcs
--------	--	-------------	----------

527308		mt 7 x cm 8	1 pz/pcs
--------	--	-------------	----------

527310		mt 7 x cm 10	1 pz/pcs
--------	--	--------------	----------

527312		mt 7 x cm 12	1 pz/pcs
--------	--	--------------	----------

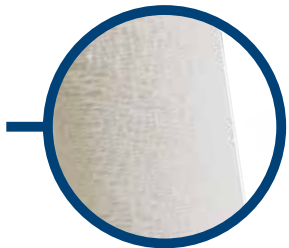
Eurofarm Spa Zona Industriale di Piano Tavola, 95032 Belpasso (CT) Italy
phone +39 095 391346 • fax +39 095 391507 eurofarm-spa.com

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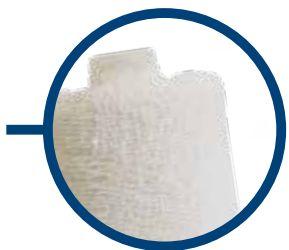
Eurozinc is a rigid medicated bandage ready to use. Impregnated with zinc oxide paste with extraordinary anti-inflammatory properties. It is able to guarantee a moist environment and will not dry during use.

Easy to apply and well tolerated by the skin. It has a skin refreshing effect after application.

Very easy to cast and adaptable to every type of body contour. Indicated for the treatment of ulcers of the lower limbs caused by venous insufficiency (varicose veins, post-phlebitis syndrome and venous edema) also commonly used for the treatment of distortions or as a support bandage.

It promotes the absorption of bruises, contusions and edema.

Each bandage is individually wrapped in a aluminium foil pouch that guarantees its wet condition over shelf life time. Eurozinc is latex free and cannot be sterilized.

articolo	item no.	♀ misure	size	♀ conf.	pkg.
507005		cm 10 x m 5		10 pz/pcs	
507007		cm 10 x m 7		10 pz/pcs	
507010		cm 10 x m 10		10 pz/pcs	
507106		cm 8 x m 6		10 pz/pcs	

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Eurozinc elast is an elastic medicated bandage ready to use. Impregnated with zinc oxide paste with extraordinary anti-inflammatory properties. It is able to guarantee a moist environment and will not dry during use.
Easy to apply and well tolerated by the skin. It has a skin refreshing effect after application.
Very easy to cast and adaptable to every type of body contour.

Indicated for the treatment of ulcers of the lower limbs caused by venous insufficiency (varicose veins, post-phlebitis syndrome and venous edema) also commonly used for the treatment of distortions or as a support bandage. It promotes the absorption of bruises, contusions and edema. Each bandage is individually wrapped in a aluminium foil pouch that guarantees its wet condition over shelf life time. Eurozinc elast is latex free and cannot be sterilized.

articolo	item no.	♀ misure	size	♂ conf.	pkg.
506005		cm 10 x m 5		10 pz/pcs	
506007		cm 10 x m 7		10 pz/pcs	
506010		cm 10 x m 10		10 pz/pcs	

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Eurozinc bielast is an elastic medicated bandage in both direction lengthwise and crosswise ready to use. Impregnated with zinc oxide paste with extraordinary anti-inflammatory properties. It is able to guarantee a moist environment and will not dry during use. Easy to apply and well tolerated by the skin. It has a skin refreshing effect after application. Very easy to cast and adaptable to every type of body contour.

Indicated for the treatment of ulcers of the lower limbs caused by venous insufficiency (varicose veins, post-phlebitis syndrome and venous edema) also commonly used for the treatment of distortions or as a support bandage. Each bandage is individually wrapped in a aluminium foil pouch, thus, guarantees its wet condition over shelf life time. Eurozinc bielast is latex free and cannot be sterilized.

articolo	item no.	♀ misure size	♀ conf. pkg.
506103		cm 6 x m 5	10 pz/pcs
506104		cm 8 x m 5	10 pz/pcs
506105		cm 10 x m 5	10 pz/pcs
506107		cm 10 x m 7	10 pz/pcs
506110		cm 10 x m 10	10 pz/pcs

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Eurozinc alginate is a semi-rigid bandage with zinc oxide paste and calcium alginate. coolant and supple. Useful for the treatment of the acute phase of joint injuries, muscle, ligaments and tendons. Very easy to cast and adaptable to every type of body contour. Eurozinc alginate is also indicated for the treatment of leg ulcers caused by venous insufficiency (varicose veins, post-phlebitis syndrome and venous edema) or as a support bandage. Eurozinc alginate is also indicated as first aid in sports injuries.

The hydrocolloidal properties ensures a refreshing effect for about 4 hours. Sodium alginate allows cooling effect also up to deep skin resulting in healing and quicker relief from pain. Once the bandage is dry, the cooling effect can be increased by conserving the bandage in a fridge. Each bandage is individually wrapped in a aluminium foil pouch, thus, guarantees its wet condition over shelf life time.
Eurozinc alginate is latex free and cannot be sterilized.

articolo	item no.	♀ misure size	♀ conf. pkg.
----------	----------	---------------	--------------

506035		cm 8 x m 5	10 pz/pcs
--------	--	------------	-----------

506025		cm 10 x m 5	10 pz/pcs
--------	--	-------------	-----------

506027		cm 10 x m 7	10 pz/pcs
--------	--	-------------	-----------

latex free

Eurofarm Spa Zona Industriale di Piano Tavola, 95032 Belpasso (CT) Italy
phone +39 095 391346 • fax +39 095 391507 eurofarm-spa.com

Eurocast f

Benda gessata con substrato
in fibra di vetro conformabile e flessibile
Fiberglass support intended for
use in the construction of most common
orthopaedic casts

Eurocast p

Benda gessata in poliestere
a presa rapida conformabile e flessibile
Polyester support intended
for use in the construction of most
common orthopaedic casts

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Eurocast F is an excellent fiberglass support intended for use in the construction of most common orthopaedic casts. Eurocast F is comprised of knitted fiberglass fabric impregnated with a water-activated polyurethane resin, providing superlative performance of molding for its casting applications.

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Eurocast P is an excellent Polyester support intended for use in the construction of most common orthopaedic casts. Eurocast P is comprised Eco-Friendly of knitted Polyester fabric impregnated with a water-activated polyurethane resin, providing superlative performance of moulding for casting applications.
Eurocast P is latex free.

articolo	item no.	♀ misure size	♀ conf. pkg.
----------	----------	---------------	--------------

511201		cm 2,5 x m 1,8	10 pz/pcs
511202		cm 5 x m 3,6	10 pz/pcs
511203		cm 7,5 x m 3,6	10 pz/pcs
511204		cm 10 x m 3,6	10 pz/pcs
511205		cm 12,5 x m 3,6	10 pz/pcs
511206		cm 15 x m 3,6	10 pz/pcs

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Fiberglass support intended for use orthopaedic

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Characteristic:

- High quality Fiberglass knitted fabric.
- Safe, clean, easy to apply.
- Low price guarantee.

Patient's benefits:

- Breathable.
- High strength yet lightweight.
- Durability.
- Good Air-Permeability, it prevents unpleasant smells.
- Prevents skin irritation.

User's benefits:

- Tack-free resin (Non-sticky to fingers)
- Secure end lamination.
- Radiotranslucent.
- Weight-bearing in 20 minutes.
- Easy to remove without producing dust.

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Polyester support intended for use orthopaedic

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Characteristic:

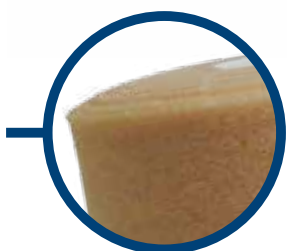
- Safe, clean, easy to apply.
- Low price guarantee.
- Eco-Friendly (Polyester).

Patient's benefits:

- Breathable
- High strength yet lightweight
- Durability
- Good Air-Permeability, it prevents unpleasant smells.
- Prevents skin irritation.

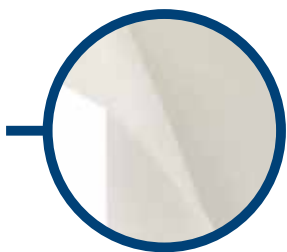
User's benefits:

- Tack-free resin (Non-sticky to fingers)
- Secure end lamination.
- Radiotranslucent
- Weight-bearing in 20 minutes.
- Easy to remove without producing dust.

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Underpadding material in thin polyurethane foam, elastic, soft, durable and highly porous.

It is commonly used under a rigid adhesive bandage to have painless removal.

It can be shaped according to the anatomy of the body and it adheres well to the skin without slippage and without causing inconstant thickness.

Eurosave, also is the best solution to ensure athletes comfort during competitions.

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Eurosoft is a undercast padding material made of 100% Polyester fibres. The construction of the bandage is designed to ease removal of the cast by limiting penetration of the cast material into the padding. It is also extra soft and highly conformable to deliver a smooth finish. It is hydrophobic in order to facilitate the rapid drying of the plaster, is not inactivated by water or by body fluids and continues to play its activities when exposed to liquids.

The high conformability ensures that the padding is perfect just with one overlay, thereby allowing the preparation of a perfect casts.

Some key characteristics of Eurosoft are: strong, yet easy to tear • high degree of self-adherence for convenient application • outstanding cushioning to protect vulnerable areas • compression resilience • it ensures patient comfort even after a long time, thanks to the consistency of its warp • radiotranslucent • Eurosoft can be sterilized by autoclave.

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Complete kit for skin traction of limbs. Composed by a transversely elastic adhesive fabric permeable to water vapour. A polyurethane foam padding to protect the ankle and foot. It uses a plate with central holes for the passage of the traction cables made of durable and abrasion-proof material and by a special elastic fixation bandage. Eurotrac is a ready for use kit. Thanks to the elastic adhesive fabric adheres perfectly to the leg without forming folds, avoiding movement and slipping.
Intended use: to be used for traction of the limbs, for skeletal alignment and for congenital hip dysplasia in infants.
Eurotrac is latex free.

articolo **item no.**  misure **size**

747101 Adulto	cm 8 x m 1,8
747102 Bambino	cm 6 x m 1,4

latex free

Collare cervicale
Cervical collar

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Cervical collar, the core is composed by the combination of elastic and soft polyurethane foam and a rigid layer in polyurethane foam creating the ideal density for a comfortable support. Covered in soft knitted 100% cotton, hypoallergenic, easy to remove and can be reapplied to allow washing thereby it enables prolonged use of the collar.

The front securing is constituted by two Velcro strips that allows a fast and precise application and removal.
Intended use: commonly used for cervical syndromes of different nature, such as support of the head when an uncontrolled movement of the cervical causes pain because of collision trauma (whiplash injury), stiff neck of rheumatic origin and sports injuries. Eurocollar is latex free.

articolo **item no.**  **misure size**

503002	Piccola
503003	Media
503004	Grande

lavabile
washable

latex free

Eurofarm Spa Zona Industriale di Piano Tavola, 95032 Belpasso (CT) Italy
phone +39 095 391346 • fax +39 095 391507 eurofarm-spa.com